




# Quick Start Guide Spitfire EX 3 & 4 Wheel


## Please Read Before Operating!

For technical assistance, call toll free (866) 621-4933 or (800) 544-5155.

- 1.** Remove your Spitfire Scooter from its packaging. Loosen the tiller adjustment knob.

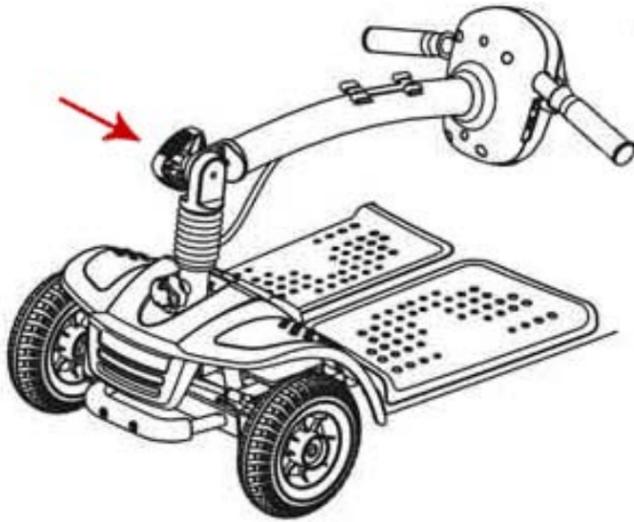


- 2.** Raise the tiller to the upright position at the desired angle and re-tighten the knob.

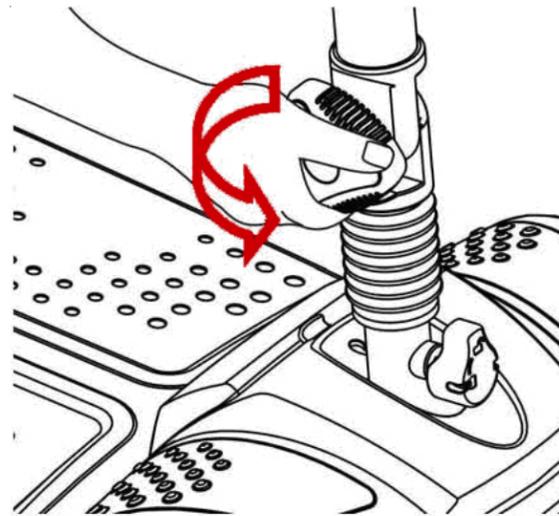


- 3.** Remove the battery pack by grasping the pack's handle and pulling upward firmly.



- 4.** Install the seat post into the seat tube in the scooter's frame to the desired height.

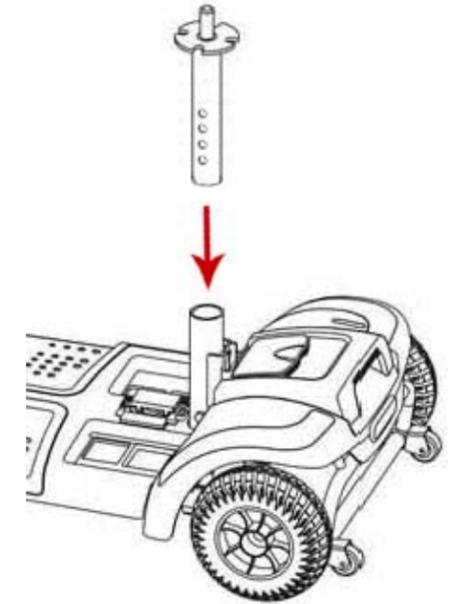

- 5.** Insert the seat post bolt into the hole at the base of the seat tube so that the bolt passes all the way through both the seat tube and the seat post to the hole on the opposite side.

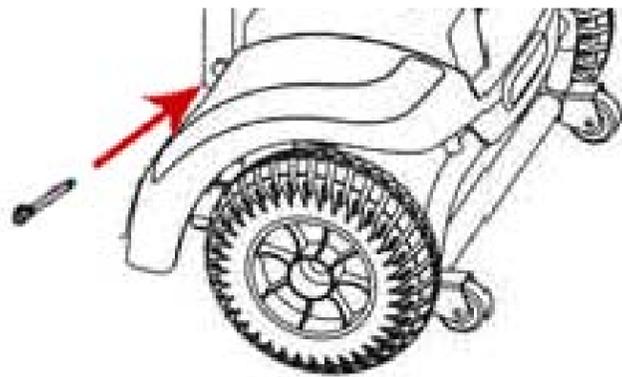


Tighten the bolt near the top of the seat tube with the included Allen wrench.


- 6.** Replace the battery pack, ensuring the battery is firmly in place.



- 7.** Attach the seat by lifting up on the handle on the right side of the seat and placing the seat down on the scooter's seat post. Release the handle to lock the seat in place. When the seat is locked, you will hear a click.

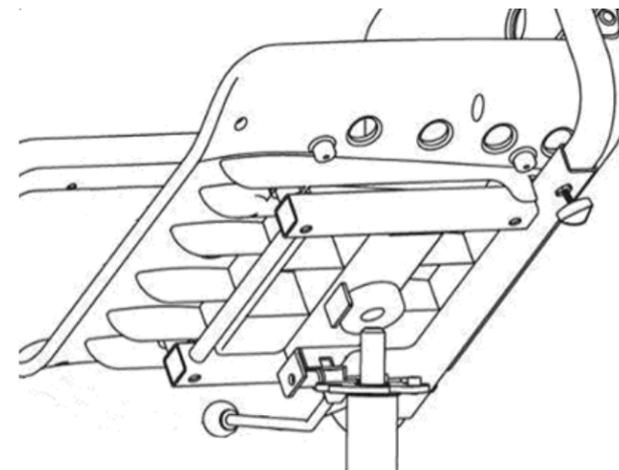


- 8.** Fully charge the scooter's batteries by inserting the round charger connector into the battery charging terminal on the scooter's battery pack and inserting the 3-pronged plug into a standard electrical outlet.

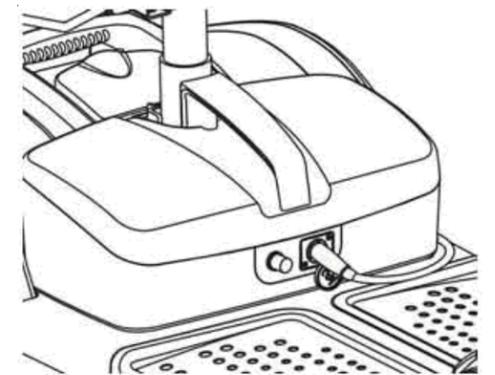

Full charging normally takes 6-8 hours, but can take as many as 10 hours depending on the batteries' charge level when starting and the ambient temperature.

- 9.** Before use, make sure that the scooter's freewheel mode lever is in the "drive" position.

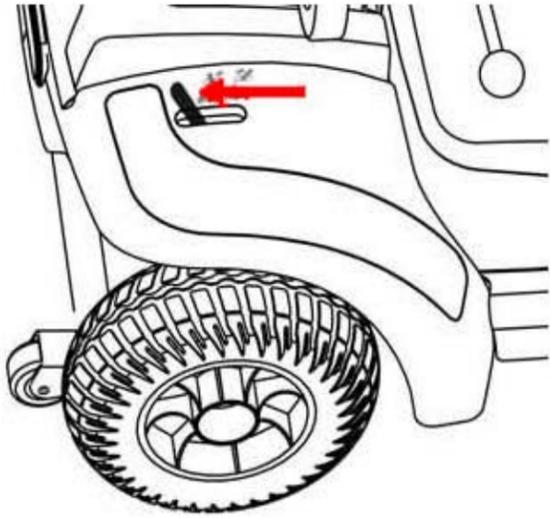

For information on the comfort adjustments available on your scooter, please consult your owner's manual.


Your Spitfire EX Scooter is designed to be easily disassembled into 5 pieces for convenient transport or storage.

The pieces are:

- Front Section (heaviest piece, 40 lbs.)
- Rear Section
- Battery Pack
- Seat
- Front Basket

To disassemble your scooter:

1. Turn key switch to "OFF."
2. Engage front wheel lock.
3. Place the freewheel mode lever in the "Drive" position.
4. Pull upward on the swivel-release lever to remove the seat from seat post.
5. Pull up on the battery pack to remove it from the scooter.
6. Pull up on the release lever to separate the front and rear sections.

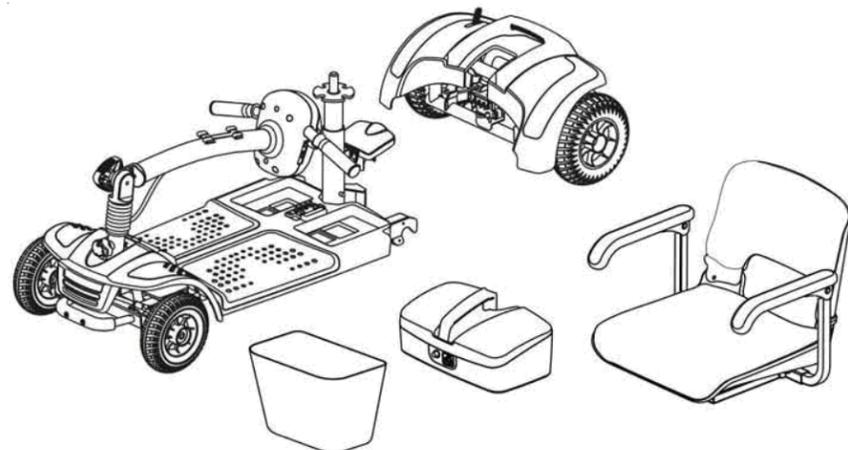

## Spitfire EX Quick Start Guide



### READ BEFORE UNPACKING

This Quick Start Guide offers basic information on initial product setup only. Read the owner's manual and all supplemental information thoroughly before operating the product for information on safety, battery charging, basic operation, comfort adjustments, disassembly and care and maintenance.

**NOTE:** The illustrations provided may not reflect your actual unit.