


15003 MANUAL HOMECARE BED

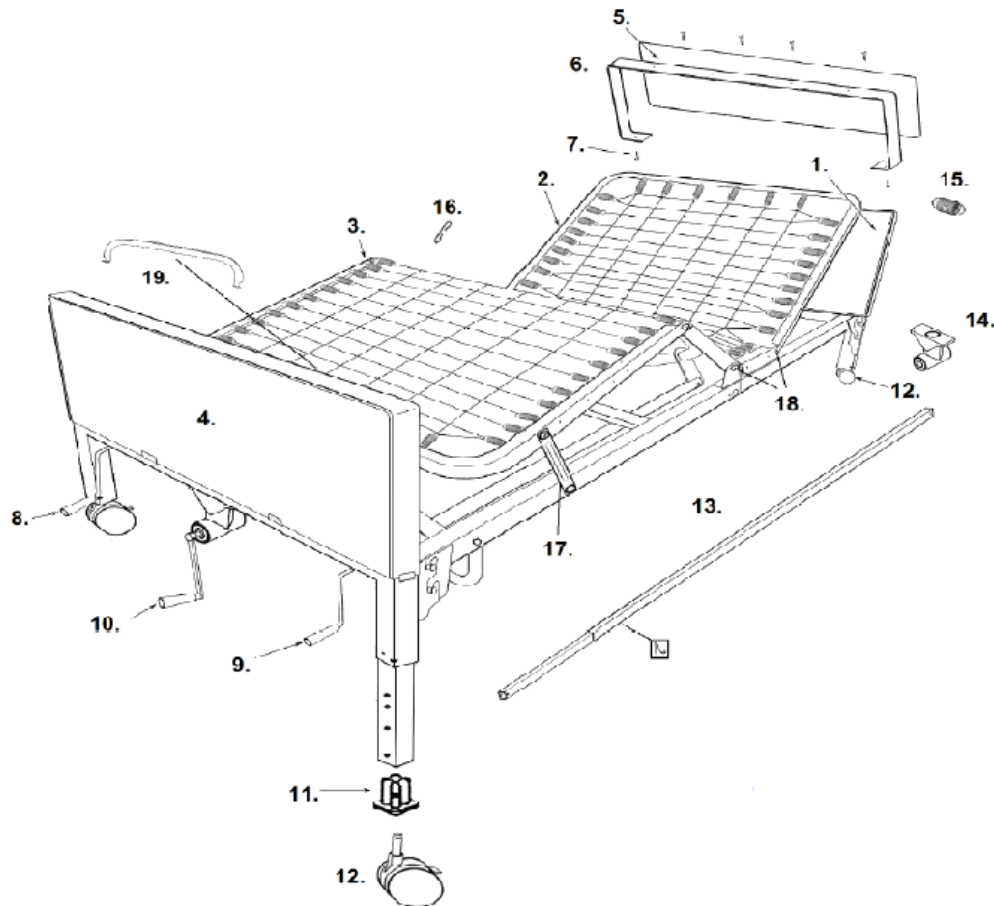

- | | |
|--|---|
| 1. Head Board- 15015H | 11. Caster Receptacle- 15005P-1027 |
| Head & Foot Board (1 of ea)- 15015 | 12. Caster w/Lock- 15044CL |
| 2. Head Spring- 15008 | Caster w/o Lock- 15044CNL |
| 3. Foot Spring- 15009 | Caster Set (2 of ea)- 15044CSET |
| 4. Foot Board- 15015F | 13. Hi/Lo Shaft- 15005S |
| 5. Head Board Panel (15.25"X35.375")- 15005P1035 | Hi/Lo Shaft Adj. Button- 15005SPN |
| Foot Board Panel (13"X35.375")- 15005P1036 | 14. Head Board Gear Box- 15005P1028 |
| 6. Head Board Panel Frame- 15005P1051 | 15. Extension Spring Head or Foot- 15005-SPRING |
| Foot Board Panel Frame- 15005P1151 | Extension Spring Side- 15005-SPRING-1 |
| 7. Panel Frame Screw- 15005PS1051 | 16. Sleep Surface Hook- 15005P-1040 |
| 8. Crank Handle Only Head/Foot- 15005P1045 | 17. Frame Linkage- 15030-FRLINK |
| Head Crank Assembly- 15003HC | 18. Spring Deck Bolt- 15030-FRLINK [1] |
| 9. Crank Handle Only Head/Foot- 105005P1045 | 19. Mattress Stop- 15005P1024 |
| Foot Crank Assembly- 15003FC | |
| 10. Hi/Lo Crank Handle Only- 15005P1047 | |
| Foot Board Gear Box- 15005PF1028 | |
| Hi/Lo Crank Handle/Gear Box- 15005P1046 | |

[1] 2 bolts included with this part.