


15004 LINAK SEMI-ELECTRIC HOMECARE BED (SERIAL # 1S1205 & LATER)

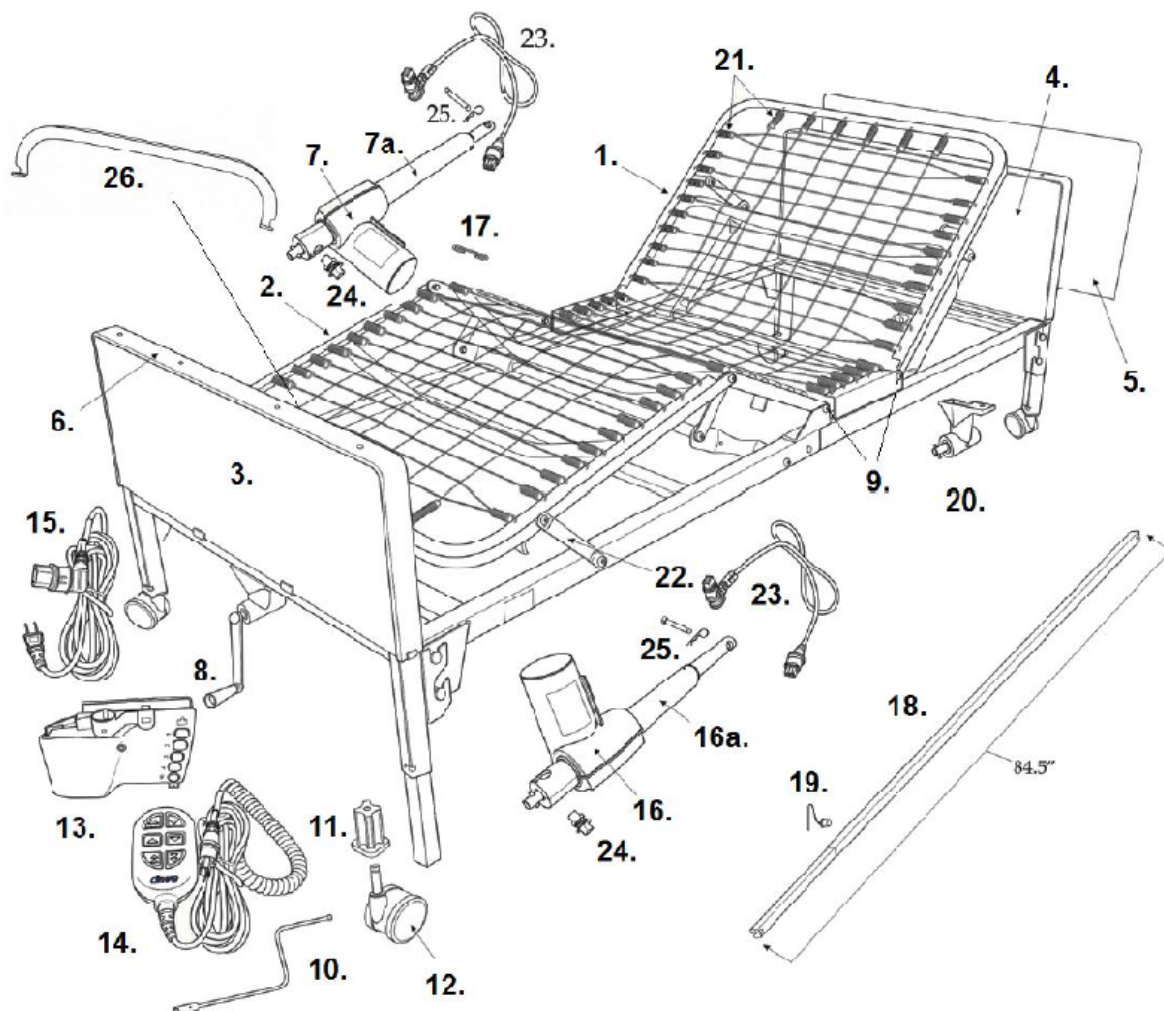

- | | |
|--|---|
| 1. Head Spring- 15008 | 13. Motor Control Box- 15005JBN-L |
| 2. Foot Spring- 15012-L | Motor Control Box Mtg. Bracket- 15005JBMB-L |
| 3. Foot Board- 15015F | 14. Hand Pendant- 15004HCN-L |
| 4. Head Board- 15015H | 15. Power Cord- 15005PC-L |
| Head & Foot Board (1 of ea)- 15015 | 16. Foot Motor- 15005MF-L |
| 5. Head Board Panel (15.25"X35.375")- 15005P1035 | 16a. Foot Motor Tube- 15005PF1025-L |
| Foot Board Panel (13"X35.375")- 15005P1036 | 17. Sleep Surface Hook- 15005P-1040 |
| 6. Foot Board Panel Frame- 15005P1151 | 18. Hi/Lo Shaft- 15005S |
| Head Board Panel Frame- 15005P1051 | 19. Hi/Lo Shaft Adjustment Button- 15005SPN |
| Panel Frame Screw- 15005PS1051 | 20. Head Board Gear Box- 15005P1028 |
| 7. Head Motor- 15005MH-L | 21. Extension Spring Head or Foot- 15005-SPRING |
| 7a. Head Motor Tube- 15005P1025-L | Extension Spring Side- 15005-SPRING-1 |
| 8. Hi/Lo Crank Handle Only- 15005P1047 | 22. Frame Linkage- 15030-FRLINK |
| Foot Board Gear Box- 15005PF1028 | 23. Head or Foot Motor Cable- 15005HFC-L |
| Hi/Lo Crank Handle/Gear Box- 15005P1046 | 24. Q/R Spring Knob- 15005P-1028 |
| 9. Spring Deck Bolt- 15030-FRLINK [1] | 25. Motor Attachment Pin- 15005P1016 |
| 10. Emergency Head/Foot Crank- 15005EC-L | 26. Mattress Stop- 15005P1024 |
| 11. Caster Receptacle- 15005P-1027 | |

12. Caster w/Lock- 15044CL
Caster w/o Lock- 15044CNL
Caster Set (2 of ea)- 15044CSET

[1] 2 bolts include with this part.

[Diagrams](#)

[Home](#)

15004 syn 1S1205_rev. 01_18_2020