


15005 LINAK FULL ELECTRIC BED (SERIAL # 1S1205..... & UP)

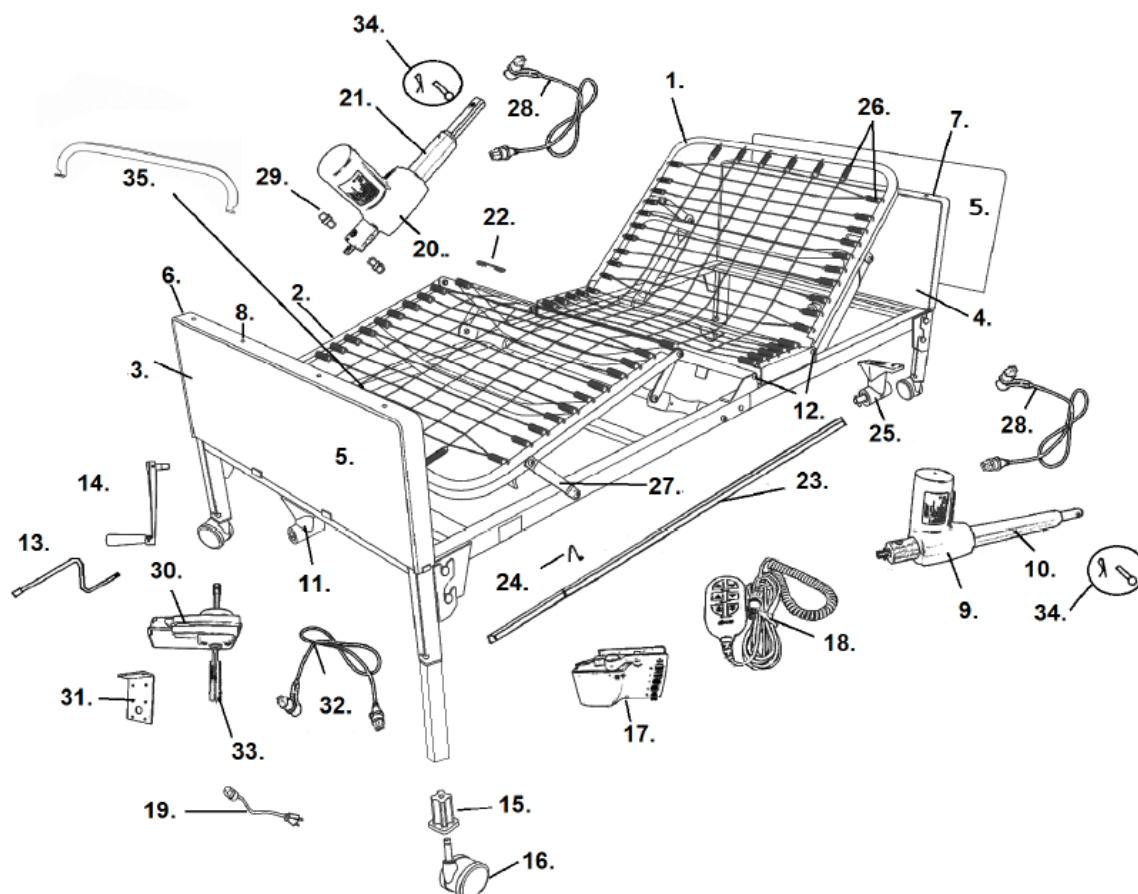

1. Head Spring- 15008
2. Foot Spring- 15013-L
3. Foot Board- 15018F
4. Head Board- 15018H
5. Head Board Panel (15.25" X 35.375")- 15005P1035
Foot Board Panel (13" X 35.375")- 15005P1036
6. Foot Board Panel Frame- 15005P1151
7. Head Board Panel Frame- 15005P1051
8. Panel Frame Screw- 15005PS1051
9. Head Motor- 15005MH-L
10. Head Motor Tube- 15005P1025-L
11. Foot Board Gear Box- 15544FGB
12. Spring Deck Bolt- 15030-FRLINK *
13. Emergency Head/Foot Crank- 15005EC-L
14. Emergency Hi/Lo Crank- 15544C
15. Caster Receptacle- 15005P-1027
16. Caster w/Lock- 15044CL
Caster w/o Lock- 15044CNL
Caster Set (2 of ea)- 15044CSET
17. Motor Control Box- 15005JBN-L

18. Hand Pendant- 15005HCN-L
19. Power Cord- 15005PC-L
20. Foot Motor- 15005MF-L
21. Foot Motor Tube- 15005PF1025-L
22. Sleep Surface Hook- 15005P-1040
23. Hi/Lo Shaft- 15005S
24. Hi/Lo Shaft Adjustment Button- 15005SPN
25. Head Board Gear Box- 15005P1028
26. Extension Spring Head or Foot- 15005-SPRING
Extension Spring Side- 15005-SPRING-1
27. Frame Linkage- 15030-FRLINK
28. Head or Foot Motor Cable- 15005HFC-L
29. Q/R Spring Knob- 15005P-1028
30. Hi/Lo Motor- 15005HL-L
31. Hi/Lo Motor Mounting Bracket- 15005P1029-L
32. Hi/Lo Motor Cable- 15005HLC-L
33. Hi/Lo Motor Coupler- 15005HLS
34. Motor Attachment Pin- 15005P1016
35. Mattress Stop- 15005P1024

Motor Control Box Mtg. Bracket- 15005JBMB-L

*** 2 bolts included with this part.**

[Diagrams](#)

[Home](#)

15005 LINAK_rev. 7_29_2020