


15300 BARIATRIC BED 42"

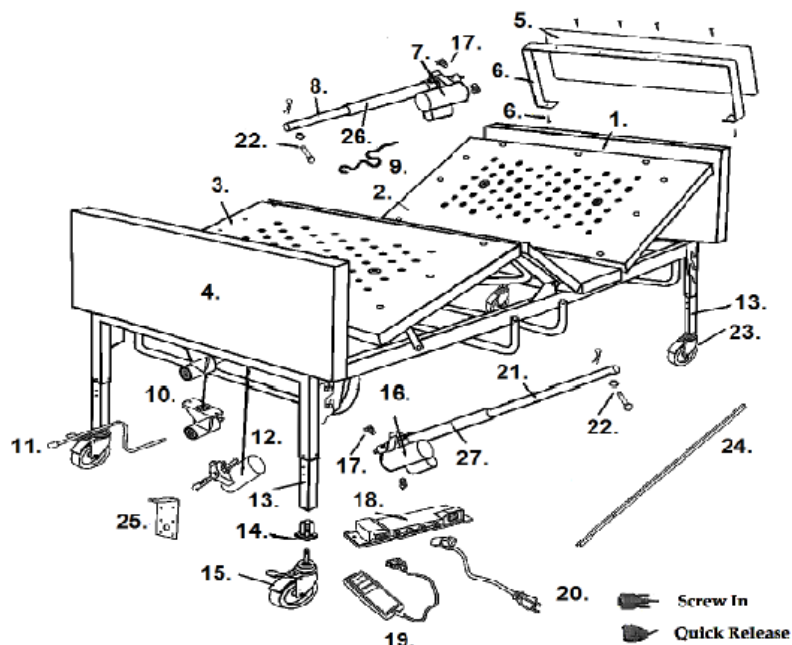

- | | |
|---|---|
| 1. Head Board- 15307H
Head & Foot Board (1 of ea)- 15307 | 13. Adj. Leg Head (565mm)- 15300P1015L [1]
Adj. Leg Foot (505mm)- 15300PF1015L [1] |
| 2. Head Frame- 15305 | 14. Caster Receptacle- 15300EC |
| 3. Foot Frame- 15306 | 15. Caster with Lock- 15300CL |
| 4. Foot Board- 15308 | 16. Foot Motor- 15300MF |
| 5. Head Board Panel- 15300P1005-1
Foot Board Panel- 15300P1006-1 | 17. Q/R Spring Knob- 15005P-1028 |
| 6. Head Board Frame- 15300P1051
Foot Board Frame- 15300P1052
Panel Frame Screw- 15005PS1051 | 18. Motor Control Box- 15005JBU |
| 7. Head Motor- 15300MH | 19. Hand Pendant Screw On- 15005HC
Hand Pendant Quick Release- 15005HCN |
| 8. Head Motor Actuating Tube- 15300PH1014 | 20. Power Cord- 15005PC |
| 9. Head Motor Extension Cord- 15300P1001 | 21. Foot Motor Actuating Tube- 15300P1014 |
| 10. Head Board Gear Box- 15005P1028
Foot Board Gear Box- 15005PF1028 | 22. Attachment Pin & Hardware- 15300P1016 |
| 11. Emergency Crank Head/Foot- 15005P1023 | 23. Caster w/o Lock- 15300C |
| 12. Hi/Lo Motor- 15300HL
Hi/Lo Motor Coupling- 15005HLS | 24. Hi/Lo Shaft- 15005S |
| | 25. Motor Bracket- 15005P1029 |
| | 26. Head Motor Tube- 15005P1025 |
| | 27. Foot Motor Tube- 15005PF1025 |

[1] Includes cable & guides, does not include part # 14.