


15303 BARIATRIC BED 54" (S/N 1B &12R)

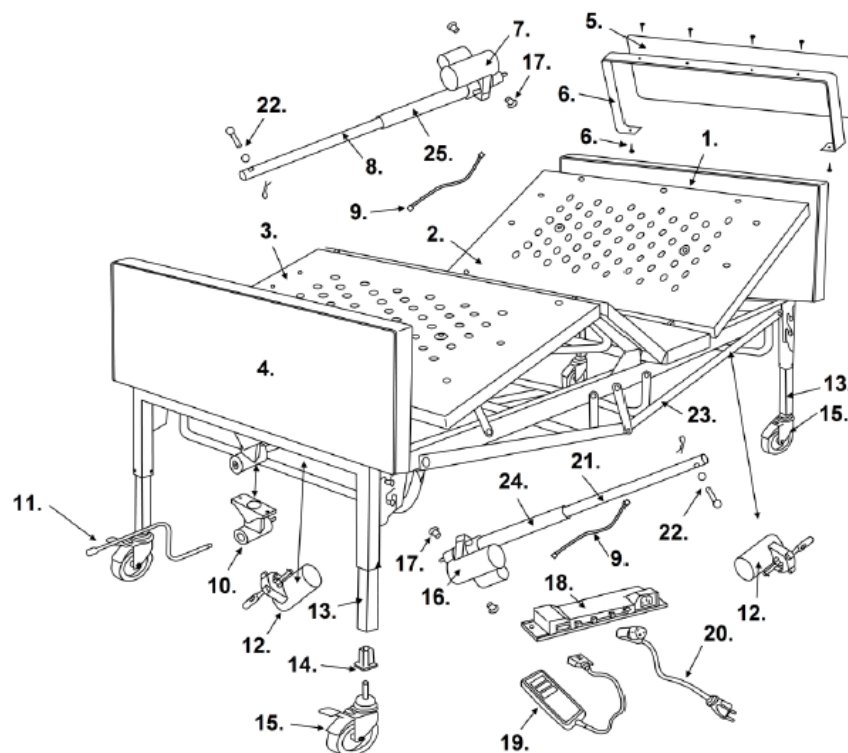

- | | |
|---|---|
| 1. Head Board- 15355H | 13. Adj. Leg Head (565mm)- 15300P1015L [2] |
| Head & Foot Board (1of ea)- 15355 | Adj. Leg Foot (505mm)- 15300PF1015L [2] |
| 2. Head Frame- 15315N [1] | 14. Caster Receptacle- 15300EC |
| 3. Foot Frame- 15325N | 15. Caster with Lock- 15300CL |
| 4. Foot Board- 15355F | Caster w/o Lock- 15300C |
| 5. Head Board Panel- 15303P1005 | 16. Foot Motor- 15302MF |
| Foot Board Panel- 15303P1006 | 17. Q/R Spring Knob- 15005P-1028 |
| 6. Head Board Frame- 15303P1051 | 18. 4 Port Motor Control Box- 15302JB-4PORT |
| Foot Board Frame- 15303P1052 | 19. Hand Pendant Screw On- 15005HC |
| Panel Frame Screw- 15005PS1051 | 20. Power Cord- 15005PC |
| 7. Head Motor- 15032MHN | 21. Foot Motor Actuating Tube- 15300P1014 |
| 8. Head Motor Actuating Tube- 15300PH1014 | 22. Attachment Pin & Hardware- 15300P1016 |
| 9. Head & Hi/Lo Motor Ext. Cord- 15300CCN | 23. Frame Support Rail- 15300FSR |
| 10. Head or Foot Board Gear Box- 15005P1028 | 24. Foot Motor Tube- 15005PF1025 |
| 11. Emergency Crank Head/Foot- 15005P1023 | 25. Head Motor Tube- 15005P1025 |
| 12. Hi/Lo Motor- 15302HL | 26. Emergency Crank Hi/Lo- 15544C |

[1] Head frame has joint pin.

[2] Includes cable & plastic guides, does not include part 14.