


P1752 PRIME LONG TERM CARE BED

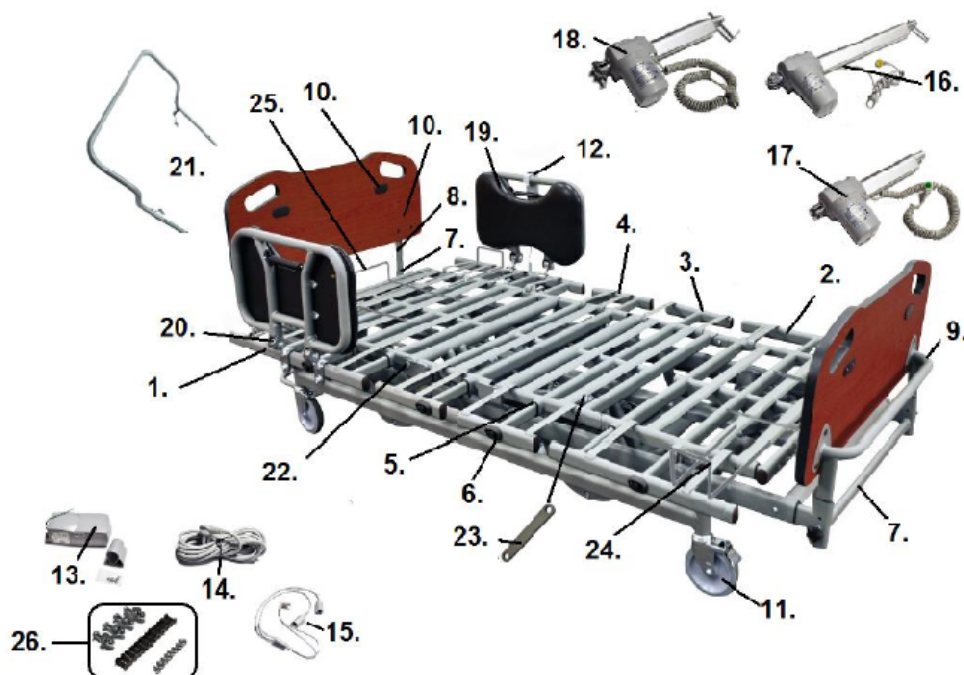

1. Head Deck- SP02-P1752-10008E
Head Deck Width Extension Left- SP02-P1752-08E
Head Deck Width Extension Right- SP02-P1752-07E
2. Foot Deck- SP02-P1752-10007E
Foot Deck Width Extension Left- SP02-P1752-12E
Foot Deck Width Extension Right- SP02-P1752-11E
3. Thigh Deck- SP02-P1752-10006E
Thigh Deck Width Extension- SP02-P1752-10E
4. Hip Deck Width Extension- SP02-P1752-09E
5. Width Extension Guides- SP02-PPB2000-SMB
6. Width Extension Release Assembly- SP02-PPB2000-35E
7. End Board Adj Mtg Brkt- SP02-PPB1750-31E
Length Extension Bolt- SP01-KBCZ31C50
Hand Wheel Knob- SP02-PPB1750-43E
8. Head Board Mounting Bracket (pr)- SP02-PPB1750-1612E
9. Foot Board Mounting Bracket- SP02-PPB1750-1712E
10. End Board Bolts Male (20ea)- SP02-UF16108051E
End Board Bolts Female (20ea)- SP02-UN90108091E
Head/Foot Board Plug (pr)- SP02-PPB2000-17-03
11. Locking Caster- SP02-UA18321520E
12. Hand Pendant (8 button)- SP01-72298
13. Motor Control Box- SP01-69159
14. Power Cord- SP01-CA65738
15. "Y" Cable- SP01-CA51336
"Y" Cable Mounting Bracket- SP02-PPB2000-36E
16. Hi/Lo Motor (2 used)- SP01-79323

- Hi/Lo Pivot Bolt & Hardware (2" length) (ea)- SP02-ML10021000701
Hi/Lo Pivot Bolt & Hardware (4" length) (ea)- SP02-ML21100970E-4
17. Foot Motor- SP01-81069
 18. Head Motor- SP01-81077
 19. Side/Assist Rails- PBASR *
Side/Assist Rail Pad- SP02-PPBASRP
 20. Side/Assist Rail Mounting Bracket- SP02-PPBRSRB
Side/Assist Rail Release Handle- SP02-PPB1750ARRH
Side/Assist Rail Liner- SP02-1750-13-03E
 21. Wall Guard- SP02-PPB1750-0211E
 22. Head/Foot/Thigh Deck Hardware- SP02-PPB2000-DB
 23. Foot Deck Pivot Bolt (ea)- SP02-PPB2000-23DB
Foot Deck Pivot Arm (pr)- SP02-PPB2000-27DB
 24. Side Mattress Retainer- SP02-ML1002-10004E
 25. Head/Foot Mattress Retainer- SP02-ML1002-10005E
 26. Hardware Pack- SP02-ML1750-0001E

* Accessory

[Diagrams](#)

[Home](#)

P1/52_rev. 07_20_2020