


PAN6300

S/N #15APAN6300D00001 & LATER

01. Head Deck- PAN6300-10003E
02. Foot Deck- PAN6300-10006E
03. Frame Pivot Bolt- SP02-ML1000-20E
04. Head, Foot, Thigh Deck Hdwr- SP02-PPB2000-DB
05. Foot Deck Pivot Bolt- PAN6300-23DB
06. Head Width Extension Left- PAN6300-DM-WE-05E
- Head Width Extension Right- PAN6300-DM-WE-04E [1]
07. Foot Width Extension Left- PAN6300-DM-WE-06E [1]
- Foot Width Extension Right- PAN6300-DM-WE-03E [1]
08. Thigh Wdth Extension Left/Right- PAN6300-DM-WE-02E [1]
09. Hip Width Extension Left/Right- PAN6300-DM-WE-01E [1]
10. Width Extension Guide- PAN6300-39E-DS
11. Width Extension Release Assembly- PAN6300-35E-DS
12. End Mattress Retainer- PAN6300-10008-02E
13. Y-Cable- PAN6302-CA51336
14. Y-CABLE Mounting Bracket- PAN6300-36E-DS
15. Wall Guard- PAN6300WB
16. Side Mattress Retainer- PAN6300-10008-01E
17. Head Board Mounting Board Bracket- PAN6300-49E
18. Foot Board Mounting Board Bracket- PAN6300-50E
19. Locking Caster- PAN6300RC
20. Hand Pendant 8 Button- PAN6300P-2
21. Control Box w/o Power Cord- PAN6300RCB-2 [1]
(sub for PAN6300RCB-2 is SP01-69159)
22. Hi/Lo Actuator - PAN6300RHLA-2
23. Foot Actuator- PAN6300RFA-2
24. End Cap 45mm- PAN6300-41E-DS
25. Side Assist Rail/Bar Pair- 0-C8359
26. Side Assist Rail Pad Left- PAN6300-13-04AE
- Side Assist Rail Pad Right- PAN6300-13-04BE
- Assist Rail Liner- SP02-1750-13-03E
27. Length Extension Bracket Bolt- SP01-KBCZ31C50
28. Side Assist Pivot Bolt- PAN6300-48E-DS

Only parts listed are available.

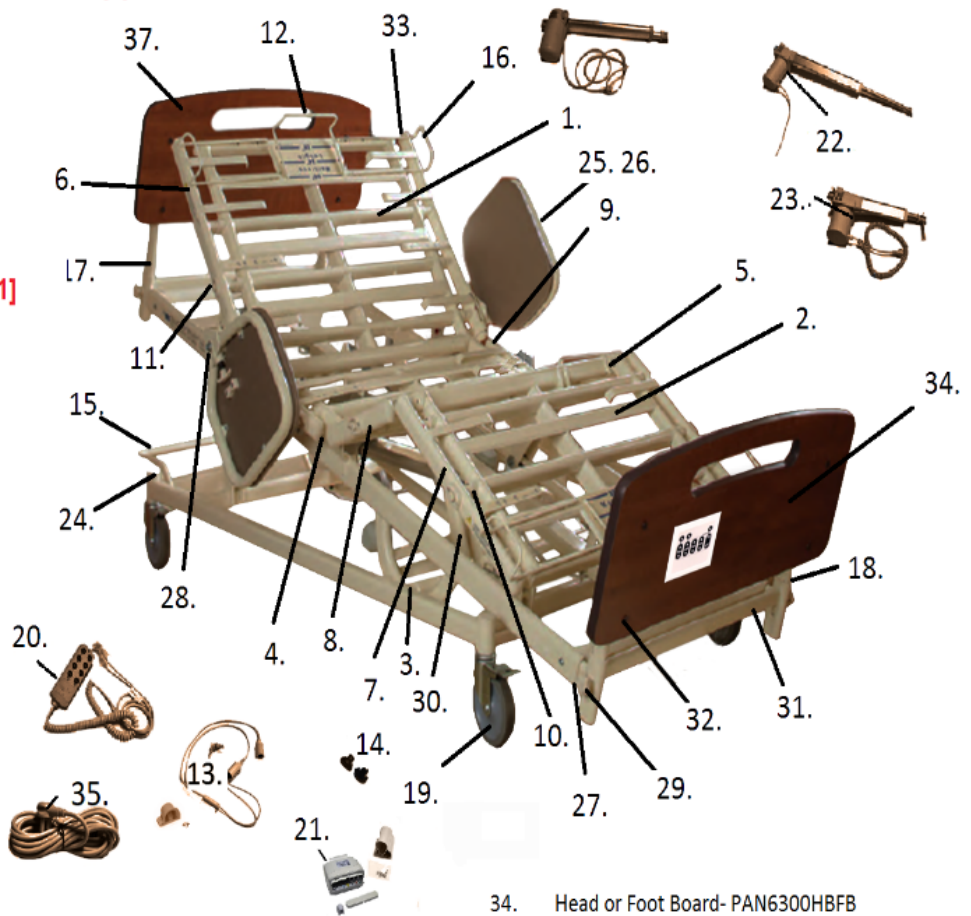

29. Hand Wheel Knob- PAN6300-43E-DS
30. Foot Deck Pivot Arm- PAN6300-27DB
31. End Board Adj. Length Mtg. Brkt.- PAN6300-47E-DS
32. End Board Mtg. Bolts (20 ea)- PAN6300-UF16108051
33. End Cap 30mm X50mm- PAN6300-40E-DS

34. Head or Foot Board- PAN6300HBFB
- Foot Board/Staff Control- PAN6300BE-SC01
- Staff Control- PAN6300SC
35. Power Cord- PAN6300RCP-2
- Sub. - SP01-CA65738
- n/s Battery Back-Up- PAN6300BB-2 *
- n/s Hardware Pack- PAN6300-01E-DS

* Accessory, not a part

[1] When stock is gone the part will no longer be available.

04/22/2021

Diagrams

Home