




PAN6502

LONG TERM CARE BED



01. Head Deck- PAN6300-10003E
02. Foot Deck- PAN6300-10006E
03. Thigh Deck- [3]
04. Head, Foot, Thigh Deck Hdwr- SP02-PPB2000-DB
05. Foot Deck Pivot Bolt- PAN6300-23DB
06. Head Width Extension Left- PAN6300-DM-WE-05E
07. Head Width Extension Right- PAN6300-DM-WE-04E [2]
08. Foot Width Extension Left- PAN6300-DM-WE-06E [2]
09. Foot Width Extension Right- PAN6300-DM-WE-03E [2]
10. Thigh Width Extension Left/Right- PAN6300-DM-WE-02E [2]
11. Hip Width Extension Left/Right- PAN6300-DM-WE-01E [2]
12. Width Extension Guide- PAN6300-39E-DS
13. Width Extension Release Assembly- PAN6300-35E-DS
14. Power Cord- PAN6300RPC-2
15. Y-Cable- PAN6302-CA51336
16. Y-CABLE Mounting Bracket- PAN6300-36E-DS
17. Wall Guard- PAN6300WB
18. Side Mattress Retainer- PAN6300-10008-01E
19. Head Board Mounting Board Bracket- PAN6300-49E
20. Foot Board Mounting Board Bracket- PAN6300-50E
21. Locking Caster- PAN6300RC
22. HandPendant- PAN6300P-2
23. Control Box- PAN6300RCB-2 [2]
24. Hi/LoActuator- PAN6300RHLA-2
25. Foot Actuator- PAN6300RFA-2
26. Brkt & Hdwr Foot Actuator- PAN6300-10009E
27. Head Actuator- [3]
28. Side Assist Rail/Bar Pair- O-C8359 [1]
29. Side Assist Rail Pad Left- PAN6300-13-04AE
30. Side Assist Rail Pad Right- PAN6300-13-04BE
31. Length Extension Bracket Bolt- SP01-KBCZ31C50
32. Side Assist Pivot Bolt- PAN6300-48E-DS
33. Hand Wheel Knob- PAN6300-43E-DS

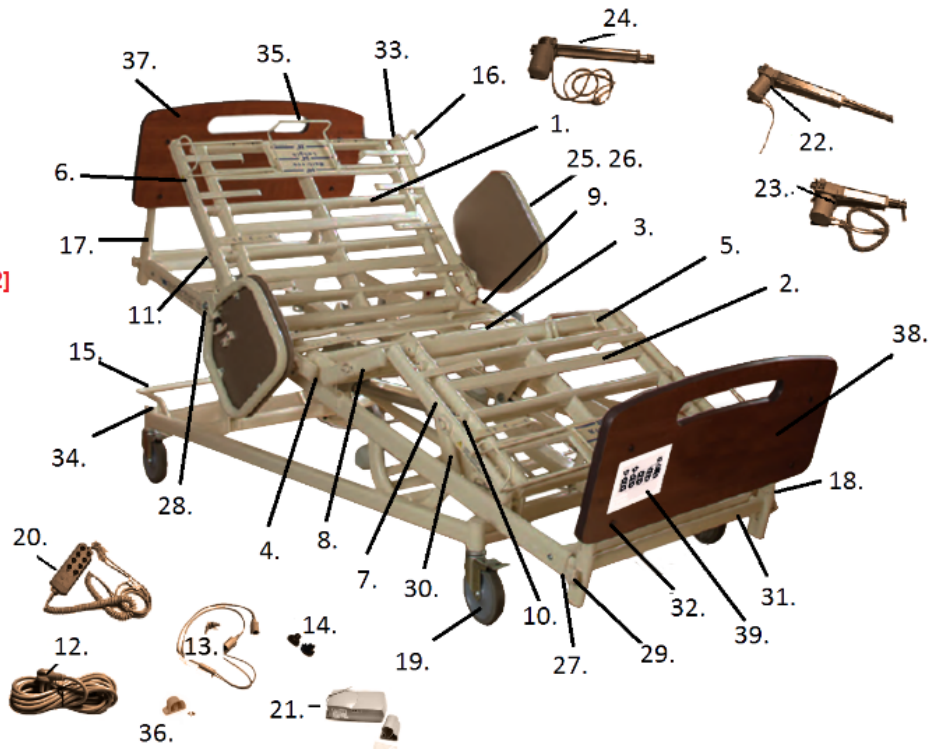

30. Foot Deck Pivot Arm- PAN6300-27DB
31. End Board Adj. Length Mtg. Brkt.- PAN6300-47E-DS
32. End Board Mtg. Bolts (ea)- PAN6300-UF16108051
33. End Cap 30mm X50mm- PAN6300-40E-DS
34. End Cap 45mm- PAN6300-41E-DS
35. End Mattress Retainer- PAN6300-10008-02E
36. Strain Relief Bracket- [3]

37. Head Board- PAN6300HBFB
38. Foot Board- PAN6300HBFB-SC01
39. Staff Control- PAN6502SC
- Staff Control Cable- SP01-51335
- Battery Back-Up- PAN6300BB-2
- Hardware Pack- PAN6300-01E-DS

[2]
[3]

[1] Accessory not a part.
[2] When stock is gone the part is no longer available.
[3] The part is no longer available.

08/26/20

[Diagrams](#)
[Home](#)