


TL500

LONG TERM CARE BED

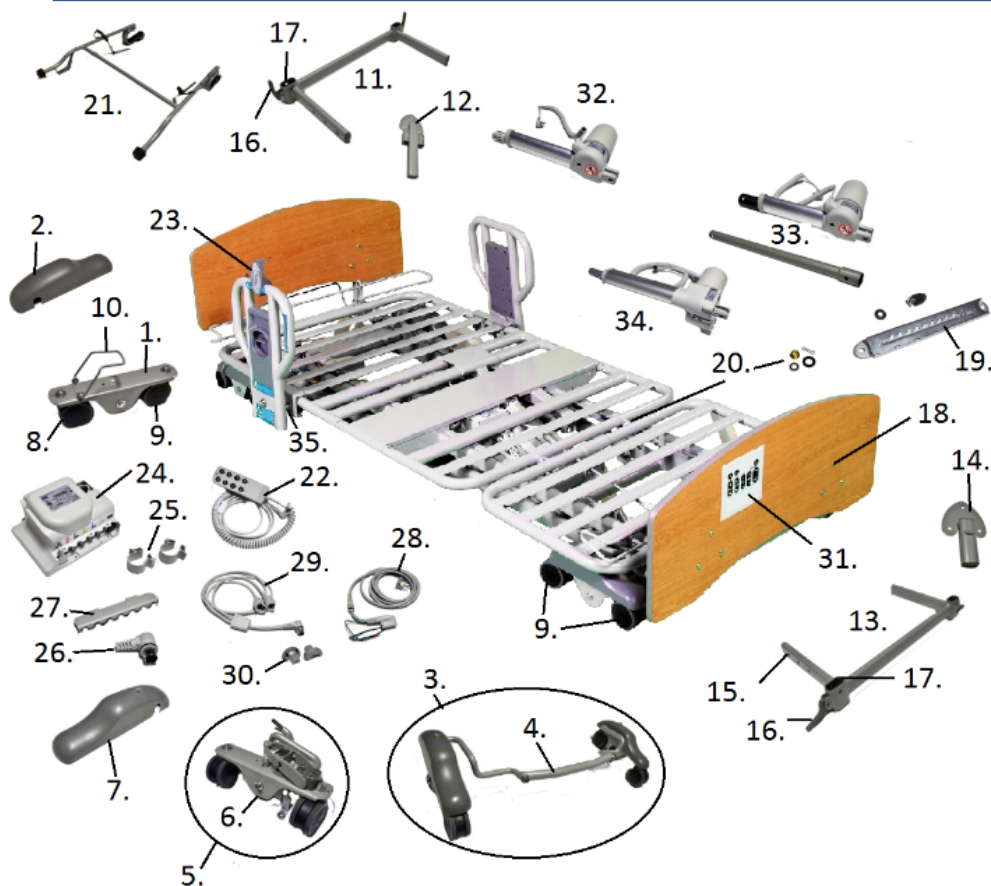

1. Head Caster Assy Left- SP01-PCB900-20E
Right- SP01-PCB900-19E
2. Head Caster Cover Left- SP01-PCB900-53E
Right- SP01-PCB900-54E
3. Foot End Caster Brake Assy- SP01-PCB900-CA
4. Brake Crossbar- SP01-PCB900-21AE
5. Foot Caster/Brake Assy Left- SP01-PCB900-17E
Right- SP01-PCB900-18E
6. Brake Stop- SP01-PCB900-24BS
7. Foot Caster Cover Left- SP01-PCB900-55E
Right- SP01-PCB900-56E
8. Locking Casters- SP01-TL500-37E

9. Non-Locking Casters- SP01-TL500-29E
10. Caster Bales- SP01-PCB900-63E
11. Head Board Adj Mtg Brkt- SP01-PCB900-12AE
12. Head Board Mtg Brkt- SP01-PCB900-10BE
13. Foot Board Adj Mtg Brkt- SP01-PCB900-11AE
14. Foot Board Mtg Brkt- SP01-PCB900-09BE
15. Adj Length Brkt Bolt- SP01-KBCZ31C50
16. Head/Foot Board Locking Lever- SP01-TL500-57E-G
17. Bracket Retaining Sleeve- SP01-TL500-63E-G
18. End Board Bolts Male- SP01-UF16108030E
End Board Bolts Female- SP01-UN90108090E
19. Foot End Ratchet Assy- SP01-PCB300-01-57E
20. Brass Leg Pivot Bushing- SP01-TL500-59E
21. Wall Guard- SP01-PCB900-41E
22. Hand Pendant- SP01-72298
Extended Cord Pendant- SP01-40662
23. Side/Assist Rail Pendant Mount- SP01-PCB900-PH
24. Control Box- SP01-63925
25. Control Box Mtg Brkt- SP01-PCB900-40E
26. Control Box Bypass Plug- SP01-PCB900-47E
27. Control Box Access Cover- SP01-PCB900-CBC
28. Power Cord- SP01-59094
29. Y-Cable- SP01-CA51336
30. Y-Cable Mtg Brkt- SP01-PCB900-62E
31. Embedded Staff Control- SP01-55658
Embedded Staff Controls Cable- SP01-51335
32. Head Actuator- SP01-63708
33. Hi/Lo Actuator- SP01-64670
Hi/Lo Adapter Cable- SP01-CA35825
34. Foot Actuator- SP01-63716
Foot Actuator Tube- SP01-FAT
35. Side/Assist Rail Pivot Bolt- SP01-ML29160430E
Hand Wheel Knob- SP01-PA23112454E-B
n/s Updated Software- SP01-SP55661
n/s Transport Brackets- SP01-PCB901TPE

7/14/20

[Diagrams](#)
[Home](#)