


1. Hanger Attachment Bolt- 13023HDWR
2. 4/6 Point Cradle- 13065
3. Mast (Complete)- 13023M
Upper Mast- 13023MU
Lower Mast- 13023ML
4. Pump Attachment Bolt- 13023PB
5. Mast Hinge Bolt- 13023MHB
6. Hand Grip (ea)- 13023H
7. Pump- 13052
- 7a. Release Lever- 13052RL
8. Spreader Bar Handle- 13023SBH
9. Spreader Bar- 13023SB
Spreader Bar Bolt- 13023SBB
10. Base- 13023B
11. Locking Caster 5"- 13023CL
3"- 13023C3L
12. Non-Locking Caster 5"- 13023C
3"- 13023C3
13. Base Tip Cover- 13023BC

When part stock specific to 13023
are gone, they will be longer available.

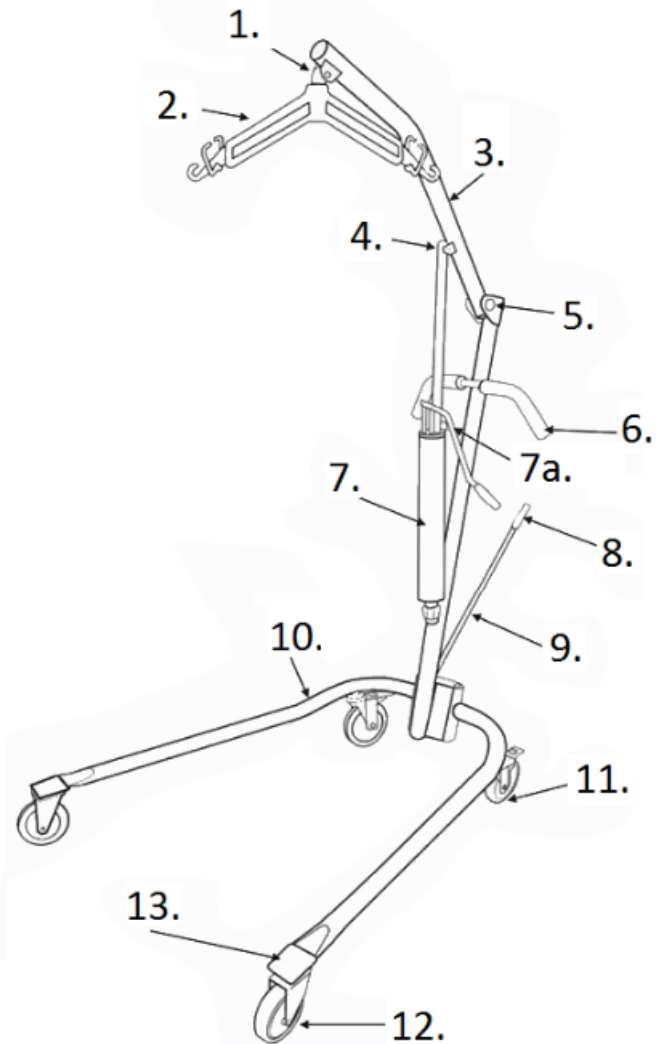

01/29/18