


# 13023SV

1. Hanger Attachment Bolt- 13023SV01
2. 4/6 Point Cradle- 13023SV02  
6 Point Cradle- 13023SV24
3. Mast Complete- 13023SV03
  - a. Upper Mast- 13023SV04
  - b. Lower Mast- 13023SV05
4. Pump Attachment Bolt- 13023SV06
5. Mast Hinge Bolt- 13023SV07
6. Push Handle Grip (ea)- 13023SV08
7. Pump- 13023SV09
  - a. Pump Release Lever- 13052RL
  - b. Pump Handle Grip- 13023SV22
  - c. Pump Handle Hardware- 13023SV09-FPHH
8. Spreader Bar Handle Grip- 13023SV10
9. Spreader Bar- 13023SV11
10. Thumb Screw- 13023SV12
11. Spreader Bar Receiver- 13023SV13
12. Base w/Casters- 13023SV15
13. Rear Caster w/o Lock- 13023SV17  
Rear Caster with Lock- 13023SV16
14. Front Caster w/o Lock- 13023SV18
15. Base Tip Cover- 13023SV19
16. Tube Plug- 13023SV20
17. Cradle Pivot- 13023SV21
18. Mast Cap- 13023SV23

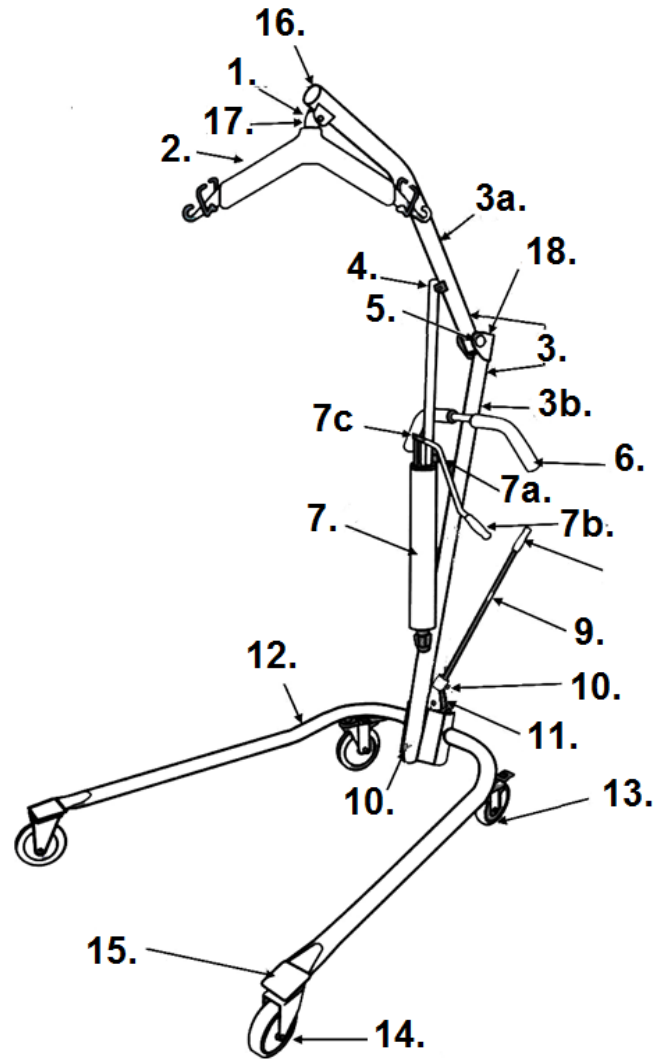

01/29/18