


CP18FBAN

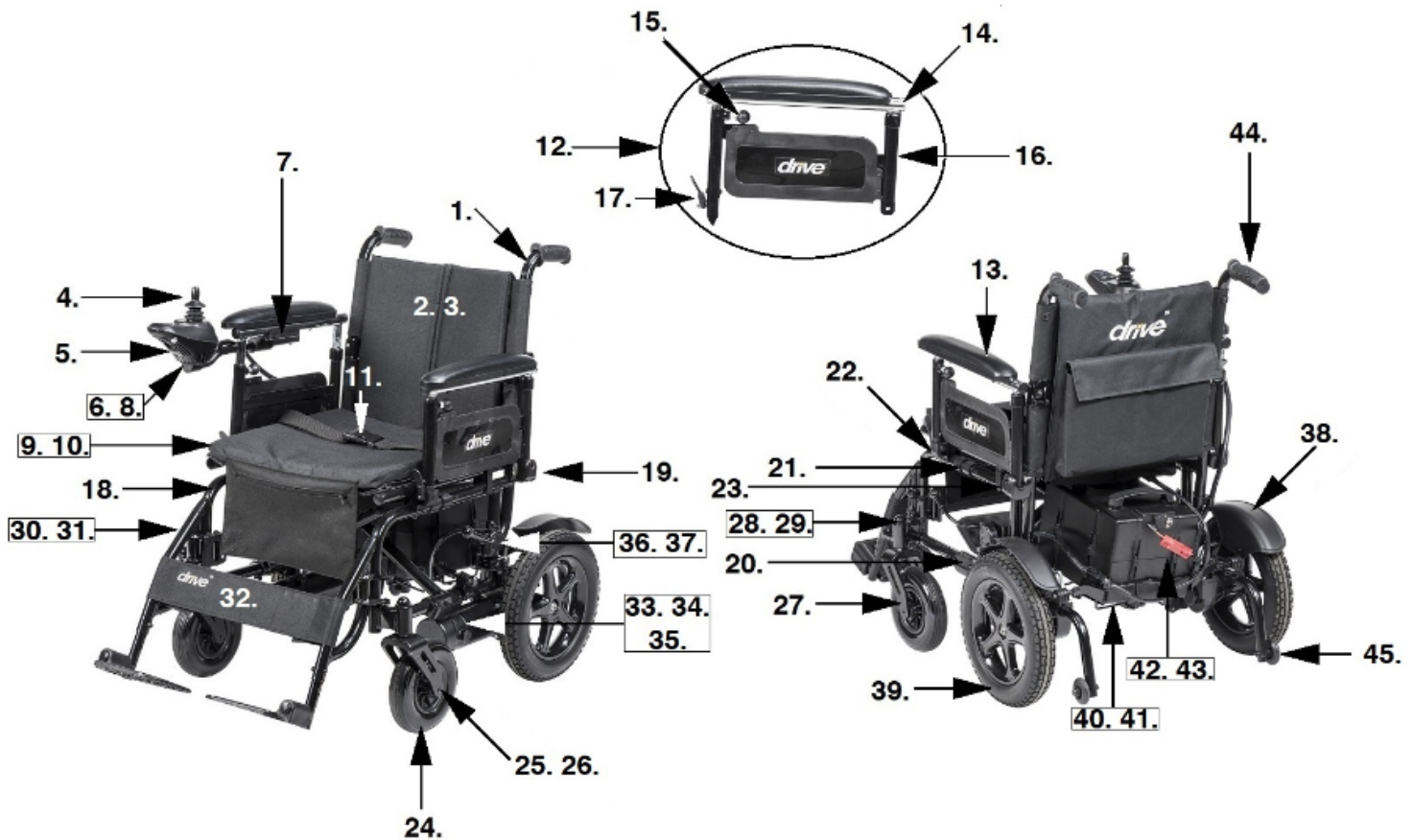

- 1. Back Cane- CP8017N
- 2. Adj. Tension Back Uph 18"- CP8005N-18
- 3. Padded Back Upholstery 18"- CP8006N-18
- 4. Joystick - CP6400PN
- 5. Joystick Tip- CP6200TN
- 6. Joystick Bracket- CP6205PN
- 7. Joystick Bracket Mount- CP6205N
- 8. Joystick Mounting Screw (ea)- SI69001
- 9. Seat Upholstery & Hardware 18"- CP8002N-18
- 10. Seat Cushion 18"- CP8001N-18
- 11. Seat Belt- CP3320N
- 12. Left Adj. Ht. Armrest- CP800NL
- 12. Right Adj. Ht. Armrest- CP800NR
- 13. Arm Pad- STDSFLNB
- 14. Upper Arm Frame- CP8004N
- 15. Height Adjustment Knob- CP8009N
- 16. Left Lower Arm Frame- CP8014NL
- 16. Right Lower Arm Frame- CP8014NR

- 17. Arm Release Lever- CP8007N
- 18. Left Side Frame- CP8017NL
- 18. Right Side Frame- CP8017NR
- 19. Left Arm Pivot Bracket- CP8006LN
- 19. Right Arm Pivot Bracket- CP8006RN
- 20. Cross Frame 18"- CP8020N-18
- 21. Cross Frame Assist 18"- CP8019N-18
- 22. Left Front Seat Saddle- CP8008NL
- 22. Right Front Seat Saddle- CP8008NR
- 23. Rear Seat Saddle- CP8008N-18
- 24. Caster- CP8038N
- 25. Caster Bearing (ea)- STDS1011
- 26. Caster Axle Assembly- CP1204N-A
- 27. Fork- CP1201N
- 28. Dust Cap- STDS1051N
- 29. Fork Stem Bearing (ea)- STDS1060
- 30. Footrest (Pr)- CPSFN
- 31. Footrest Accessory Kit (ea)- CP1098N
(Anti-Rattler, Bumper & Release Lever)-

- 32. Calf Strap- CP8039N
- 33. Left Motor- CP8050NL
- 33. Right Motor- CP8050NR
- 34. Motor Brush- CP5201N
- 35. Motor Brush Cap- CP5202N
- 36. Left Brake- CP8035NL
- 36. Right Brake- CP8035NR
- 37. Brake Tip (ea)- CP8110
- 38. Left Rear Fender- CP801NL
- 38. Right Rear Fender- CP801NR
- 39. Rear Wheel- CP8012N
- 40. Battery Box Basket 18"- CP8015N-18
- 41. Front Battery Basket Clips (pr)- CP5005CFN
- 41. Rear Battery Basket Clips (pr)- CP5005CRN
- 42. Battery Box 1 Cable- CP5211N-1
- 43. Battery (ea) (2 needed)- LRA402208
- 44. Hand Grip- STDS871
- 45. Anti-Tipper (ea)- CP8025

09/02/2021

[Diagrams](#)
[Home](#)