


COBALT


SHARK JOYSTICK & CONTROLLER

1. Joystick- LPW182001
Joystick Overlay- DK-REMD01
Joystick Cable- LPW182001-02
Substitute Cable- **No Longer Available**
2. Gear Box Assm.- Left – LDR420002-05FN
Right – LDR420002-06FN
Motor Brush (each)- LDR420002-07
Axle Key- TITAN-40
3. Circuit Breaker- LRM302107
4. Battery (12V21Ah)- LRM412123
5. Charger (2A)- LRM302111
6. Bearing Assm.- LDR410006-01
7. Caster Fork Assm.- **No Longer Available**
8. Front Caster Grey- LDR420007
Black- LDR420007-02
- 8a. Caster Bearing- STDS1011
- 8b. Caster Axle- STDS1M2210
9. Rear Wheel- Grey- **No Longer Available**
Black- LDR420008-02
10. Frame Lock Bolt Assm.- LDR420010
11. Anti-tipper Wheel- LDR420009-01
12. Seat Assm.- LDR420011
13. Front Shroud Blue - **No Longer Available**
Front Shroud Red - **No Longer Available**
Rear Shroud Blue – LDR420012-02BFN
Rear Shroud Red – LDR420012-02RFN
14. Footrest- LDR410016
15. Battery Bag Hold Down Strap- LDR420003-02
16. Shroud Knob- LDR410115-03
17. Controller-LPW180024
18. Joystick Arm- LDM400001-04
18a. Joystick Screws (pr)-LDM400001-01
19. Joystick Mounting Bracket- LDM400001-05
Joystick Adjustment Knob- LDM400001-07
20. Arm Assm.- Left – LDR420016L,
Right – LDR420016R
21. Seat Post 19.45mm- LDR420000-FN
22. Battery Cable- LDR420000-01 *
23. Dust Cap- LDR4110
24. Front Articulating Bar- LDR420000-26
25. Handle- LDR420000-19
26. Contact Block Set- LDR420001-FN
27. Front Frame- **No Longer Available**
28. Battery Bag- LDR420003-01
29. Seat Plate 19.45mm- LDR420011-FN
30. Shroud Extension Cable- LPW182001-03

* Uses 40 amp fuse.

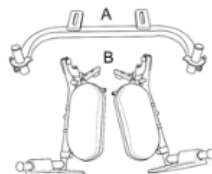

To attach ELRs:
A. AE2500
B. LK3JELR

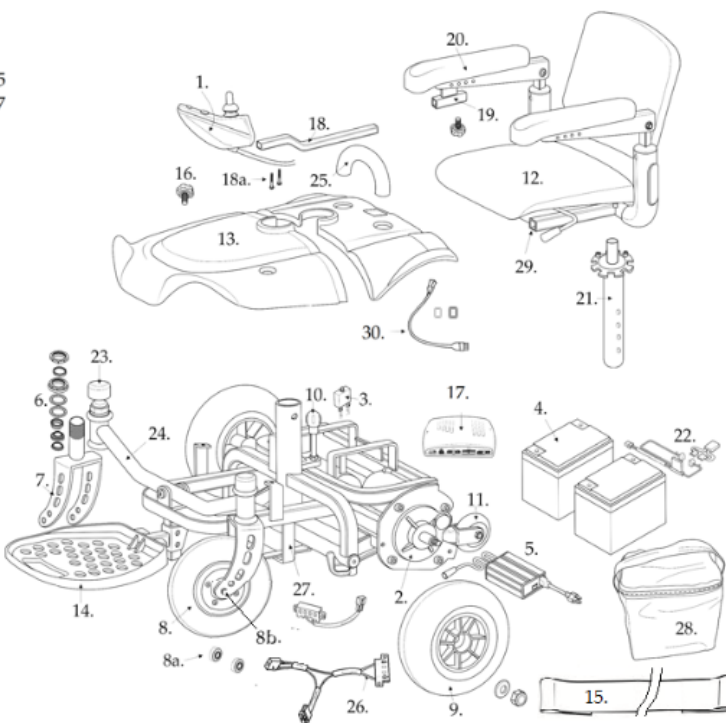

01/07/2021

Diagrams

Home