


CIRRUS PLUS EC

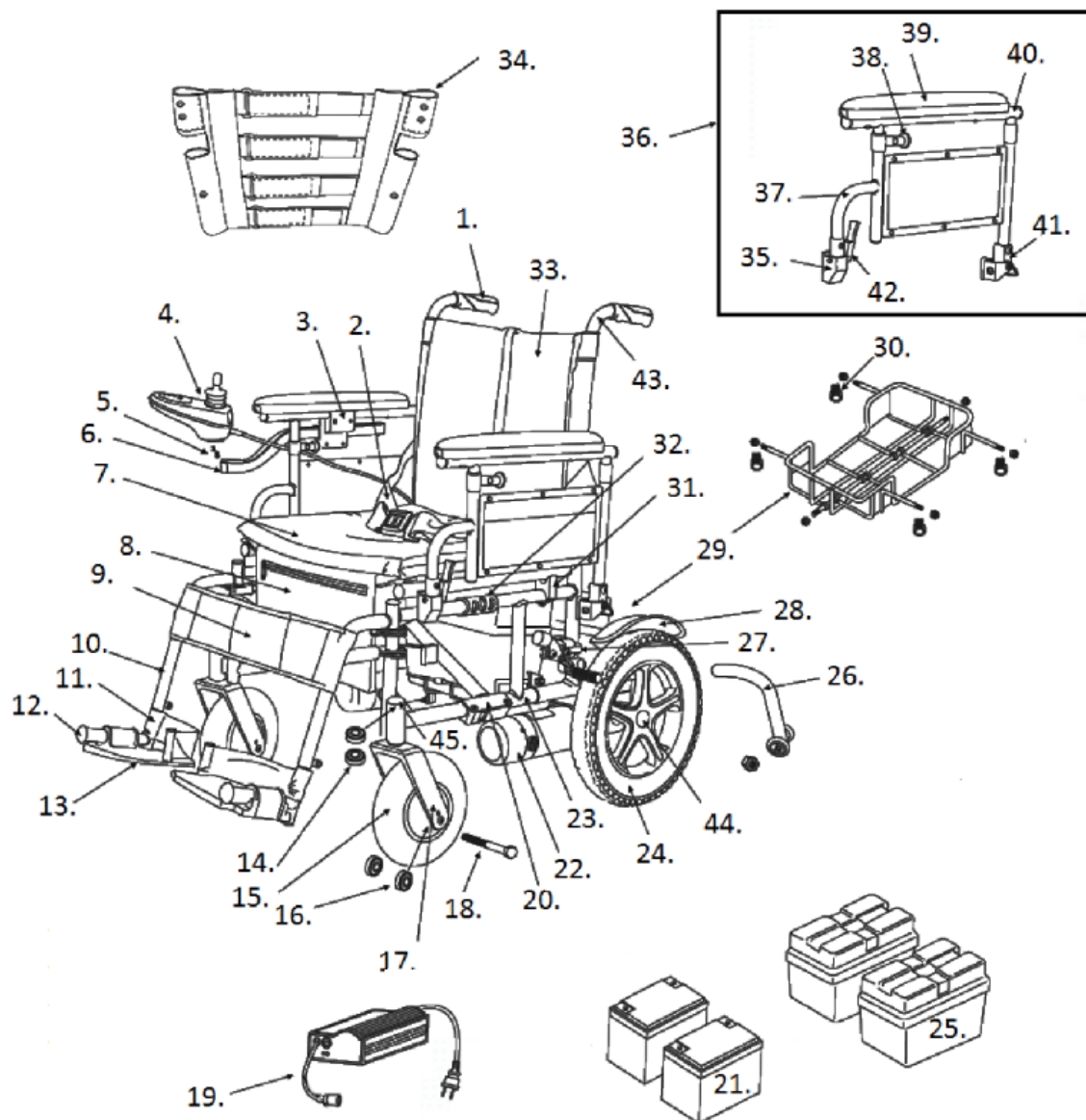

1. Handgrip- STDS871
2. Seat Belt- STDS850
3. Arm Clamp*- CPN6205
* Clamp to clamp button head-screw size: M6X1.00X6mm-
4. Joystick- CP6200PN
Joystick Tip- CP6200T
Joystick Boot- CP6200PN-BOOT
Joystick Over Lay- NVSI-OVERLAY
5. Joystick Screws- SI69001
6. Joystick Arm- CP6205P
7. Seat Upholstery 16"- CP8002-16

23. Side Frame Left- CPN8017L
Side Frame Right- CPN8017R
24. Rear Wheel- CP8012
25. Battery Case 1 Plug- CP5211-1
Battery Case 2 Plug- CP5211-2
Battery Case Set - CP5115
Battery Case Set & Batteries- CPLUSBATT
Batt. Set Case & 50ah Batteries- CPLUSBATT-50
Battery Strap- VENT-09
26. Anti-Tipper (ea)- CP8025
27. Brake Left- CP8035L
Brake Right- CP8035R

- Seat Upholstery 18"- CP8002-18
- Seat Upholstery 20"- CP8002-20
- Seat Upholstery 22"- CP8002-22
- Seat Upholstery 24"- CP8002-24
- Seat Cushion 16"- CP8001-16
- Seat Cushion 18"- CP8001-18
- Seat Cushion 20"- CP8001-20
- Seat Cushion 22"- CP8001-22
- Seat Cushion 24"- CP8001-24
- 8. Seat Bag- CPN8010
- 9. Calf Strap 16"- CP6039 [1]
- Calf Strap 18"- CP8039
- Calf Strap 20"- CP2039 [1]
- Calf Strap 22"- CP2239
- Calf Strap 24"- CP2489
- 10. Footrest Left- CPNSFL
- Footrest Right- CPNSFR
- Footrest Pair- CPNSF
- Elevating Leg Rest Pair- CPELR
- 11. Heel Loop (ea) [2]
- Heel Loop (pr)- STDS831
- 12. Bumper- STDS1026
- 13. Foot Plate Left [2]
- Foot Plate Right [2]
- 14. Fork Stem Bearing (ea)- STDS1060
- 15. Caster- CP8040
- 16. Axle Bearing (ea)- STDS1011
- 17. Fork- CP1204
- 18. Caster Axle- CP1204-A
- 19. Charger- CPC [3]
- Fuse for CPC charger- CPCFUSE
- 20. Cross Frame 16"- CP8020-16 [1]
- Cross Frame 18"- CP8020-18
- Cross Frame 20"- CP8020-20
- Cross Frame 22" [2]
- Cross Frame 24"- CP8020-24
- 21. Battery (ea) (2 needed)- LRA402208
- 22. Motor Assembly Left- CP8050L
- Motor Assembly Right- CP8050R
- Motor Brush (ea)- CP5201
- Motor Brush Cap (ea)- CP5202
- 28. Fender Left [2]
- Fender Right [2]
- 29. Folding Basket w/o Clips 16"- CP8015-16
- Folding Basket w/o Clips 18"- CP8015-18
- Folding Basket w/o Clips 20"- CP8015-20
- Folding Basket w/o Clips 22"- CP8015-22
- Folding Basket w/o Clips 24"- CP8015-24
- 30. Basket Clip Rear- CP5005C
- Basket Clip Front- CP5005CF
- 31. Seat Saddle- CPN8005
- 32. Cross Frame Link 16"- CP8019-16
- Cross Frame Link 18"- CP8019-18 [1]
- Cross Frame Link 20"- CP8019-20
- Cross Frame Link 22" [2]
- Cross Frame Link 24"- CP8019-24
- 33. Back Upholstery 16"- CP8006-16
- Back Upholstery 18"- CP8006-18
- Back Upholstery 20"- CP8006-20
- Back Upholstery 22"- CP8006-22
- Back Upholstery 24"- CP8006-24
- 34. Tension Back 16"- CP8005-16
- Tension Back 18"- CP8005-18
- Tension Back 20"- CP8005-20
- Tension Back 22" [2]
- Tension Back 24"- CP8005-24
- 35. Front Seat Saddle/Arm Receiver Lt.- CPN8008L
- Front Seat Saddle/Arm Receiver Rt.- CPN8008R
- 36. Armrest Assembly Left- CPN800L
- Armrest Assembly Right- CPN800R
- 37. Lower Arm Frame- CPN8014
- 38. Adjustment Lock- CPN8009
- 39. Full Arm Pad- STDSFLNB
- 40. Upper Arm Frame- CPN8004
- 41. Pivot Bracket- CPN8006
- 42. Release Lever- CPN8007
- 43. Back Cane- CPN8017
- Back Cane Adjustment Knob- STDS895
- 44. Hub Cap- CPN8021 [1]
- 45. Dust Cap- STDS1051

[1] When stock is gone the part will be no longer available.

[2] Part is no longer available.

[3] When stock is gone use LRA402210