


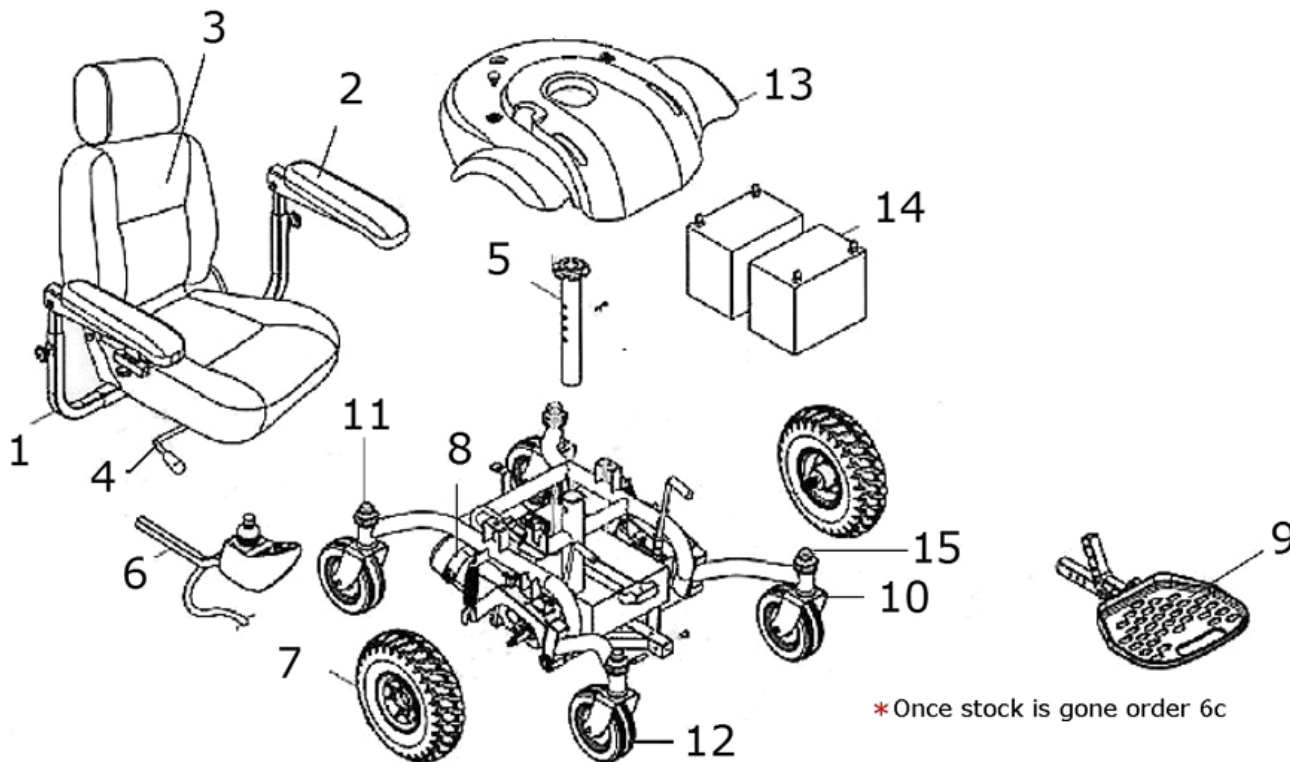

* Once stock is gone order 6c

1	LDR410011-09	Left Armrest Bracket ***	7	LDR410008	Drive Wheel Assembly
	LDR410011L	Left Armrest Assembly includes bracket and armrest pad	8	LDM400002-01	Gearbox Assembly Left ***
	LDR410011-10	Right Armrest Bracket ***		LDM400002-02	Gearbox Assembly Right ***
	LDR410011R	Right Armrest Assembly includes bracket and armrest pad	9	LDR410016	Footrest Assembly
2	LDR410110	Armrest Pad	10	LDM400013	Caster Fork
3	ST205-181820	18" Captain Seat **	11	LRB402348	Caster Fork Bearings
	ST205-181820-W	18" Captain Seat Assembly includes seat, armrest assembly & seat bracket **	12	LDM400005	Caster Assembly
	ST301-201820	20" Captain Seat	13	LDM400010B	Blue Shroud ***
	ST301-201820-W	20" Captain Seat Assembly includes seat, armrest assembly & seat bracket		LDM400010R	Red Shroud ***
NS	LDR410017	Seatbelt	14	LRA402208	35AH Battery each
4	LRC402414	Seat Bracket - Captain Seat	15	LDM400013-02	Dust Cap
5	LRC402414-13	Seat Post	15	STDS1M2210	Caster Axle
6a	LDM400001	Dynamic Joystick *	NS	LRA402209	3.5A Battery Charger
	DC GLA81043	Dynamic Joystick Overlay	NS	LDM400000-12	Wiring Harness ***
b	LDM400001PG	P&G Joystick	NS - Not Shown		
c	LDM400001PG-C	P&G Joystick Assembly	* When stock is gone order part 6c.		
			** When stock is gone part is		
			*** No longer available.		

09/09/2019