



IMAGE EC

1. Head Rest 18"- ECI6015 ***
20"- **NLA**
2. Seat 18"- **NLA**
20"- **NLA**
3. Joystick Tip- CP6200T
4. Joystick (gray Plug)- S350169701
(yellow plug)- S350169701-REV2 *
5. Joystick Mtg Screws- SI69001
6. Joystick Arm- GTI6045 ✓
7. Seat Plate- ECI6007
8. Wire Harness Complete- SF9038GT ***
9. Shroud- ECI6011 **NLA**
10. Footplate- SF9035 ***
11. Caster- ECI6040
12. Fork- ECI6020
13. Controller (gray plug)- S350169738
(yellow plug)1 S350169738-REV2 *
14. Suspension- **NLA**
15. Fender Shroud
Burgundy Left- ECI6010BUL
Right-- ECI6010BUR
Blue Left- **NLA**
Right- **NLA**
16. Drive Wheel- SF9060
17. Motor 3200 rpm Left- ECI6018L-3200 **
Right- ECI6018R-3200 **
4000 rpm Left- ECI6018L ***
Right- ECI6018R ***
18. Seat Post- SF9010
19. Battery (2 needed)- LRA402208
20. Charger- **NLA**
21. Charger Cord- S31050PC
22. Arm Assembly Left- SF9007L
Right- SF9007R
23. Adjustment Knob- GTI6046 ***
24. Arm Pad- SF9007GT
25. Battery Cover- ECI6012
26. Rear Frame- ECI6043
27. Articulating Beam- ECI6030

ELR adapter
A. ECELR-P
B. SFELR
C. LK3JELR

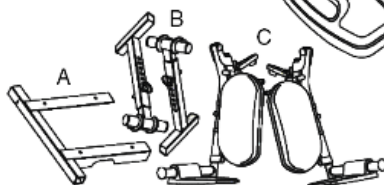

28. Drive Wheel Nut- ECI6018
29. Fork Stem Cap- GTI6044 ***
30. Fork Bearing (ea)- STDS2A5303
31. Center Shroud- ECI6013
32. Front Frame- ECI6042
33. Caster Axle- ECI6039
34. Caster Bearing (pr)- ECI6041

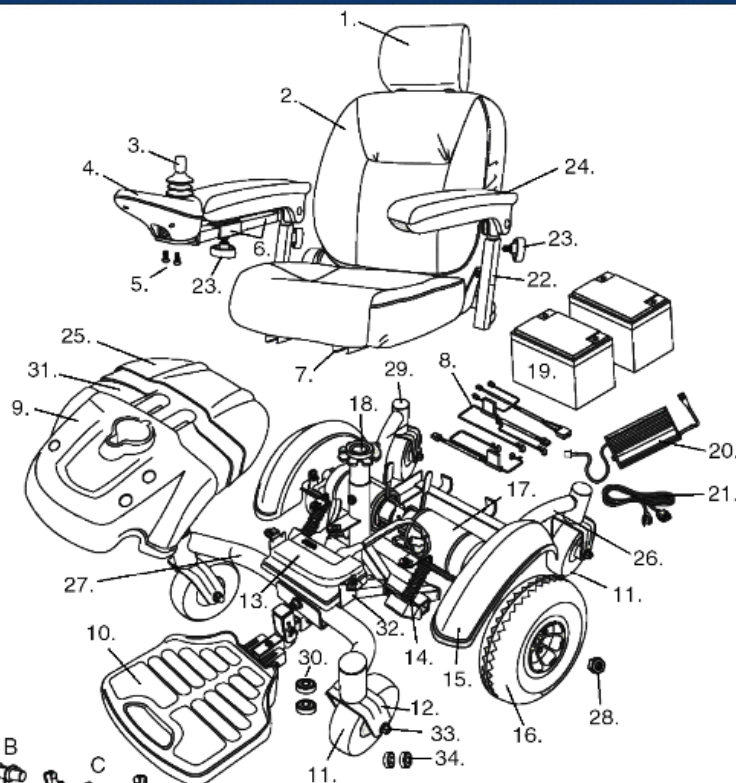

* 18" Seat Ser # 2W367R1304000898 and later.
20" Seat Ser # 2W367R1304000165 and later.
** Ser # 2W301B1109000869 and later.
*** When stock is gone, the part is no longer available.

01/08/2021