


TITAN AXS

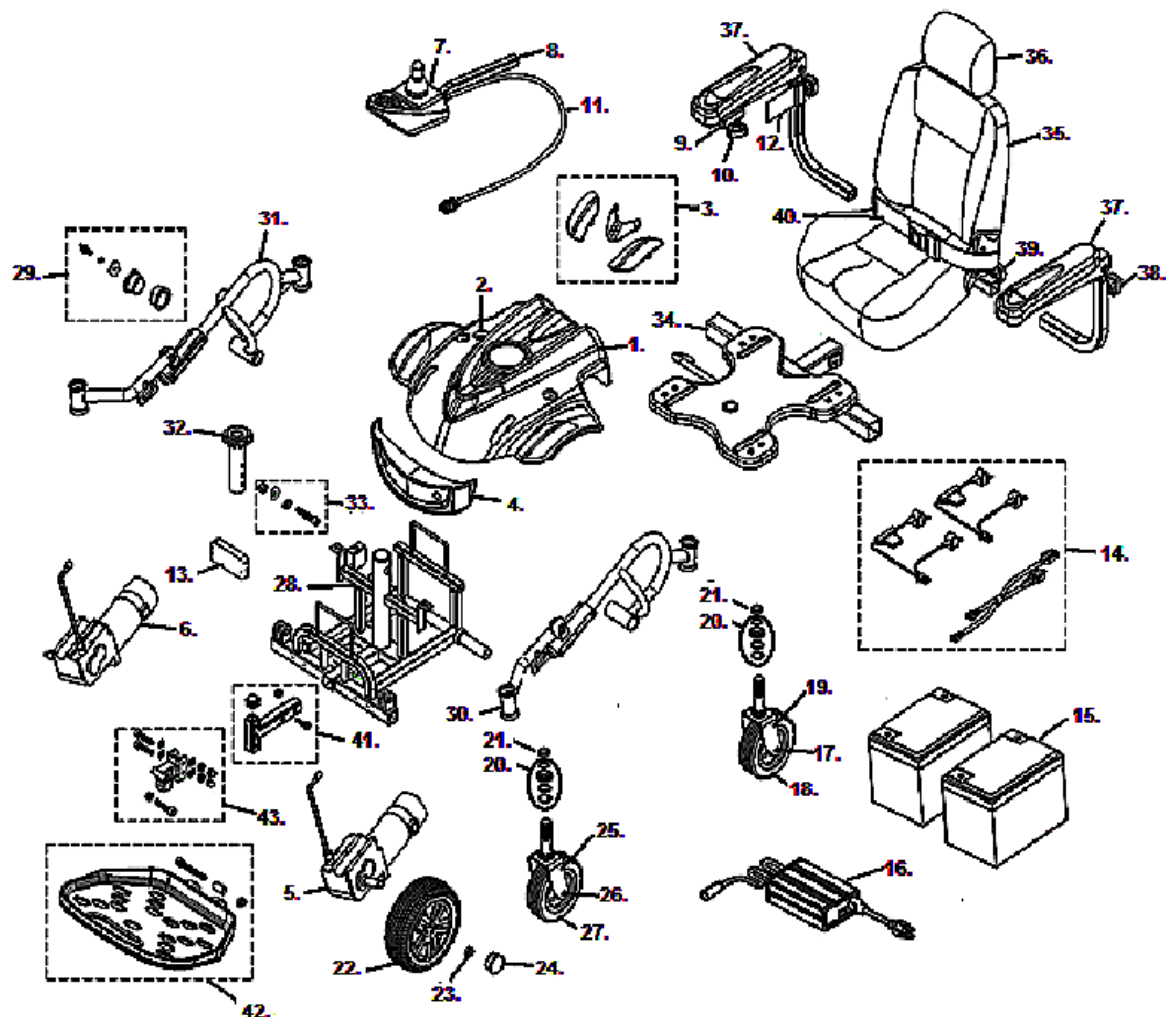

- | | |
|---|--|
| 1. Shroud- C08-084-00100 | 27. Front Caster - TITAN-03 |
| 2. Shroud Knob- VENT-86 | 27. Front Caster- TITAN-03N |
| 3. Decoration Set- S08-084-00118 | 27. Front Caster Bearing- STDS1011 |
| 4. Front Cover & Hardware- TITANMID-01 | 28. Main Frame- TITANMID-05 |
| 5. Motor, Gear Brake Assy. Left- C09-084-00201 | 29. Frame Bushing Assembly- TITANMID-06 |
| 6. Motor, Gear Brake Assy. Right- C09-084-00101 | 30. Side Frame Assembly Left- TITANMID-07 |
| 7. Joystick w/o Extension Cable- LINX-JOYSTICK | 31. Side Frame Assembly Right- TITANMID-08 |
| Joystick Tip Linx- DC GPL55470 | 32. Seat Post - TITAN-39 |
| 8. Joystick Arm- LDM400001-04 | 33. Seat Post Bolt Assembly- TITAN-06 |
| Joystick Mtg. Screw- LDM400001-01 | 34. Seat Plate- C07-072-01001 |
| 9. Joystick Mtg. Bracket- TITAN-24 | 35. Captain Seat 16 x 16- TITANAXS-01 |
| 10. Joystick Knob- LDM400001-07 | 35. Captain Seat 16 x 18- TITANAXS-02 |
| 11. Joystick Extension Cable- GLM-BUS150-A | 35. Captain Seat 18 x 16- TITANAXS-03 |
| 12. Joystick Velcro Cable Tie- C34-072-00100 | 35. Captain Seat 18 x 18- TITANAXS-04 |
| 13. Controller- DLX-PM40-A-AXS | 35. Captain Seat 20 x 18- TITANAXS-05 |
| Controller to Shroud Cable- GLM-EXT064-A | 35. Captain Seat 20 x 20- TITANAXS-06 |
| 14. Wire Harness- TITAN-12 | 35. Captain Seat 22 x 20- TITANAXS-07 |
| 15. Battery 35 ah (2 needed)- LRA402208 | 35. Pan Seat 18 x 18- ST501-181818-AXS |
| 16. Charger 3 Amp- LRA402209 | 35. Pan Seat 20 x 18- ST502-201820-AXS |
| 17. Rear Caster Axle & Nut- TITAN-04 | 36. Headrest- ST-HEADREST |
| 18. Rear Caster- TITAN-03 | 37. Armrest Assembly Left- LDR410011L |
| | |

Rear Caster Bearing- STDS1011
19. Rear Fork- LDM400013
20. Fork Bearing Set Assembly- LRB402348
21. Dust Cap- LDR4110
22. Drive Wheel- TITAN-11
23. Axle Nut- ECI6018
24. Hub Cap- LRM412202-HC
25. Front Fork- LDM400013
Front Fork- TITANMID-09 [2]
26. Caster Axle & Nut- TITAN-04
Caster Axle & Nut- TITAN-04N [2]

37. Armrest Assembly Right- LDR410011R
37. Arm Pad- LDR410110
37. Arm Pad Screw (pr)- LDR420018
38. Armrest Height Adj Knob- S31052
39. Armrest Width Adj Knob- S31052
40. Seatbelt 48"- STDS850-PW
40. Seatbelt 60"- STDS855-PW
41. Footrest Hanger Assembly- TITANMID-02
42. Footrest Assembly- TITANMID-03
43. Footrest Mtg Bracket Assembly- TITANMID-04
not shown ELR Adapter- AE2500
not shown ELR - LK3JELR

[1] When current chrome stock is gone the part will no longer be chrome plated. It will be black.
[2] Serial number (21)2W2006200536 and after.

Seat Plate to Seat Guide

Picture of ELR Installed

TITAN AXS_rev. 02_17_2021

Diagrams

Home