


TITAN

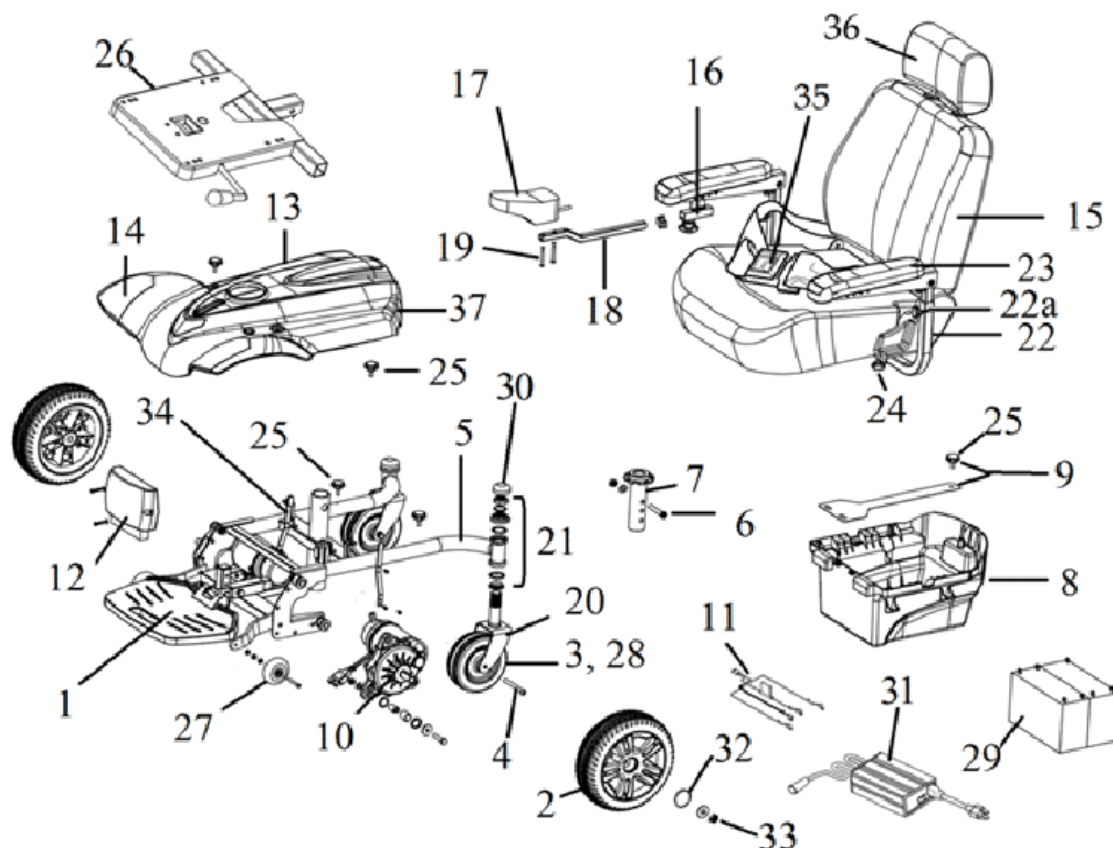

1. Footplate & Attaching Bracket- TITAN-01

Footrest Depth Adj. Bolt- TITAN-14

2. Drive Wheel Assembly- TITAN-11

3. Caster Assembly- TITAN-03

4. Caster Axle- TITAN-04

5. Frame- TITAN-05

6. Seat Post Bolt- TITAN-06

7. Seat Post (1L Serial #- TITAN-38

Seat Post (VW or 2W Serial #- TITAN-39

8. Battery Tray Assembly- TITAN-07

9. Battery Hold Down Strap- TITAN-08

10. Motor & Gear Box Left- TITAN-10

Motor & Gear Box Right- TITAN-09

Axle Key- TITAN-40

11. Battery Harness- TITAN-12

Circuit Breaker- LRA402307

12. Controller Shark- TITAN-13

Control/Shroud Cable Shark- LPW182001-03

Controller Linx- DLX-PM40-A-TITAN

13. Shroud- TITAN-16

14. Decoration Set Red- TITAN-17

Decoration Set Blue- TITAN-18

18. Joystick Arm- LDM400001-04

19. Joystick Mount Screws (pr)- LDM400001-01

20. Caster Fork- LDM400013

21. Fork Stem Bearing- LRB402348

22. Armrest Assembly Left- LDR410011L

Armrest Assembly Right- LDR410011R

22a. Armrest Height Knob- LDR410011-12

23. Arm Pad- LDR410110

Arm Pad Screws (pr)- LDR420018

24. Armrest Locking Knob-

1L Serial #- LDR410011-11

VW or 2W Serial #- S31052

25. Battery Tray/Shroud Knob-

1L Serial #- LDR410115-03

VW Serial #- VENT-86

26. Seat Plate (1L Serial #- LRC402414

Seat Plate (VW or 2W Serial #- C07-072-01001

27. Anti-Tipper Wheel (ea)-

1L Serial #- LDR420009-01

VW or 2W Serial #- TITAN-41

28. Caster Bearing- STDS1011

29. Battery (ea) (2 needed)- LRA402208

15. Captain's Seat Assembly-

16"X16"X18"- TITAN-29

16"X18"X18"- TITAN-30

18"X16"X18"- TITAN-31

18"X18"X16.5"- TITAN-19

18"X18"X18"- TITAN-20

20"X18"X18"- TITAN-21

20"X20"X18"- TITAN-32

22"X20"X19.5"- TITAN-33

Pan Seat 18"- ST501-181818

16. Joystick Bracket Knob- LDM400001-07

Joystick Bracket - TITAN-24

Joystick Bracket Sub.- LDM400001-05

17. Joystick Shark- LPW182001

Joystick Overlay Shark- DK-REMD01

Joystick/Shroud Cable Shark- LPW182001-02

Joystick Tip Shark- DC GJS0090

Joystick Boot Shark- DC GJS0091

Joystick Linx- LINX-JOYSTICK *

Joystick/Control Cable Linx- GLM-BUS150-A

Joystick Tip Linx- DC GPL55470

30. Dust Cap- LDR4110

31. Charger 3 amp- LRA402209

32. Hub Cap- LRM412202-HC

33. Drive Wheel Nut & Washer- TITAN-28

34. Clutch Handle Assembly- TITAN-26

Clutch Handle Tip- TITAN-27

35. Seat Belt- STDS855-PW

36. Headrest- ST-HEADREST

37. Rear Reflectors (pr)- TITAN-37

Not Shown-

ELR Adapter- AE2500

ELR- LK3JELR

*** Serial # VW017398 & higher
or (21)2W1704202883 & higher.**

[Diagrams](#)

[Home](#)

Titan_rev. 02/12/2021