


TRIDENT 2A Serial

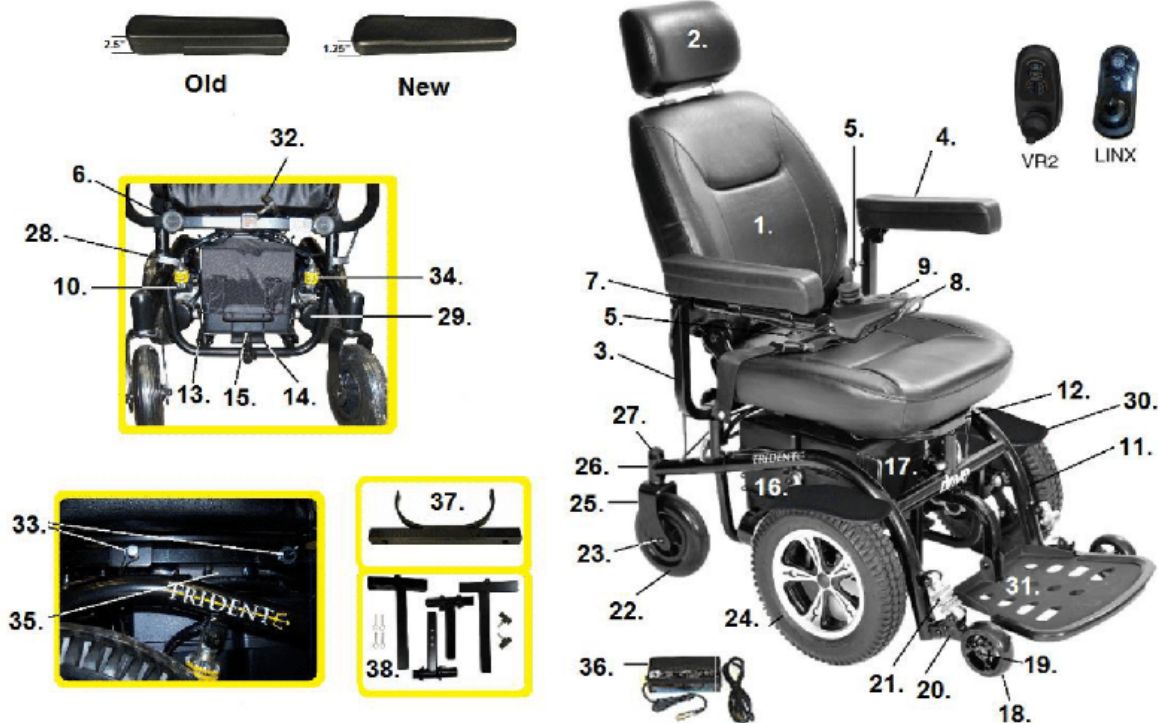

1. Seat Assembly w/Headrest 18"- TRID-2A48 [2]
Seat Assembly w/Headrest 20"- TRID-2A49 [2]
Seat Assembly w/Headrest 18"- TRID-2A01
Seat Assembly w/Headrest 20"- TRID-2A02
2. Headrest- TRID-2A39
3. Armrest Assembly Old Left- [5]
Armrest Assembly Old Right- TRID-2A03 [6]
Armrest Assembly New Left- TRID-2A04N [3]
Armrest Assembly New Right- TRID-2A03N [3]
4. Arm Pad- TRID-2A05
Arm Pad Screws (pr)- TRID-2A51
Arm Pad- TRID-2A05N [3]
Arm Pad Screws (pr)- TRID-2A51N [3]
Joystick Cable Clips (3 ea)- TRID-2A46
5. Knob Joystick or A/R Height- TRID-2A43
6. Knob Armrest Width- TRID-2A44
7. Joystick Mounting Bracket- TRID-2A38
8. Joystick Fixed Arm- TRID-2A36
Joystick Swing-Away Arm- TRID-2A37
Joystick Fixed Arm Linx- TRID-2A36-LINX
Joystick Swing-Away Arm Linx- TRID-2A37-LINX
9. Joystick VR2- S350169701
Joystick Linx- LINX-JOYSTICK
Joystick Tip Linx- DC GPL55470
Joystick/Cable Linx- CUM-BUS450-A [4]
10. Motor & Gear Box VR2 Left- TRID-2A29
Motor & Gear Box VR2 Right- TRID-2A30
Motor Brush & Cap (pr)- TRID-2A31
Motor & Gear Box Left Linx- TRID-2A29-LINX [4]
Mtr & Gear Box Right Linx- TRID-2A30-LINX [4]
Motor Brush Linx (ea)- CP5201 [4]
Motor Brush Cap Linx (ea)- CP5202 [4]
Motor Key Way- TRID-2A52
11. Wheel Guard Left - TRID-2A41
Wheel Guard Right- TRID-2A42
12. Footrest & Attaching Bolt- TRID-2A07
Footrest Bolt- TRID-2A08
Footrest Bracket & Knob- TRID-2A09
13. Accessory Knob- TRID-2A44
14. Casters (4 ea)- TRID-2A13
Caster Bearing- STDS1011
15. Drive Wheel w/Nut & Washer- TRID-2A15
16. Fork- TRID-2A10
17. Fork Stem Bearing- TRID-2A11
18. Fork Stem Cap- TRID-2A12
19. Height Adjustment Pin (3 ea)- TRID-2A28
20. Anti-Tipper Articulating Arm- TRID-2A18
21. Anti-Tipper Shock Absorber- TRID-2A19
22. Rear Caster- TRID-2A13
23. Caster Axle- TRID-2A14
24. Drive Wheel w/Nut & Washer- TRID-2A15
25. Fork- TRID-2A10
26. Fork Stem Bearing- TRID-2A11
27. Fork Stem Cap- TRID-2A12
28. Height Adjustment Pin (3 ea)- TRID-2A28
29. Motor & Gear Box VR2 Left- TRID-2A29
Motor & Gear Box VR2 Right- TRID-2A30
Motor Brush & Cap (pr)- TRID-2A31
Motor & Gear Box Left Linx- TRID-2A29-LINX [4]
Mtr & Gear Box Right Linx- TRID-2A30-LINX [4]
Motor Brush Linx (ea)- CP5201 [4]
Motor Brush Cap Linx (ea)- CP5202 [4]
Motor Key Way- TRID-2A52
30. Wheel Guard Left - TRID-2A41
Wheel Guard Right- TRID-2A42
31. Footrest & Attaching Bolt- TRID-2A07
Footrest Bolt- TRID-2A08
Footrest Bracket & Knob- TRID-2A09
32. Accessory Knob- TRID-2A44
33. Casters (4 ea)- TRID-2A13
Caster Bearing- STDS1011
34. Drive Wheel w/Nut & Washer- TRID-2A15
35. Fork- TRID-2A10
36. Fork Stem Bearing- TRID-2A11
37. Fork Stem Cap- TRID-2A12
38. Height Adjustment Pin (3 ea)- TRID-2A28

- Joystick/Cntrl Cable Linx- GLM-BUS150-A [4]
10. Motor Mount Arm- TRID-2A20
 11. Main Frame- TRID-2A22
 - Upper Frame- TRID-2A50 [2]
 13. Battery Tray Slides (pr)- TRID-2A25
 14. Battery Tray- TRID-2A24
 15. Battery Tray Release Pin- TRID-2A26
 16. Battery Case 1 Plug- TRID-2A32
 - Battery Case 2 Plug- TRID-2A33
 - Battery Case 1 Plug Linx- TRID-2A32
 - Battery Case 2 Plug Linx- TRID-2A54-LINX [4]
 - Battery Cases (pr) Linx- TRID-2A53-LINX [4]
 - Battery (2 needed)- LRA402208
 - Circuit Breaker- LRA402307
 17. Battery Box Hold Down Cover- TRID-2A45
 - Battery Strap- TRID-2A27
 18. Anti-Tipper Wheel- TRID-2A16
 19. Anti-Tipper Axle- TRID-2A17

33. Seat Brackets (4 ea)- TRID-2A00
34. Motor Mount Shock Absorber- TRID-2A21
 - Shock Absorber Hardware- TRID-2A34
35. Controller- TRID-2A35
 - Controller Linx- TRID-2A35-LINX [4]
36. Charger- CPC
 - Fuse for CPC Charger- CPCFUSE
 - When stock is gone use LRA402210
37. Seat Plate Adapter Bracket- TRID-2A47 [2]
38. ELR Adapter Bracket- SFELR [1]

[1] LK3JELR use with 37 & 38.

[2] Serial # 2A1406001520 and later.

[3] Serial # 2A1707600001 and later.

[4] Serial # 2A1608600105 and later.

[5] When stock is gone the part is no longer available.

[6] Stock is gone the part is no longer available.

[Diagrams](#)

[Home](#)