


TRIDENT HD

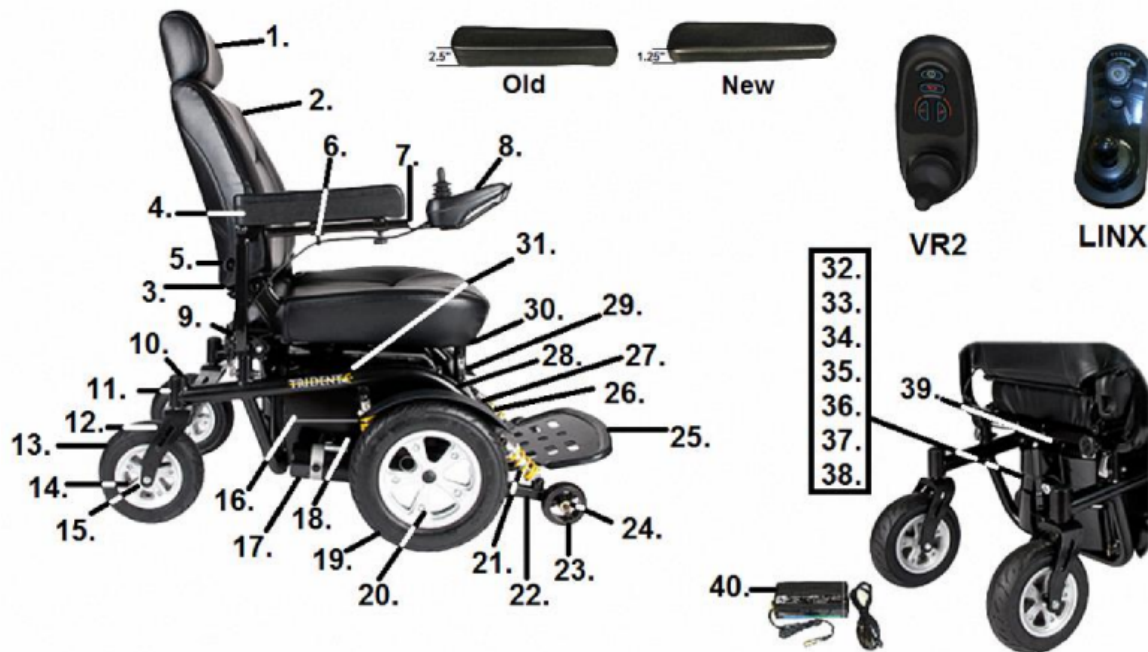

1. Headrest- TRIDHD-26
2. 22" Seat- TRIDHD-07
24" Seat- TRIDHD-08
22" Pan Seat Assembly- TRIDHD-33
24" Pan Seat Assembly- TRIDHD-34
3. Armrest Assembly Left- TRIDHD-30
Right- TRIDHD-29 ***
*** (w/o Joystick Bracket)
Armrest Assembly Left- TRIDHD-30N [3]
Right- TRIDHD-29N [3]
4. Arm Pad- TRID-2A05
Arm Pad Screws (pr)- TRID-2A51
Arm Pad- TRID-2A05N [3]
Arm Pad Screws (pr)- TRID-2A51N [3]
5. Armrest Height Knob- TRID-2A43
6. Joystick Mount- TRID-2A38
7. Joystick Arm VR2- TRID-2A36
Linx- TRID-2A36-LINX [1]
8. Joystick VR2- S350169701
Linx- LINX-JOYSTICK [1]
Joystick Tip Linx- DC GPL55470
Extension Cable- GLM-BUS150-A [2]
9. Armrest Width Knob- TRID-2A44
10. Dust Cap- TRIDHD-04
11. Fork Stem Bearing- TRIDHD-05
Sub.- STDS2050
12. Fork- TRIDHD-01
13. Fork Stem Bearing- TRIDHD-05
14. Fork Stem Bearing- TRIDHD-05
15. Fork Stem Bearing- TRIDHD-05
16. Drive Wheel Complete- TRIDHD-06
(includes mounting nut & washer)
17. Anti-Tipper Shock Absorber- TRID-2A19
18. Anti-Tipper Arm- TRID-2A18
19. Anti-Tipper Wheel- TRID-2A16
20. Anti-Tipper Axle- TRID-2A17
21. Footrest & Bolt- TRIDHD-21
22. Footrest Bracket & Knob- TRIDHD-22
23. Fender (Left or Right)- TRIDHD-19
24. Main Frame- TRIDHD-10
25. Height Adjustment Pins (3 ea)- TRID-2A28
26. Upper Frame- TRIDHD-11
27. Seat Latch- TRIDHD-20
28. Rear Battery Cover- TRIDHD-25
29. Battery Case
VR2 Pair- TRIDHD-15
1 Plug- TRIDHD-15-1
2 Plug- TRIDHD-15-2
Linx Pair- TRID-2A53-LINX
1 Plug- TRIDHD-15-1
2 Plug- TRIDHD-15-2-LINX
30. Battery Tray- TRIDHD-13
31. Battery Tray Slides- TRIDHD-12
32. Battery Tray Release Pin- TRIDHD-14
33. Battery Tray Hold Down Bolt- TRIDHD-16
34. Battery (2 needed)- LRB402308

- 13. Caster Tire Only- TRIDHD-24
- 14. Caster Complete- TRIDHD-02
 - Bearings are not replaceable.
- 15. Caster Axle- TRIDHD-03
- 16. Motor Shock Absorber- TRIDHD-23
- 17. Motor/Gearbox Assembly
 - VR2 Left- TRIDHD-17
 - VR2 Right- TRIDHD-18
 - Linx Left- TRIDHD-17-DY [1]
 - Linx Right- TRIDHD-18-DY [1]
- Motor Brush (ea) VR2 & Linx- SF5012
- Axle Keyway- TRID-2A52
- 18. Motor Mount- TRIDHD-09
- 19. Drive Tire Only- TRIDHD-27

- 39. Controller VR2- TRIDHD-28
 - Linx- TRIDHD-28-LINX
 - Battery to Controller Cable- TRIDHD-31
- 40. Charger- CPC
 - Fuse for CPC Charger- CPCFUSE
 - When Stock is gone use LRA402210.
- ELR Adapter kit
- Seat Plate Bracket- TRID-2A47
- ELR Bracket- SFELR
- ELR- LK3JELR

- [1] Ser # 2A160800105 & later
- [2] Linx Joystick Only
- [3] Ser # 2A1707600001 & later

[Diagrams](#)

[Home](#)

rruent HD_rev. 01_28_2022