


TITAN LTE

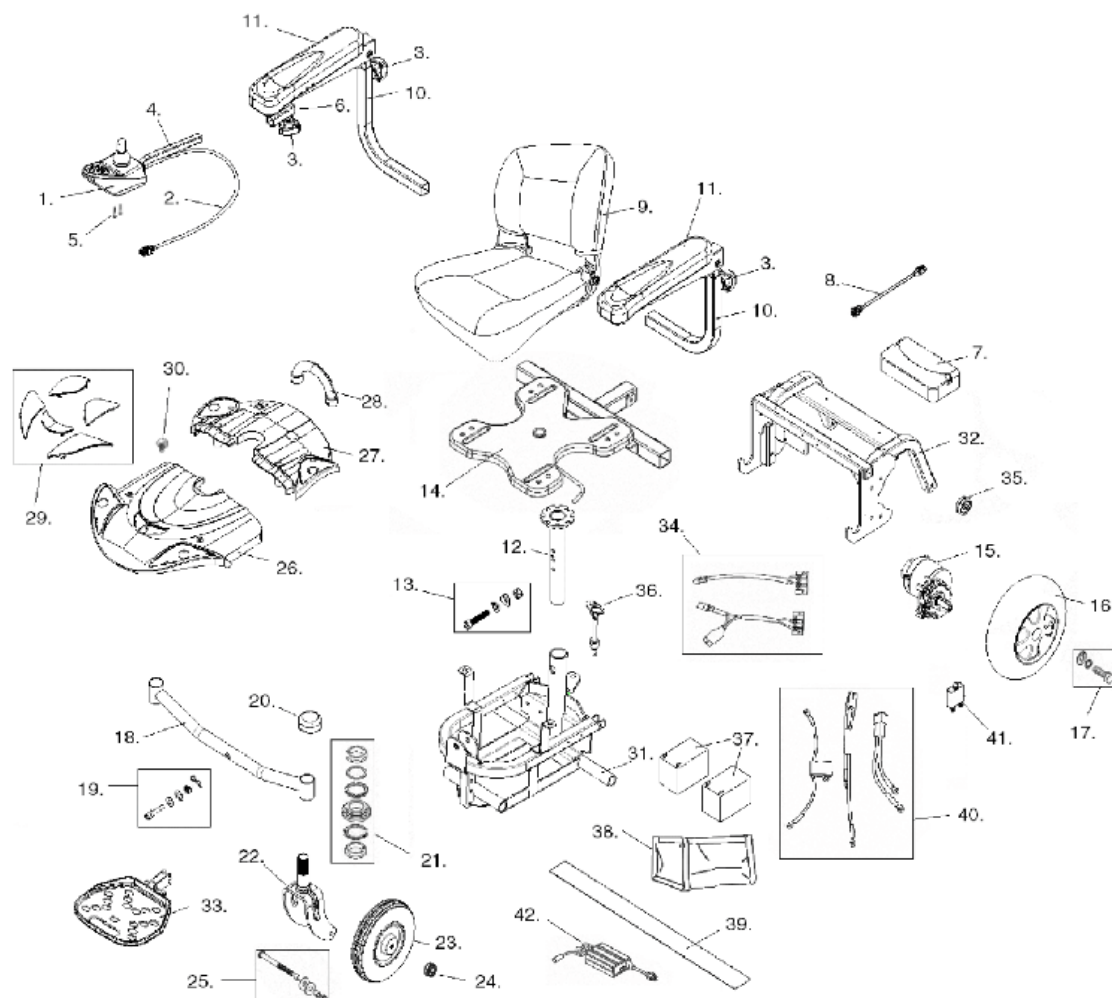

1. Joystick- LINX-JOYSTICK
Joystick Tip Linx- DC GPL55470
2. Joystick Cable- GLM-BUS100-A
3. Locking Knob- LDM400001-07
4. Joystick Arm- LDM400001-04
5. Joystick Mtg Screw & Washer- LDM400001-01
6. Joystick Arm Mtg Bracket- LDM400001-05
7. Controller- DLX-PM40-A-LTE
8. Controller to Shroud Cable- GLM-EXT064-A
Controller Cable Clamp- C10-085-00500
9. Seat Assembly w/o Arms- DST-013
10. Armrest Assembly Left- LDR410011L
Armrest Assembly Right- LDR410011R
11. Arm Pad- LDR410110
12. Seat Post *- C07-069-00201
13. Seat Post Bolt- TITANLTE-11
14. Seat Plate- C07-072-01001
15. Motor Gearbox Brake Left- C00-085-00200

19. Articulating Bar Mtg Hdwr- TITANLTE-13
20. Dust Cover- C26-084-00100
21. Fork Stem Bearing Assembly- DST-010
22. Fork - C03-017-00800
23. Caster- C05-085-00100
24. Caster Bearing- SEC9031-B
25. Caster Axle- TITANLTE-03
26. Front Shroud- C08-085-00100
27. Rear Shroud- C08-085-00500
28. Handle- TITANLTE-16
29. Red Decorations Set- TITANLTE-07
Blue Decorations Set- TITANLTE-08
30. Shroud Knob- VENT-86
31. Front Frame- C01-085-00100
32. Rear Frame- C01-085-00200
33. Footrest & Mount- TITANLTE-09
34. Contact Block Assy & Hrwr- TITANLTE-14
35. Anti-Tipper Assembly- TITANLTE-06

15. Motor, Gearbox, Brake Left- C09-085-00200
Motor, Gearbox, Brake Right- C09-085-00100
Brake Only Left- C09-085-00700
Brake Only Right- C09-085-00800
Motor Key Way- C88-085-00100
16. Rear Wheel- S31132B-HD
17. Rear Wheel Mtg Bolt & Washers- TITANLTE-04
18. Articulating Bar- TITANLTE-01

35. Anti-Tipper Assembly- TITANLTE-00
36. Frame Lock Bolt Assembly- TITANLTE-05
37. Battery 12V 20Ah (2 needed)- LRM412123
38. Battery Bag- C17-085-00100
39. Battery Strap- C31-005-00400
40. Wire Harness- TITANLTE-12
41. Circuit Breaker 20 Amp- LRM302107
42. Charger (2 amp)- LRM302111

*** When current stock is gone the part will no longer be chrome plated. It will be black.**

[Diagrams](#)

[Home](#)

in LTE_rev. 08_27_2020