


VENTURA 4

1. Forward-Reverse Assembly- VENT-01
Throttle Lever Only- VENT-87
2. Face Plate Assembly Folding Seat- VENT-04
Face Plate Assembly Capt. Seat- VENT-05
3. Speed Dial Assembly- S31001
Speed Dial Knob- LRM412103-11
4. Battery Gauge
LED- C11-076-00100
Analog- C11-038-00101
5. Turn Signal Switch Capt. Seat- VENT-72
6. Head Light Switch- S31122
Head Light Switch Sub- GT8072W38
Headlight Switch Cover- GT8072W43
7. Fast/Slow Switch- GT8072W41
8. Horn Switch- S31012
Horn Switch Cover- GT8072W43
9. Flasher Switch- GT8072W32
10. Turn Signal Relay Capt. Seat- VENT-76
11. Key Switch- S41160
Key- S4KEY
12. Handle Bar Assembly- VENT-03
13. Mirror Right- VENT-53
Mirror Left- VENT-54
14. Turn Signal Left- GT8072W35
Turn Signal Right- GT8072W36
Turn Signal Bulb- GT8072W33
15. Basket- S35003-PL
16. Basket Holder- VENT-55
17. Front Decoration Red- VENT-61
Front Decoration Blue- VENT-62
18. Front Tiller Cover- VENT-09
19. Tiller Switch Assembly- VENT-02
20. Rear Tiller Cover- VENT-07
21. Charger Socket- S31147
22. Head Light Assembly- VENT-44
23. Front Shroud- VENT-10
24. Stem Bending Set- VENT-13
25. Steering Axle Assembly- VENT-27
26. Front Fork Assembly- VENT-23
27. Steering Rod Assembly- VENT-26
28. Front Wheel Assembly- VENT-74
Front Wheel Pneumatic- VENT-56
Pneumatic Tire Only- VENT-59
Inner Tube- VENT-60
29. Hub Cap- VENT-70
30. Rear Axle Bolt- VENT-71
31. Front Bumper Assembly- VENT-14
32. Front Frame- VENT-17
33. Controller- VENT-08
34. Middle Shroud- VENT-11
35. Rear Frame- VENT-15
36. Shroud Kit Red (screw on)- [1]
Shroud Kit Blue (screw on)- [1]
Shroud Kit Red (snap on)- VENT-78 [3]
Shroud Kit Blue (snap on)- VENT-79 [3]
37. Shock Absorber- VENT-15

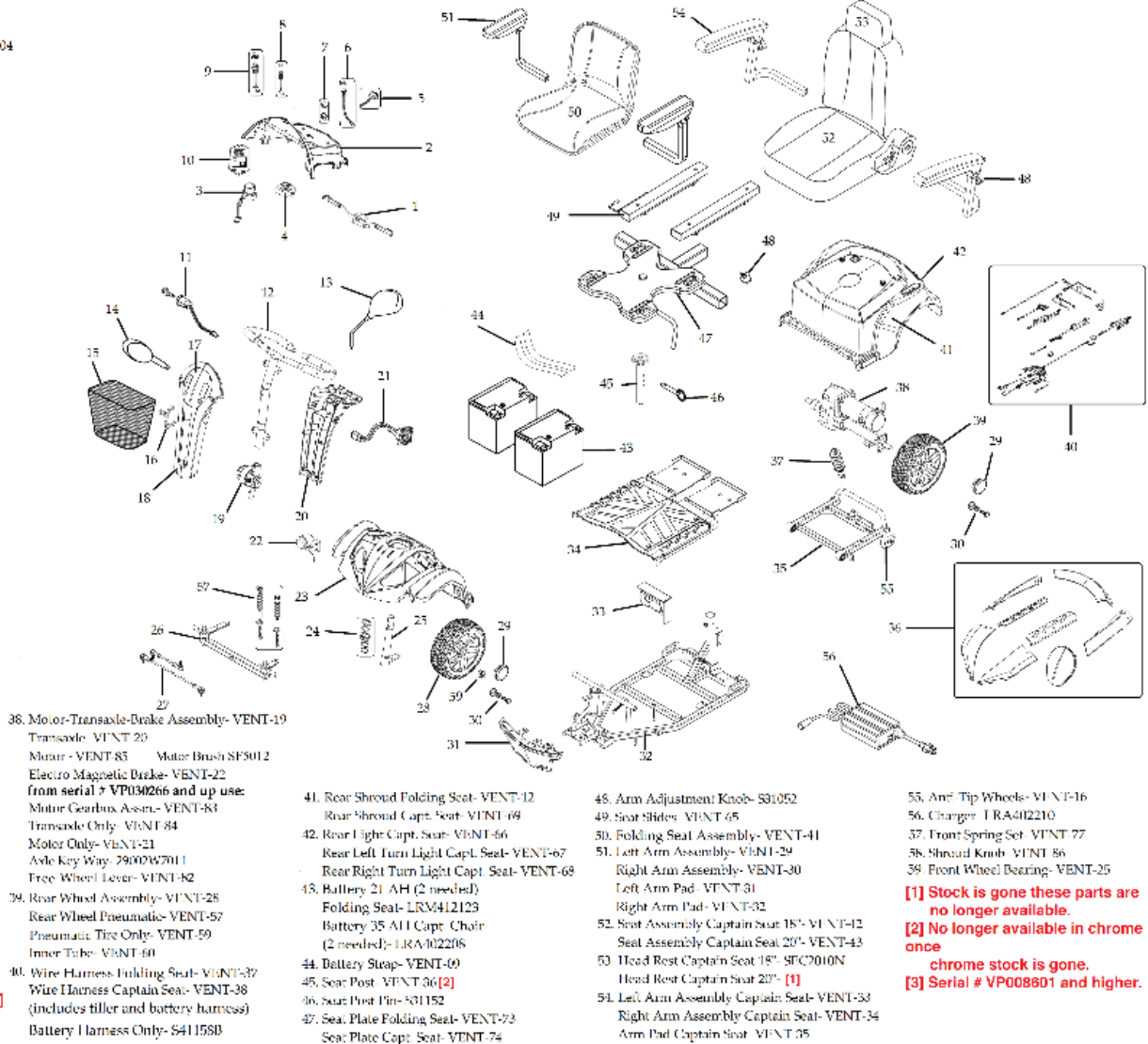

38. Motor-Transaxle-Brake Assembly- VENT-19
Transaxle- VENT-20
Motor- VENT-85 Motor Brush SF8012
Electro Magnetic Brake- VENT-22
from serial # VP030266 and up use:
Motor Gearbox Assm- VENT-83
Transaxle Only- VENT-84
Motor Only- VENT-21
Axle Key Way- 79007W7011
Free Wheel Lever- VENT-82
39. Rear Wheel Assembly- VENT-25
Rear Wheel Pneumatic- VENT-57
Pneumatic Tire Only- VENT-59
Inner Tube- VENT-60
40. Wire Harness Folding Seat- VENT-37
Wire Harness Captain Seat- VENT-38
(includes tiller and battery harness)
Battery Harness Only- S41158B

41. Rear Shroud Folding Seat- VENT-12
Rear Shroud Capt. Seat- VENT-69
42. Rear Light Capt. Seat- VENT-66
Rear Left Turn Light Capt. Seat- VENT-67
Rear Right Turn Light Capt. Seat- VENT-68
43. Battery 21 AH (2 needed)
Folding Seat- LRM412123
Battery 35 AH Capt. Chair
(2 needed)- LRM402206
44. Battery Strap- VENT-09
45. Seat Post- VENT-36 [2]
46. Seat Post Pin- S31152
47. Seat Plate Folding Seat- VENT-73
Seat Plate Capt. Seat- VENT-74
48. Arm Adjustment Knob- S31052
49. Seat Slides- VENT-65
50. Folding Seat Assembly- VENT-41
51. Left Arm Assembly- VENT-29
Right Arm Assembly- VENT-30
Left Arm Pad- VENT-31
Right Arm Pad- VENT-32
52. Seat Assembly Captain Seat 18"- VENT-42
Seat Assembly Captain Seat 20"- VENT-43
53. Head Rest Captain Seat 18"- SFC2010N
Head Rest Captain Seat 20"- [1]
54. Left Arm Assembly Captain Seat- VENT-33
Right Arm Assembly Captain Seat- VENT-34
Arm Pad Captain Seat- VENT-35

55. Anti-Tip Wheels- VENT-16
 56. Charger- TRA402210
 57. Front Spring Set- VENT-77
 58. Shroud Knob- VENT-86
 59. Front Wheel Bearing- VENT-25
- [1] Stock is gone these parts are no longer available.
[2] No longer available in chrome once chrome stock is gone.
[3] Serial # VP008801 and higher.

02/17/2021

Diagrams

Home