


BOBCCAT 4

1. Throttle- ~~BC31009~~ (NLA)
Throttle New- ~~BC31009~~ (NLA)*
Sub.- S12-069-00300
Throttle Harness Adapter- ~~BC31048~~ (NLA)**
2. Speed Dial- BC31001
Speed Dial Knob- LRM412103-11
3. Battery Gauge- BC31208B
4. Upper Tiller Cover- BC31004
5. Key Switch- BC31006
Key- BCKEY
6. Lower Tiller Cover Blue- ~~BC31208B~~ (NLA)
Red- ~~BC31209~~ (NLA)
7. Tiller Swivel Assembly- BC350109 *****
S02-050-00200 *****
8. Front Shroud- ~~BC350106~~ (NLA)
9. Rear Shroud Blue- BC350111B
Red- ~~BC350111R~~ (NLA)
10. Rear Frame- BC31130
11. Front Frame- BC350108
12. Transaxle/Motor/Brake- BC351125
Transaxle Only- BC350112
Motor Only- BC31125
Brake Only- BC310EB
13. Battery Case w/Batteries- BC31133
Battery Case Only- BC31140
14. Battery Harness w/Jumper- BC31068
Fuse 40 amp- BC40AFUSE
Fuse 5 Amp- BC5AFUSE
15. Charger Port & Cable- BC31147 *****
16. Batteries (2 needed)- LRM402109
17. Tiller Bearing Set- BC31021
18. "L" Steering Axle- BC350101(NLA)
19. Tie Rod Set- BC350103
20. Front Wheel Black Rim- BC350100
Gray Rim- BC350100-G (NLA)***
21. Seat- BC35045
22. Armrest Left- BC35040L
Right- BC35040R
Arm Pad- BC953014
23. Adjustment Knob- S31052
24. Seat Post- BC31049
25. Seat Post Pin- S31152
26. Seat Plate- ~~BC31049~~ (NLA)
27. Wire Harness Front- BC31048
Fuse 10 Amp- SF10FBF

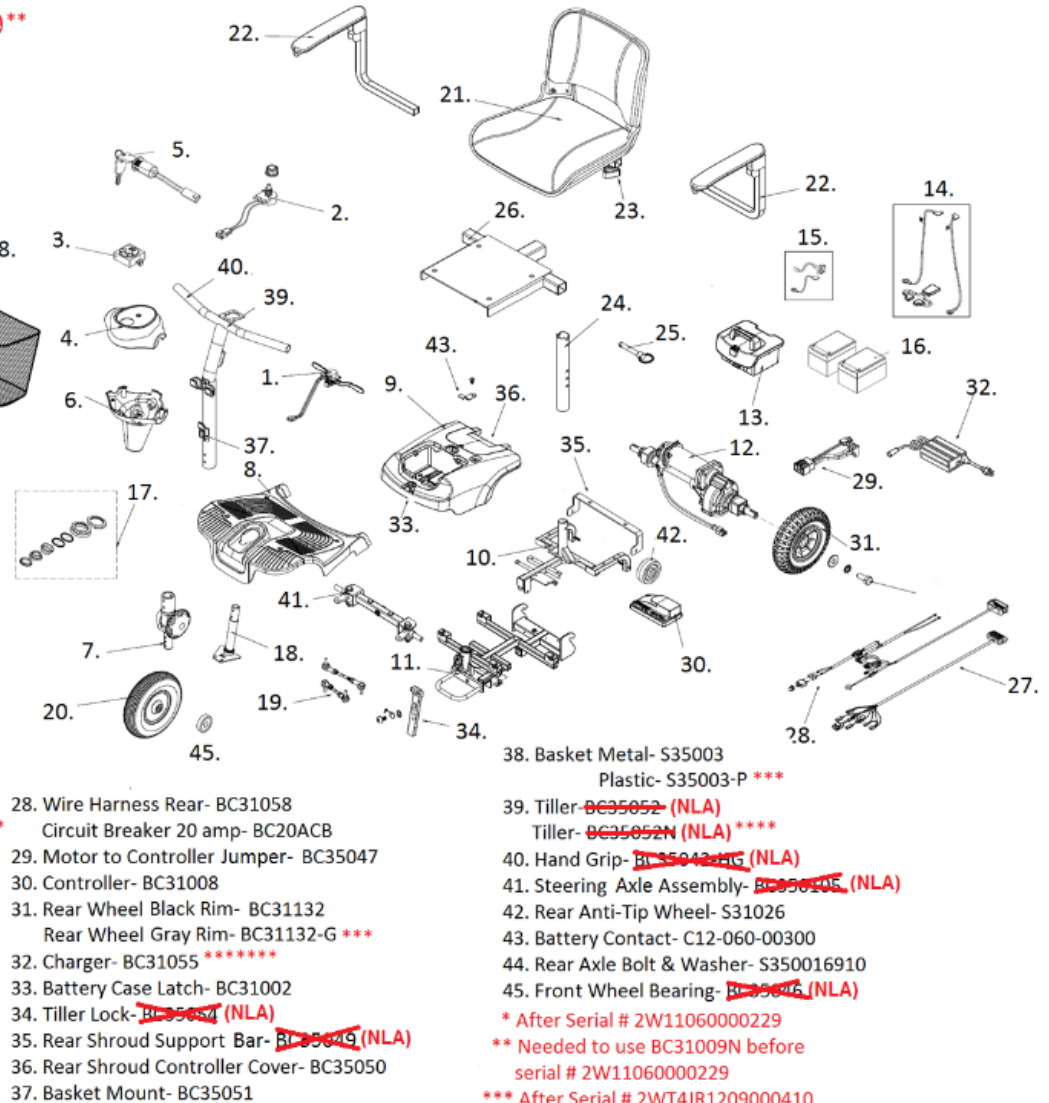

28. Wire Harness Rear- BC31058
Circuit Breaker 20 amp- BC20ACB
29. Motor to Controller Jumper- BC35047
30. Controller- BC31008
31. Rear Wheel Black Rim- BC31132
Rear Wheel Gray Rim- BC31132-G ***
32. Charger- BC31055 *****
33. Battery Case Latch- BC31002
34. Tiller Lock- ~~BC35049~~ (NLA)
35. Rear Shroud Support Bar- ~~BC35049~~ (NLA)
36. Rear Shroud Controller Cover- BC35050
37. Basket Mount- BC35051

38. Basket Metal- S35003
Plastic- S35003-P ***
39. Tiller- ~~BC35052~~ (NLA)
Tiller- ~~BC35052N~~ (NLA) ****
40. Hand Grip- ~~BC35048~~ (NLA)
41. Steering Axle Assembly- ~~BC35045~~ (NLA)
42. Rear Anti-Tip Wheel- S31026
43. Battery Contact- C12-060-00300
44. Rear Axle Bolt & Washer- S350016910
45. Front Wheel Bearing- ~~BC35045~~ (NLA)

* After Serial # 2W11060000229
** Needed to use BC31009N before serial # 2W11060000229
*** After Serial # 2WT4JR1209000410
**** After Serial # 2WT1JB1106000389
***** Before Serial # VP006688
***** After Serial # VP006688
***** When stock is gone the part is NLA

12/03/20

Diagrams

Home