


BOBCAT X3

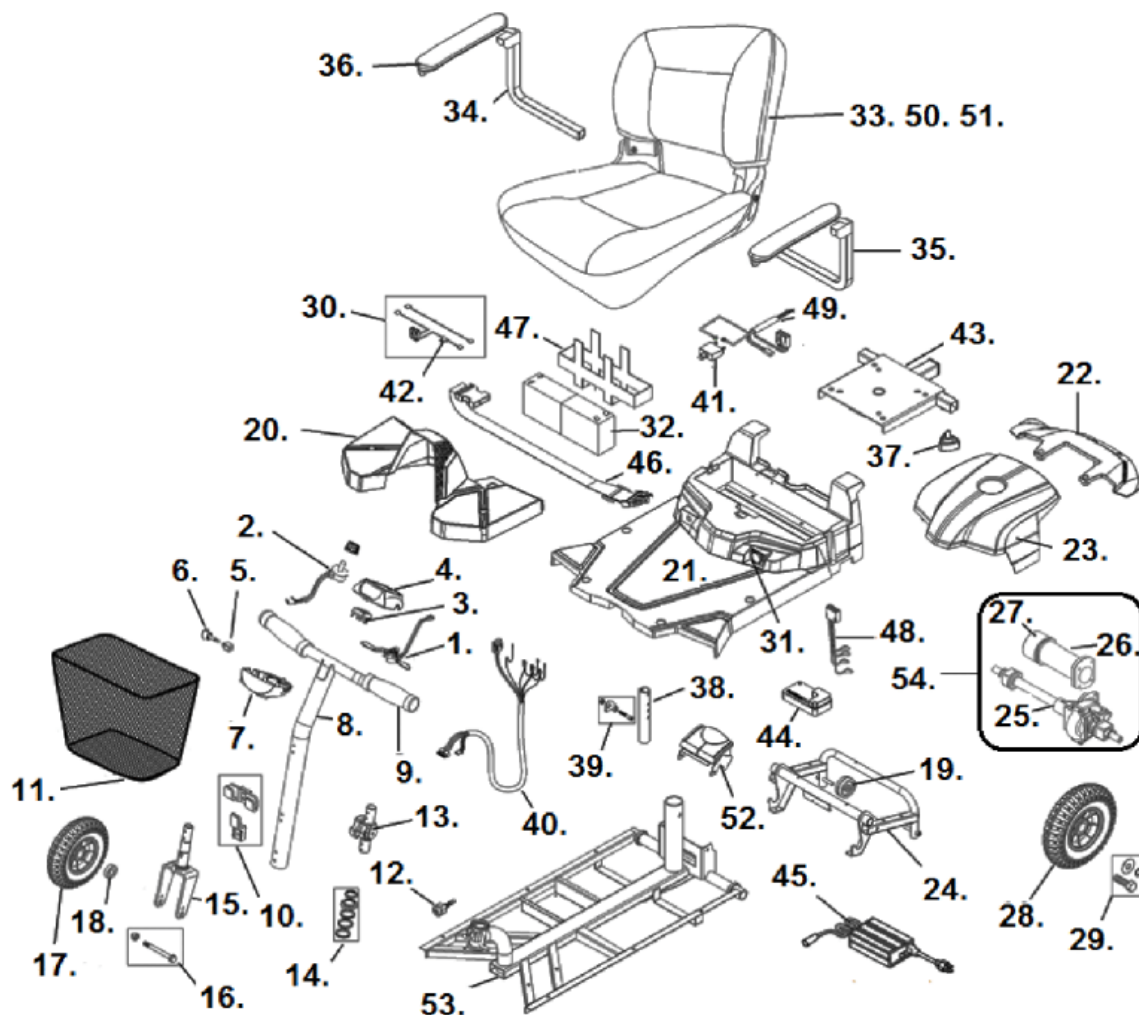

1. Throttle Old- *

New- S12-069-00300

* Stock is gone use new style.

2. Speed Dial- C12-071-00100

3. Battery Gauge- BC31208B

4. Upper Tiller Cover- C08-050-00301

5. Key Switch- C18-071-00100

6. Key- C18-071-00200

7. Lower Tiller Cover- C08-071-00100

8. Tiller- C02-071-00100

9. Hand Grip Foam- S31003

TPR Rubber- C20-007-00200

10. Basket Mounting Set- BC35051

11. Basket- S35003-P

12. Tiller Lock Set- C08-069-01000

27. Electromagnetic Brake- C09-071-00300

28. Rear Wheel- BC31132

29. Axle Bolt & Washer- S350016910

30. Battery Harness & Jumper- C18-071-00600

31. Charger Port- C18-071-00800

32. Battery (2 needed)- BATT12V10AH

33. Seat/Seat Plate (no arms)- S07-071-00100

34. Right Armrest Assembly- C07-071-00400

35. Left Armrest Assembly- C07-071-00300

36. Arm Pad & Screws- C07-071-00500

37. Adjustment Knob- S31052

38. Seat Post **- C07-071-00100

39. Seat Post Bolt Assembly- C07-071-00101

40. Front Wire Harness- C18-071-00300

41. 20 Amp Circuit Breaker- BC20ACB

- 12. Tiller Lock Set- C02-069-00401
- 13. Tiller Swivel Assembly- C02-069-00401
- 14. Tiller Bearing Set- C29-016-00301
- 15. Fork- C03-070-00100
- 16. Front Axle Assembly- ***
- 17. Front Wheel- BC31142
- 18. Front Wheel Bearing- 4900WB **
- 19. Anti-Tipper Wheel- S31026
- 20. Front Shroud- C08-071-00300
- 21. Footrest Shroud- C08-071-00200
- 22. Rear Shroud- C08-071-00400
- 23. Battery Cover Red- C08-071-00512
- 24. Rear Frame- C01-070-00300
- 25. Transaxle- BC350112
- 26. Motor- C09-071-00200
- Motor/Brake Assembly-C09-071-00100

- 41. 20 Amp Circuit Breaker- BC20ACB
- 42. 40 Amp Fuse- BC40AFUSE
- 43. Seat Plate- C07-050-00203-C
- 44. Controller- C10-034-00100
- 45. Charger 2 Amps- LRM302111
- 46. Battery Hold Down Strap- C17-071-00200
- 47. Battery Cradle- C17-071-00100
- 48. Motor Cable- C18-071-00400
- 49. Power Cable Set- C18-071-00500
- 50. Seat Upholstery- C07-071-00600
- 51. Back Upholstery- C07-071-00700
- 52. Frame Latch- C08-069-02900-C
- 53. Front Frame- C01-070-00102
- 54. Motor/Brake/Transaxle Assembly- S09-071-00100

**** When current stock is gone the part will no longer be chrome plated. It will be black.**

***** No longer available.**

[Diagrams](#)

[Home](#)

cat X3_rev. 01_06_2021