


BOBCAT X4

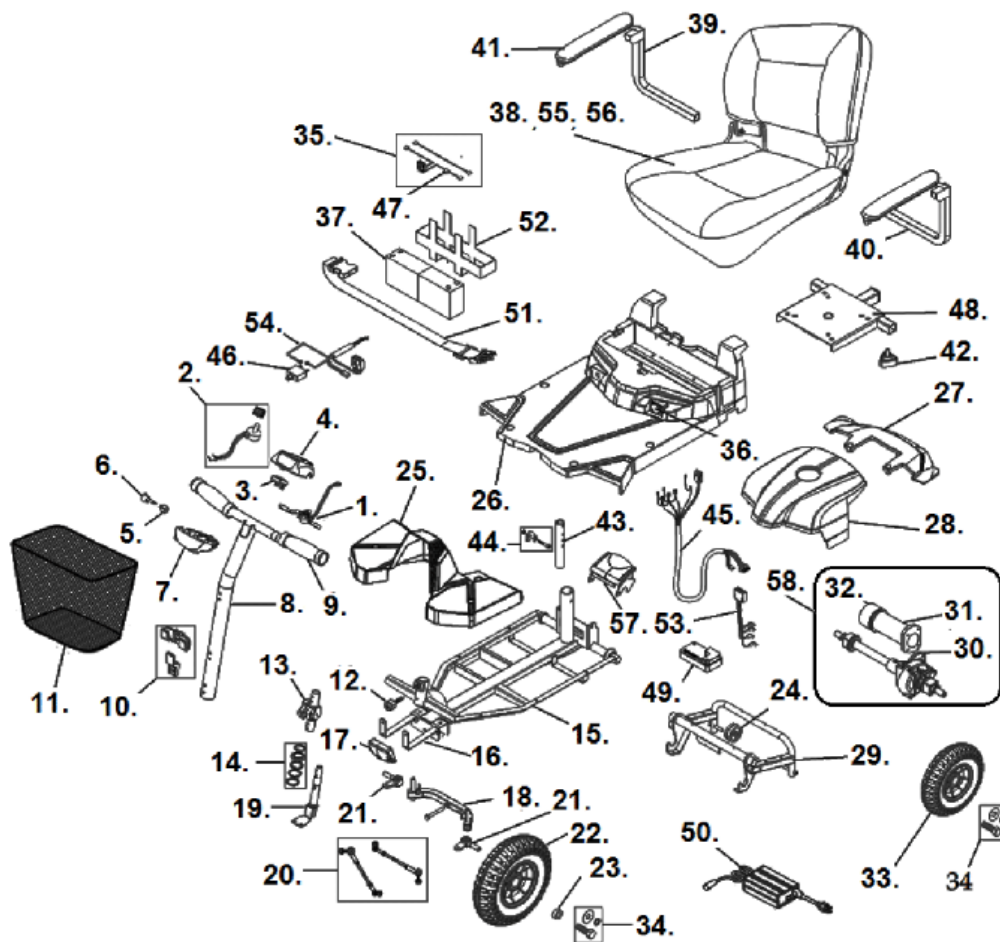

1. Throttle Old- *

New- S12-069-00300

* Stock is gone use new style.

2. Speed Dial- C12-071-00100

3. Battery Gauge- BC31208B

4. Upper Tiller Cover- C08-050-00301

5. Key Switch- C18-071-00100

6. Key- C18-071-00200

7. Lower Tiller Cover- C08-071-00100

8. Tiller- C02-071-00100

9. Hand Grip Foam- S31003

TPR Rubber- C20-007-00200

10. Basket Mounting Set- BC35051

11. Basket- S35003-P

12. Tiller Lock Set- C08-069-01000

13. Tiller Swivel Assembly- C02-069-00401

30. Transaxle- BC350112

31. Motor- C09-071-00200

Motor & Brake Assembly- C09-071-00100

32. Electromagnetic Brake- C09-071-00300

33. Rear Wheel- BC31132

34. Axle Bolt & Washer- S350016910

35. Battery Harness & Jumper- C18-071-00600

36. Charger Port- C18-071-00800

37. Battery (2 needed)- BATT12V10AH

38. Seat/Seat Plate (no arms)- S07-071-00100

39. Right Armrest Assembly- C07-071-00400

40. Left Armrest Assembly- C07-071-00300

41. Arm Pad & Screws- C07-071-00500

42. Adjustment Knob- S31052

43. Seat Post **- C07-071-00100

44. Seat Post Bolt Assembly- C07-071-00101

13. Tiller Swivel Assembly- C02-069-00401
14. Tiller Bearing Set- C29-016-00301
15. Front Frame- C01-069-00103
16. Front Bumper Bracket- C01-071-00100
17. Front Bumper- C08-069-04800
18. Front Fork- S03-069-00200
19. "L" Steering Wheel- C03-069-00101
20. Tie Rods- BC350103
21. Steering Axle- C03-010-00305
22. Front Wheel- BC350100
23. Front Wheel Bearing- **
24. Anti-Tipper Wheel- S31026
25. Front Shroud- C08-071-00600
26. Footrest Shroud- C08-071-00200
27. Rear Shroud- C08-071-00400
28. Battery Cover Red- C08-071-00512
29. Rear Frame- C01-070-00300

44. Seat Post Bolt Assembly- C07-071-00101
45. Front Wire Harness- C18-071-00300
46. 20 Amp Circuit Breaker- BC20ACB
47. 40 Amp Fuse- BC40AFUSE
48. Seat Plate- C07-050-00203-C
49. Controller- C10-034-00100
50. Charger 2 Amps- LRM302111
51. Battery Hold Down Strap- C17-071-00200
52. Battery Cradle- C17-071-00100
53. Motor Cable- C18-071-00400
54. Power Cable Set- C18-071-00500
55. Seat Upholstery- C07-071-00600
56. Back Upholstery- C07-071-00700
57. Frame Latch- C08-069-02900-C
58. Motor/Brake/Transaxle Assembly- S09-071-00100

**** When current stock is gone the part will no longer be chrome plated. It will be black.**

**** No longer available.**

[Diagrams](#)

[Home](#)

at X4_rev. 12_07_2020