


COBRA GT4

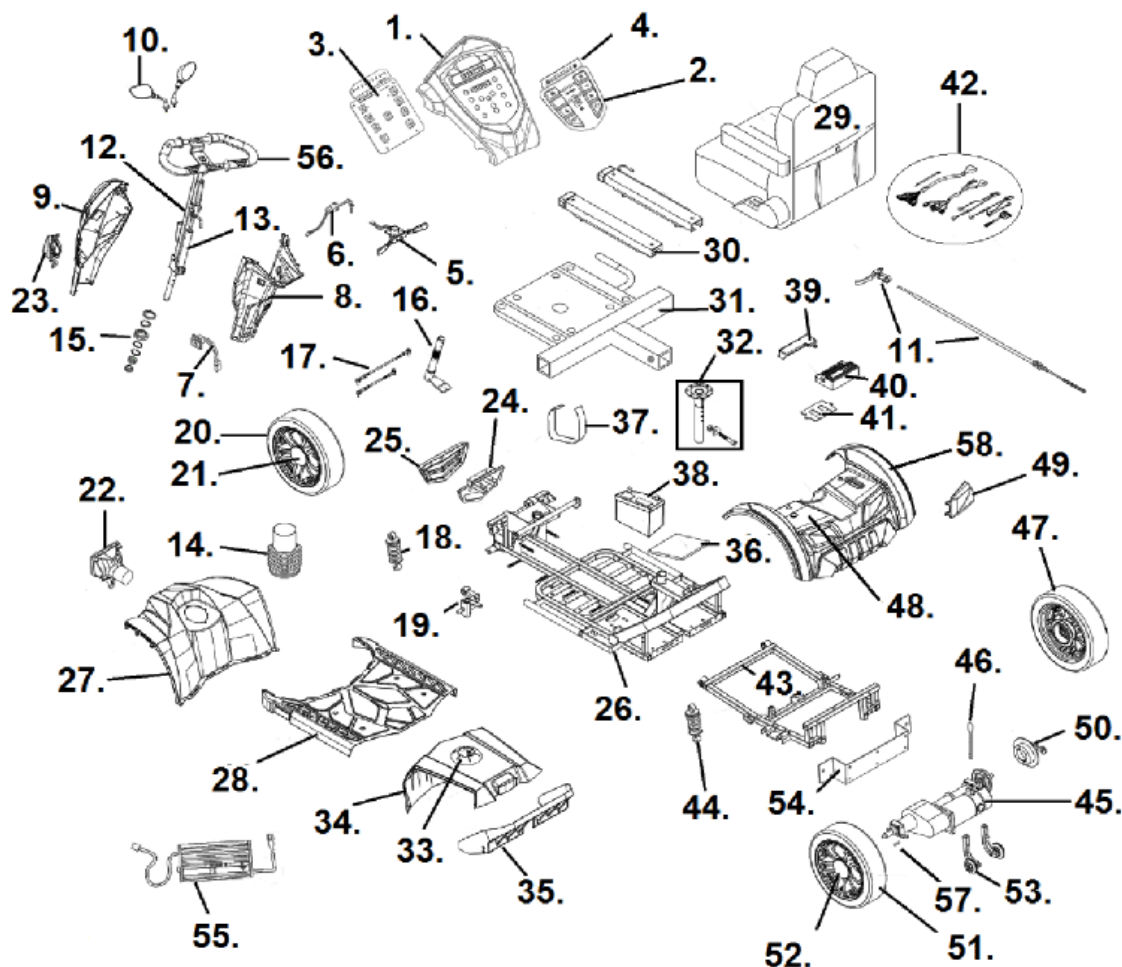

1. Upper Tiller Cover- S08-062-00165
2. Control Panel Overlay- C10-062-00200
3. Tiller PC Board- C10-051-00507
4. Battery Indicator Overlay- C10-062-00300
5. Throttle- S12-051-00100
6. Key Switch- C18-001-01501
Key - C37-001-02300
7. Charger Port- C12-051-00200
8. Rear Tiller Cover- S08-062-00300
9. Front Tiller Cover- S08-062-00477
10. Mirror Set- S37-062-00177
11. Hand Brake Assembly- S35-062-00100
Brake Handle- C35-045-00300
Cable Assembly- S35-062-00300
Brake Cable- C35-062-00101
12. Delta Tiller Assembly- S02-062-00100
13. Tiller Tilt Cylinder- C02-029-00200

33. Seat Post Boot- C25-039-00100
34. Main Battery Shroud- S08-062-00677
35. Rear Battery Shroud- C08-062-01600
36. Rubber Battery Pad- C25-022-00500
37. Battery Strap- C31-029-00101
38. 70ah Battery (ea 2 needed)- LRC402407
39. Controller Cover- C26-051-00501
40. Controller - C10-029-00400
41. Controller Holder- S10-062-00100
42. Wire Harness- S18-062-00400
Negative Battery Cable- C18-061-01602
Positive Battery Cable- C18-061-01702
43. Rear Frame- S01-062-00200
44. Rear Shock Absorber 1050#- C04-062-00200
45. Motor-Brake-Transaxle Assy./Keyway- S09-062-00400
Motor-Brake-Transaxle Assy./Spline **- C09-062-03101
Transaxle with Keyway- C09-062-02100

14. Tiller Boot- C25-052-00100
15. Tiller Bearing Set- S02-001-00601
16. "L" Steering Bracket- S03-029-00200
Steering Bracket Spacer- 9367120850752
17. Tie Rod Set- S03-062-00100
18. Front Shock Absorber 150#- C03-062-00300
19. Shock Absorber Arm Assy. Rt.- C03-022-02108
Shock Absorber Arm Assy. Lt.- C03-022-02208
20. Front Wheel Assembly 8" Rim *- S05-062-00300
Tire 8" Rim (front or rear)- C06-051-00100
Tube 8" Rim (front or rear)- C06-051-00201
Front Wheel Assy. 14" Rim *- S05-062-00100
Tire 14" Rim (front or rear) (50 psi)- C05-063-00202
Tube 14" Rim (front or rear)- C05-062-00500
Front Wheel Bearing- STDS2A5303
21. Hub Cap 8" Rim- C26-060-00501 *
Hub Cap 14" Rim- C05-047-00600 *
22. Headlight Assembly- C13-062-00100
Bulb- C13-062-01300
23. Front Signal Light Right- C15-062-00102
Front Signal Light Left- C15-062-00502
24. Front Bumper & Bracket- C01-062-00300
25. Front Bumper Cover- C08-062-00501
26. Front Frame- S01-062-00100
27. Front Shroud- S08-062-00277
28. Middle Shroud (w/o reflectors)- S08-062-00777
Reflector (ea)- C24-062-00200
29. 20" Seat Assembly/Arms- S07-062-00100
22" Seat Assembly/Arms- S07-062-00200
Headrest Only- C07-062-00400
Arm Assembly Left- C07-061-01700
Arm Assembly Right- C07-061-01800
Arm Pad- C07-061-01900
30. Seat Slides- C07-016-00600
31. Seat Plate- C07-062-00202
32. Seat Post- C07-005-00900 ***
Seat Post Bolt- C07-005-FSPBOLT
- Transaxle with Spline **- C09-062-03201
Motor - C09-062-02000
Motor Brush- C09-022-02300
Motor Brush Cap- C09-022-02700
Electro-Magnetic Brake- C09-062-01100
Motor Transaxle Coupler- S31097GT
46. Free Wheel Lever- C09-062-00903
47. Rear Wheel Assy. 8" Rim Right *- S06-062-00700
Rear Wheel Assy. 14" Rim Right -
With Keyway- S06-062-00100
With Spline **- S06-062-00800
48. Rear Shroud - S08-062-00577
49. Rear Signal Light Right- C14-062-00201
Rear Signal Light Left- C14-062-00301
50. Brake Pad Assy. 8" Rim *- C06-062-00402
Brake Pad Assy. 14" Rim *- C06-022-02600
51. Rear Wheel Assy. 8" Rim Left. *- S06-062-00600
Tire 8" Rim (front or rear)- C06-051-00100
Tube 8" Rim (front or rear)- C06-051-00201
Rear Wheel Assy. 14" Rim Left-
With Key Way- S06-062-00200
With Spline **- S06-062-00900
Tire 14" Rim (front or rear) (50 psi)- C05-063-00202
Tube 14" Rim (front or rear)- C05-062-00500
52. Hub Cap 8" Rim- C26-060-00501 *
Hub Cap 14" Rim- C05-047-00600 *
53. Anti-Tipper Set (pr)- S28-062-00100
54. Rear Bumper- C01-062-01201
55. Charger - C12-005-00800
Charger AC Power Cord- SF9056
56. Handle Bar (fits Lf. Or Rt.)- C02-062-00101
57. Motor Key Way- 29002W7011
58. Rear Shroud Decoration Left- C08-062-02077
Rear Shroud Decoration Right- C08-062-02177

* 8" rim used on serial number 2W13102000073 and before.

** Serial number P166233 and newer.

*** When stock is gone use C07-062-00600.

Dimension Diagram