


Daytona 4GT

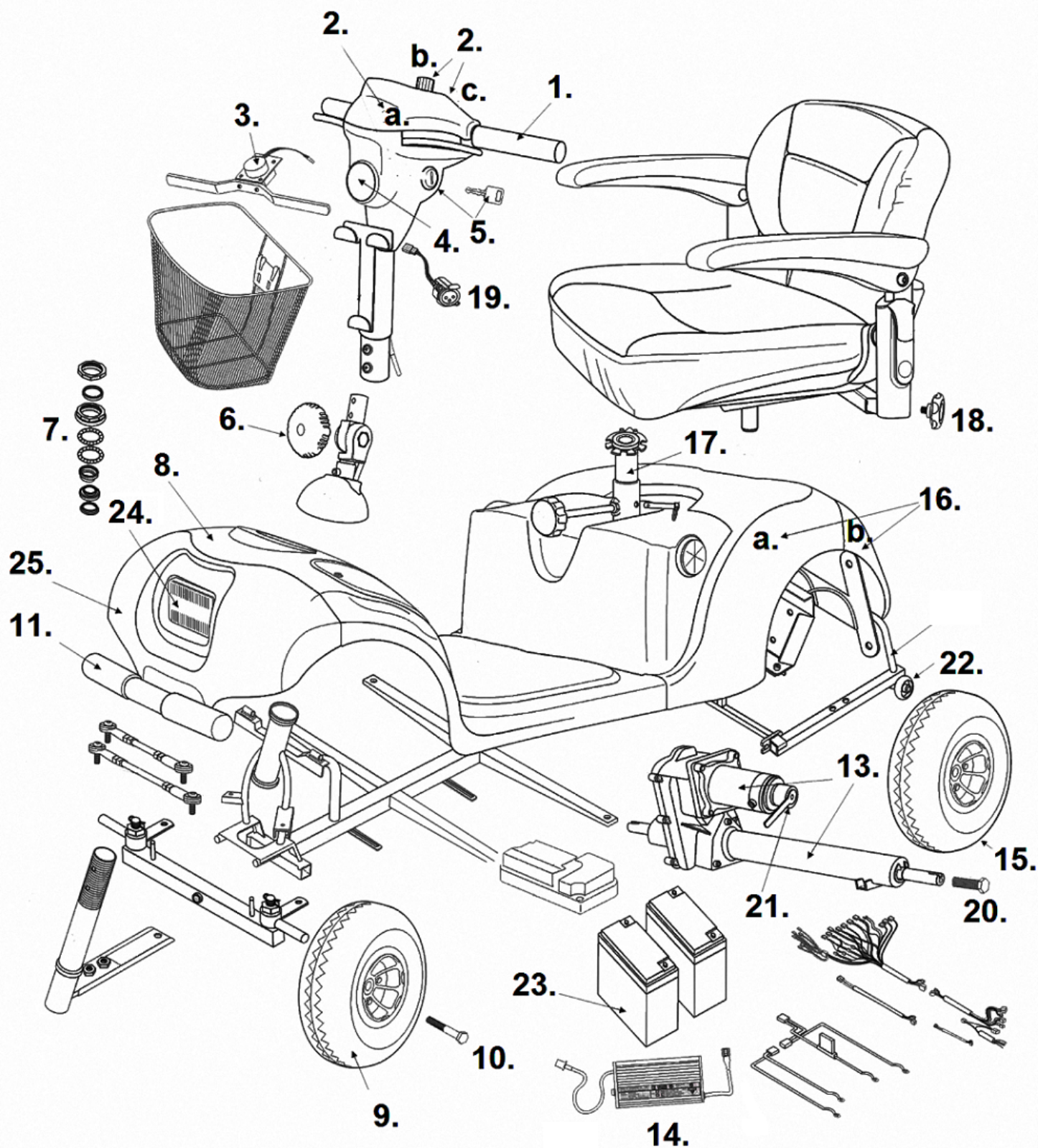

1. Hand Grip - S31003
2. a. Tiller Cover Black - S31209BK

10. Front Axle - G6942W36
11. Front Frame - S31218

17. Seat Post - S31053
18. Adjustment Knob - S31052

b. Speed Pot - S31001
c. Horn Button - S31012
3. Throttle - S12-034-00300
4. Head Light - S31022
5. Key Switch - S31006
Key - S3KEY
6. Tiller Adjustment Knob - S31028
7. Tiller Bearing Assembly - S31021
8. Front Shroud Blue - S41210GTB
Red - S41210GTR
9. Front Wheel - S41243

12. Front Bumper - S31014GT
13. Motor/Transaxle - SF31030GT [1]
13. Transaxle - S31031GT
13. Motor Coupler - S31097GT
13. Axle Key Way- S31030-KEY
14. Battery Charger - LRA402210
15. Rear Wheel - S31035
16. a. Rear Shroud Blue - S31018GTB
16. a. Rear Shroud Red - S31018GTR
16. b. Rear Bumper - S31019GT

19. Charger Port - S31147
20. Rear Axle Bolt - S350016910
21. Electro-Magnetic Brake - S31033BGT
22. Anti-Tipper Wheel - S31026
23. Battery (2 needed) - LRM412123
24. Lower Head Light - S41101GT
25. Head Light Shroud - S41103GT

[1] When stock is gone the part will be no longer available.

12/07/2020

[Diagrams](#)

[Home](#)