


KING COBRA GOLF

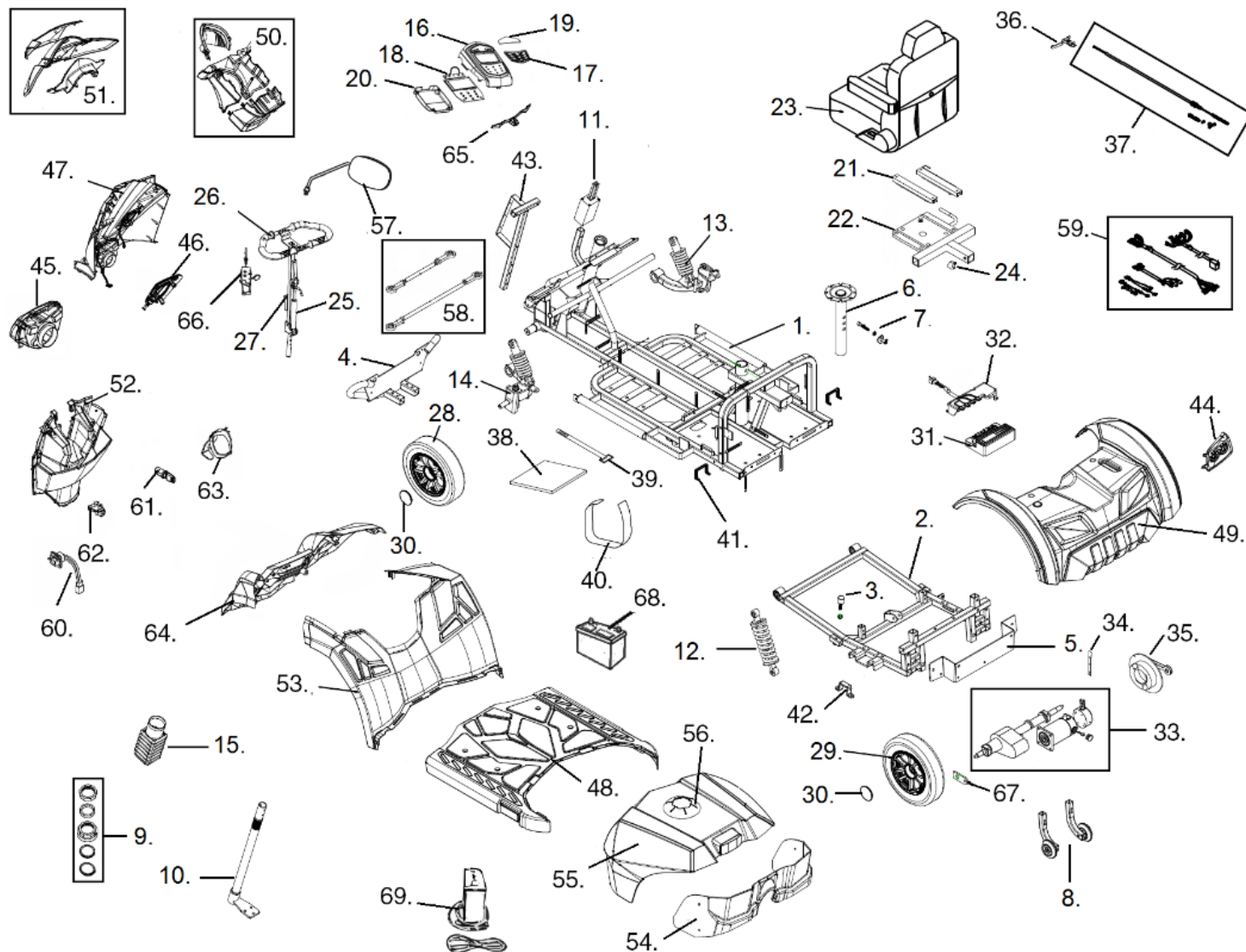

- | | |
|--|---|
| 1. Front Frame- C01-062-01605 | 34. Free Wheel Lever- C09-062-00903 |
| 2. Rear Frame- C01-062-00207 | 35. Wheel Brake- C06-062-00402 |
| 3. Frame Bumper- KC-001 | 36. Hand Brake- C35-045-00300 |
| 4. Front Bumper- C01-062-02001 | 37. Brake Cable Assemble- KC-005 |
| 5. Rear Bumper- C01-062-01201 | 38. Battery Pad- C25-022-00500 |
| 6. Seat Post- C07-005-00900 | 39. Velcro Strap- C31-029-00101 |
| 6. Sub- C07-062-00600 | 40. Velcro Strap- C31-005-00200 |
| 7. Seat Post Bolt- C07-005-FSPBOLT | 41. Battery Buckle- C01-003-01200 |
| 8. Anti-Tippers (pr)- KC-003 | 42. Motor Mount- C09-029-03601 |
| Anti-Tipper Wheel (ea)- C28-032-00100 | 43. Front Tiller Shroud Mounting Bracket- C01-062-01702 |
| 45. 9. Tiller Bearing Set- S02-001-00601 | 44. Rear Light Assembly Left- C14-062-00301 |
| 10. "L" Steering Axle- C03-062-01001 | Rear Light Assembly Right- C14-062-00201 |
| 11. DC to DC Converter- C12-061-01000 | 45. Head Light- C13-062-01700 |
| 12. Rear Shock Absorber 1050LBS- C04-062-00201 | |

13. Front Shock Absorber 150LBS- C03-062-00300
14. Absorber Arm Left- C03-062-00701
Absorber Arm Right- C03-062-00501
15. Tiller Boot- C25-062-00300
16. Upper Tiller Cover- C08-051-01003
17. Control Panel Overlay- C10-051-01000
18. Tiller PC Board- C10-051-01100
19. PC Board Cover- C10-051-00900
20. Lower PC Board Cover- C08-051-01100
21. Seat Slides Set- C07-016-00600
22. Seat Plate- C07-062-00202
23. 20" Seat Assembly- S07-062-00100
22" Seat Assembly- S07-062-00200
Arm Assembly Left- C07-061-01700
Arm Assembly Right- C07-061-01800
Arm Pad- C07-061-01900
Headrest Only- C07-062-00400
24. Accessory Knob- S31052
25. Tiller Cylinder- C02-062-01100
26. Handle Bar (left or right)- C02-062-00101
27. Tiller- C02-062-00601
28. Front Wheel Assembly- S05-062-00600
Front Tire Only (100/80-10)- C05-062-00900
Tube (3.50-10)- C05-062-00600
Bearing (pr)- 9653069040000
Axle Nut- KC-010
29. Rear Wheel Assembly Left- S06-062-01000
Rear Wheel Assembly Right- S06-062-01100
Rear Tire Only- C06-062-01400
Tube (3.50-10)- C05-062-00600
Axle Nut- KC-011
30. Hub Cap- C05-047-00600
31. Controller (Rhino 2 180A)- C10-062-00600
32. Controller Cover- C26-051-00501
33. Motor-Transaxle-Brake Assembly- C09-062-03401
Motor- C09-062-01200
Motor Brushes (4 ea)- C09-029-04400
Motor Brush Caps (4 ea)- C26-051-00100
Transaxle- C09-062-02401
Brake- C09-062-01100

46. Front Turn Signal Left- C15-062-00502
Front Turn Signal Right- C15-062-00102
47. Front Tiller Shroud Assembly- S08-062-01118
Front Tiller Shroud Decoration Set- KC-006
48. Foot Shroud Assembly- S08-051-01000
Foot Shroud Decoration Set- KC-007
49. Rear Shroud Assembly- S08-062-00518
Rear Shroud Decoration Set- KC-008
50. Lower Handle Bar Cover- S08-062-01200
51. Front Handle Bar Cover- S08-062-01018
52. Rear Tiller Shroud Assembly- S08-062-01301
53. Front Shroud Rear- C08-062-02800
54. Rear Battery Shroud- C08-062-03600
55. Main Battery Shroud- KC-004
56. Seat Post Boot- C25-039-00100
57. Mirror Left- C37-062-00400
Right- C37-062-00300
58. Tie Rod Set-KCG-001
59. Wire Harness Complete- KC-002
Speed Sensor Switch- C16-051-00202
High Speed Turn Sensor- C16-062-00101
G Sensor- C16-062-00100
60. Charger Port- C12-051-00200
61. Key Switch- C18-001-01501
Key - C37-001-02300
Key Switch Cable- C18-062-00802
62. USB Connector- C16-062-00201
USB Cover- C26-062-00400
63. Cup Holder- C26-062-00300
64. Front Shroud Forward- C08-062-03800
65. Throttle- S12-051-00100
Throttle Cable- C18-051-01200
66. Tiller Adjustment Assembly- KC-009
67. Speed Sensor Magnet- C06-062-00700
68. Battery (2 needed)- LRC402407
69. Charger- C12-005-00800
Charger AC Power Cord- SF9056

01/12/2021

[Diagrams](#)

[Home](#)