



PHOENIX HD3

Delta Tiller s/n
(21)2W1801100007
and later.

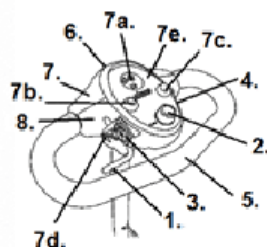


1. Throttle- S12-034-00400
2. Speed Control- S31001
Speed Control Knob- LRM412103-11
3. Charger Port- S31147
4. Key Switch- S31006
KEY- S3KEY
5. Delta Bar Cover- S35010HD-03
(Upper & Lower Cover)
6. Headlight Assembly- S35010HD-02
7. Front Tiller Cover- C08-034-01100
 - a. Battery Gauge LED- C11-076-00100
 - b. Headlight Switch- S31122
 - c. Horn Button- S31012
 - d. USB Assembly- S35010HD-01
 - e. Control Panel Assembly- C08-034-01500
Face Plate Only- C08-034-01200
8. Rear Tiller Cover
Upper- C08-034-01300
Lower- C08-034-01400

Standard tiller before s/n
(21)2W1801100007

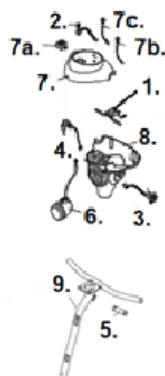

1. Throttle Old- *
New- S12-034-00300
* Stock is gone use new style.
2. Speed Control- S31001
Speed Control Knob- LRM412103-11
3. Charger Port- S31147
4. Key Switch- S31006
Key- S3KEY
5. Handgrip Foam- S31003
Handgrip TPR Plastic- C20-007-00200
6. Headlight Assembly- S31022
7. Front Tiller Cover- S31004GT
 - a. Battery Gauge LED- C11-076-00100
LED- C11-076-00100
Analog- C11-033-00101
 - b. Headlight Switch- S31122
 - c. Horn Button- S31012
8. Lower Tiller Cover- S31209BK
9. Tiller Standard- S31002
Tiller Ext. Foot Platform- C02-034-00100

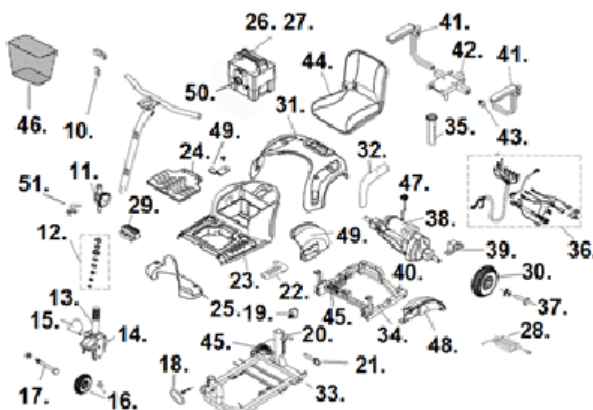

10. Basket Bracket- BC35051
11. Tiller Swivel Assy.- S350109
Tiller Adj. Knob- S31028
12. Tiller Bearing Assy.- S31021
13. Fork- S31036-3HD
Ext Foot Platform- C03-009-00101
14. Fork Cover- S350127
15. Blue Decoration- **NLA**
Red Decoration- **NLA**
16. Front Wheel- S31142-3HD
Front Wheel Bearing- 9503H79021
10mm ID – 22mm OD
17. Front Axle- S05-009-00100
18. Tow Handle- **NLA**
19. Tension Knob- S350169715
20. T Handle Frame lock- S350169716

33. Front Frame
Non-Ext Foot Platform- **NLA**
Ext Foot Platform- C01-035-00800
34. Rear Frame
Non-Ext Foot Platform- S31130-3HD
Ext Foot Platform- C01-035-00900
35. Seat Post- S31049
36. Wire Harness (CTE)- S31048-HD
Wire Harness (S Drive)- S18-034-00400
37. Rear Axle Nut- S350112AN
38. Motor/Brake/Transaxle- S31125HD
Motor/Brake- S31031-HD
Transaxle- S350112-HD
Axle Key- 29002W7011
39. Transaxle Bracket- S350112BKT-HD
40. Anti-tip Wheel-S31026

- | | |
|---|---|
| <ul style="list-style-type: none"> 21. Seat pin- S350169717 22. Main Shroud Decoration <ul style="list-style-type: none"> Blue Left- S3501BL Blue Right- S3501BR [1] Red Left- S3501RL Blue Right- S3501RR 23. Main Shroud- S350110 24. Controller Cover- S350113 25. Foot Shroud- S350114 <ul style="list-style-type: none"> Ext Foot Platform- C08-035-01500 26. Battery Pack- S350169707-HD <ul style="list-style-type: none"> Case Only- S350169706-HD Charger Port- S350169706-HDCP 27. Battery (2 needed)- LRM412123 28. Charger- LRM302111 29. Controller (CTE)- S31008GT-HD <ul style="list-style-type: none"> Controller (S-Drive)- C10-067-00200 * * After Serial # (21)2W1801100007 30. Rear Wheel- S31132B-HD 31. Rear Shroud- S350111 <ul style="list-style-type: none"> Lifting Handle- S35011FH 32. Rear Shroud Decoration <ul style="list-style-type: none"> Blue Left- S35011B-L Blue Right- S35011B-R Red Left- S35011R-L Red Right- S35011R-R | <ul style="list-style-type: none"> 41. Armrest Assy. Left- VENT-29 <ul style="list-style-type: none"> Right- VENT-30 Armrest Pad Left- VENT-31 Right- VENT-32 42. Seat Plate- S31039-HD 43. Armrest- Adj. Knob- S31052 44. 17.5" Seat & Seat Plate- S35045-HD <ul style="list-style-type: none"> 17.5" Seat only- S35045-HD18 17.5" Back Cover- S35045BC-HD 17.5" Seat Cover- S35045SC-HD 20" Seat & Seat Plate- S35045-HD20 20" Seat, Seat Plate/Arms- S35045-HD20-W-ARM 20" Back Cover- C07-034-01500 20" Seat Cover- C07-034-01600 45. Frame Contact Front- S31056HD <ul style="list-style-type: none"> Rear- S31057HD 46. Basket- S35003-PL 47. Free Wheel Knob- S31031K 48. Mud Flap Left- S35048MFL <ul style="list-style-type: none"> Right- S35048MFR 49. Battery Contact- S31060 50. Battery Box Charge Port- S350169706-HDCP 51. Tiller Lock- NLA |
|---|---|

[1] When stock is gone the part is no longer available.

[Diagrams](#)

[Home](#)

...ix HD3_rev. 08_24_2021