
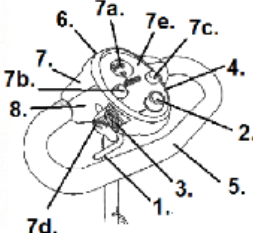
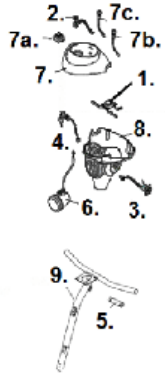


Phoenix HD4

Delta Tiller s/n (21)2W1801100007 and later.	Standard tiller before s/n (21)2W1801100007
 1. Throttle- S12-034-00400 2. Speed Control- S31001 Speed Control Knob- LRM412103-11 3. Charger Port- S31147 4. Key Switch- S31006 KEY- S3KEY 5. Delta Bar Cover- S35010HD-03 (Upper & Lower Cover) 6. Headlight Assembly- S35010HD-02 7. Front Tiller Cover- C08-034-01100 a. Battery Gauge LED- C11-076-00100 b. Headlight Switch- S31122 c. Horn Button- S31012 d. USB Assembly- S35010HD-01 e. Control Panel Assembly- C08-034-01500 8. Rear Tiller Cover Upper- C08-034-01300 Lower- C08-034-01400	 1. Throttle Old- * New- S12-034-00300 * Stock is gone use new style. 2. Speed Control- S31001 Speed Control Knob- LRM412103-11 3. Charger Port- S31147 4. Key Switch- S31006 Key- S3KEY 5. Handgrip Foam- S31003 Handgrip TPR Plastic- C20-007-00200 6. Headlight Assembly- S31022 7. Front Tiller Cover- S31004GT a. Battery Gauge LED- C11-076-00100 b. Headlight Switch- S31122 c. Horn Button- S31012 8. Lower Tiller Cover- S31209BK 9.

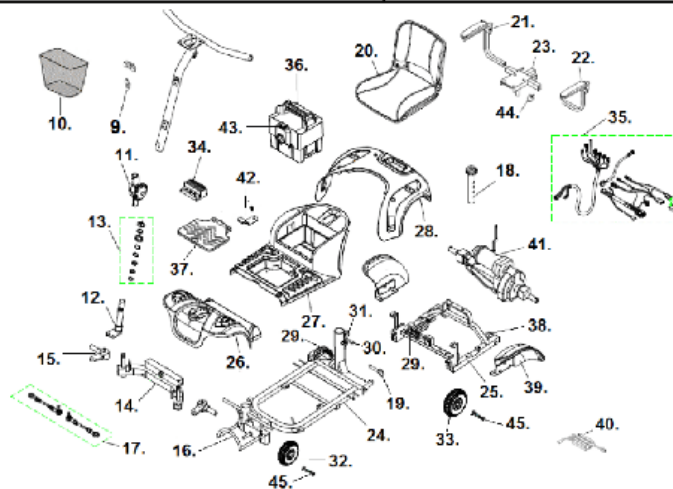

- | | |
|---|--|
| <p>9. Basket Bracket- BC35051</p> <p>10. Basket- S35003-PL</p> <p>11. Tiller Swivel Assembly- S350109
Tiller Locking Knob- S31028</p> <p>12. Steering Post- S350101 [1]</p> <p>13. Tiller Bearing Set- S31021</p> <p>14. Front Fork- S350105HD</p> <p>15. Steering Axle- S350102 [1]</p> <p>16. Front Bumper- S350104-4HD</p> <p>17. Connecting Rod (ea)- S350103</p> <p>18. Seat Post- S31049</p> <p>19. Seat Post Pin- S350169717</p> <p>20. 17.5" Seat & Seat Plate- S35045-HD
17.5" Seat Only- S35045-HD18
17.5" Back Cover- S35045BC-HD
17.5" Seat Cover- S35045SC-HD
20" Seat & Seat Plate- S35045-HD20
20" Seat, Plt & Arms- S35045-HD20-W-ARM</p> | <p>28. Rear Shroud- S350111
Lifting Handle- S35011FH
Rear Decoration Blue Left- S35011B-L
Rear Decoration Blue Right- S35011B-R
Rear Decoration Red Left- S35011R-L
Rear Decoration Red Right- S35011R-R</p> <p>29. Frame Contact Front- S31056HD
Frame Contact Rear- S31057HD</p> <p>30. Frame Lock Handle- S350169716</p> <p>31. Tension Knob- S350169715</p> <p>32. Front Wheel Assembly- S350100-4HD
Front Wheel Brg (pr)- S350100B-4HD</p> <p>33. Rear Wheel Assembly- S31132B-HD</p> <p>34. Controller (CTE)- S31008GT-HD
Controller (S-Drive)- C10-067-00200 *
* After Serial # (21)2W1801100007</p> <p>35. Wire Harness (CTE)- S31048-HD
Wire Harness (S Drive)- S18-034-00400</p> |
|---|--|

- 20" Back Cover- C07-034-01500
- 20" Seat Cover- C07-034-01600
- 21. Armrest Complete Left- VENT-29
- Armrest Complete Right- VENT-30
- 22. Arm Pad Left- VENT-31
- Arm Pad Right- VENT-32
- 23. Seat Plate- S31039-HD
- 24. Front Frame- S350108
- 25. Rear Frame- S31130-4HD
- 26. Foot Shroud- S350107
- Front Decoration Blue Left- S3501BL-1
- Front Decoration Blue Rt- S3501BR-1 [1]
- Front Decoration Red Left- S3501RL-1
- Front Decoration Red Right- S3501RR-1
- 27. Main Shroud- S350110
- Main Decoration Blue Left- S3501BL
- Main Decoration Blue Right- S3501BR
- Main Decoration Red Left- S3501RL
- Main Decoration Red Right- S3501RR

- 36. Battery Pack Assembly- S350169707-HD
- Battery Case Only- S350169706-HD
- Battery (need 2)- LRM412123
- 37. Controller Cover- S350113
- 38. Anti-tip Wheel- S31026
- 39. Mud Flap Left- S35048MFL
- Mud Flap Right- S35048MFR
- 40. Battery Charger- LRM302111
- 41. Motor-Transaxle Assembly- S31125HD
- Motor-Brake- S31031-HD
- Transaxle- S350112-HD
- Transaxle Bracket- S350112BKT-HD
- Free Wheel Knob- S31031K
- Motor Axle Key- 29002W7011
- 42. Battery Contact- S31060
- 43. Bat. Box Chrgr Port- S350169706-HDCP
- 44. Armrest Knob- S31052
- 45. Front-Rear Axle Bolt- S350112AN

[1] When stock is gone part is no longer available.