


SPITFIRE DST 3

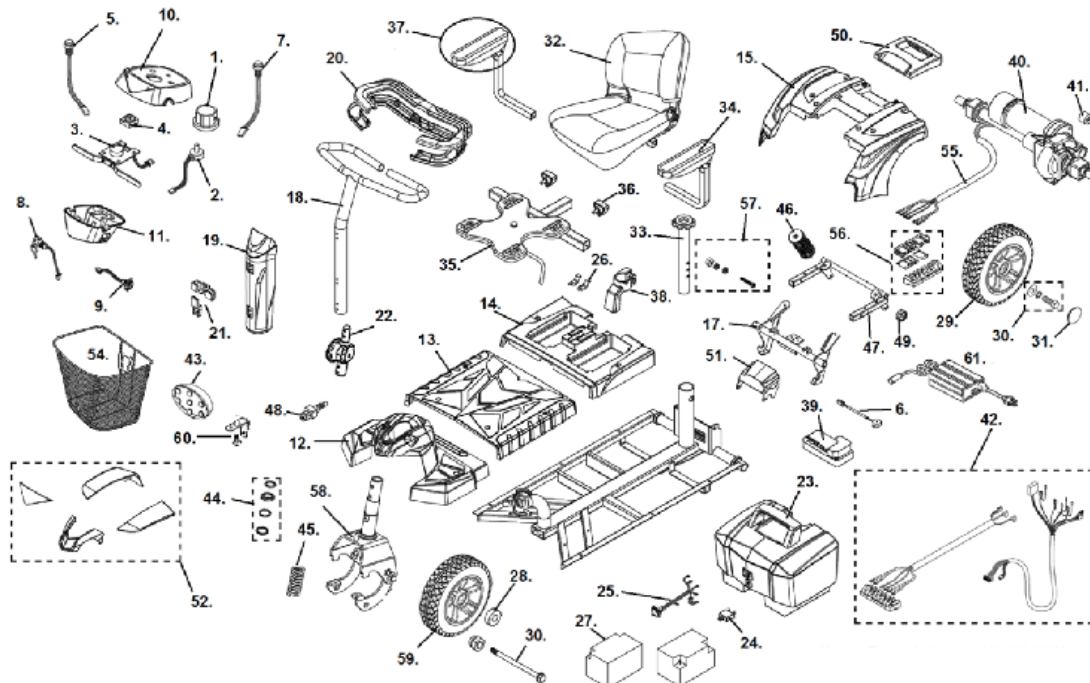

1. Speed Dial Knob- LRM412103-11
2. Speed Dial- BC31001
3. Throttle Old- *
- New- S12-069-00300
4. Battery Gauge- C11-076-00100
5. Horn Button- SC31034
6. Horn- C12-050-00501
7. Light Switch- C16-069-00400
8. Key Switch- C18-069-00101
- Key- BCKEY
9. Charger Port- C18-071-00800
10. Upper Tiller Cover- C08-069-00101
11. Lower Tiller Cover- C08-069-06600
12. Front Shroud- C08-070-00100
13. Foot Shroud- SC31024
14. Battery Box Holder- SC31027
15. Rear Shroud- C08-069-05500
16. Front Frame- C01-070-00400
17. Latching Frame- C01-069-01801
18. Tiller- C02-069-00100
19. Front Tiller Cover- C08-069-06500
20. Handle Bar Cover- SC31054
21. Basket Holder- BC35051
22. Tiller Swivel Assembly- C02-069-00601
23. Battery Box w/Batteries 20AH- SC31050
- Battery Box only 20 AH- SC31051

33. Seat Post- C07-069-00201 ***
34. Left Armrest Complete- C07-028-00404
- Right Armrest Complete- C07-028-00504
35. Seat Plate- DST-016
36. Arm Rest Knob- S31052
37. Arm Pad Left- VENT-31
- Arm Pad Right- VENT-32
38. Battery Box Hold Down- SC31047
39. Controller Curtis- SC31052
- Dynamic- C10-069-00100-3 **
40. Motor, Brake & Transaxle- S09-069-00100
- Transaxle Only- BC350112
- Motor w/Brake- C09-069-00102
- Brake Only- C09-069-00201
41. Motor to Transaxle Coupler- C09-069-00300
42. Wire Harness Complete Curtis- DST-008
- Wire Harness Complete Dynamic- DST-018 **
43. Head Light- C13-069-00100
- Head Light Harness Dynamic- DST-019 **
44. Fork Stem Bearing Set- DST-010
45. Front Spring- C03-075-01000
46. Rear Spring- C04-075-00200
47. Rear Frame- C01-069-01901
48. Tiller lock- C08-069-01000
49. Anti-tipper Wheel- S31026
50. Rear Shroud Handle- C08-069-05500

Battery Box Only 20 AH- SC31031
Negative Battery Cable- C18-069-01201
Positive Battery Cable- C18-069-01101
Wire Harness Complete- S18-069-00100
24. 20 Amp Circuit Breaker- BC20ACB
25. Battery Box Charger Port- SC31031
26. Battery Frame Contact- C12-060-00300
27. 20 AH Battery (2 needed)- LRM412123
28. Wheel Bearing- 9653060020002
29. Rear Wheel Assembly- C06-060-00200
30. Axle Bolt Assembly- DST-005
31. Hub Cap- SC31025
32. Seat w/Seat Plate w/o Armrests- DST-013

30. Rear Shroud Handle- C06-069-03600
51. Frame Latch- DST-012
52. Shroud Decorations Red- DST-001R
Shroud Decorations Blue- DST-001B
53. Front Axle- DST-006
54. Basket- S35003-P
55. Motor Wire Harness- SC31020
56. Contact Block Set w/Motor Harness- SC31033
57. Seat Post Bolt Assembly- DST-009
58. Front Fork- DST-014
59. Front Wheel Assembly- C05-060-00103
60. Headlight Bracket w/Screw- C13-069-00200
61. Battery Charger- LRM302111

* Stock is gone use new style throttle.

** No longer available in chrome once stock is gone.

*** After serial # VP142824.

[Diagrams](#)

[Home](#)

File DST 4_rev. 02_19_2019