


SCOUT 4

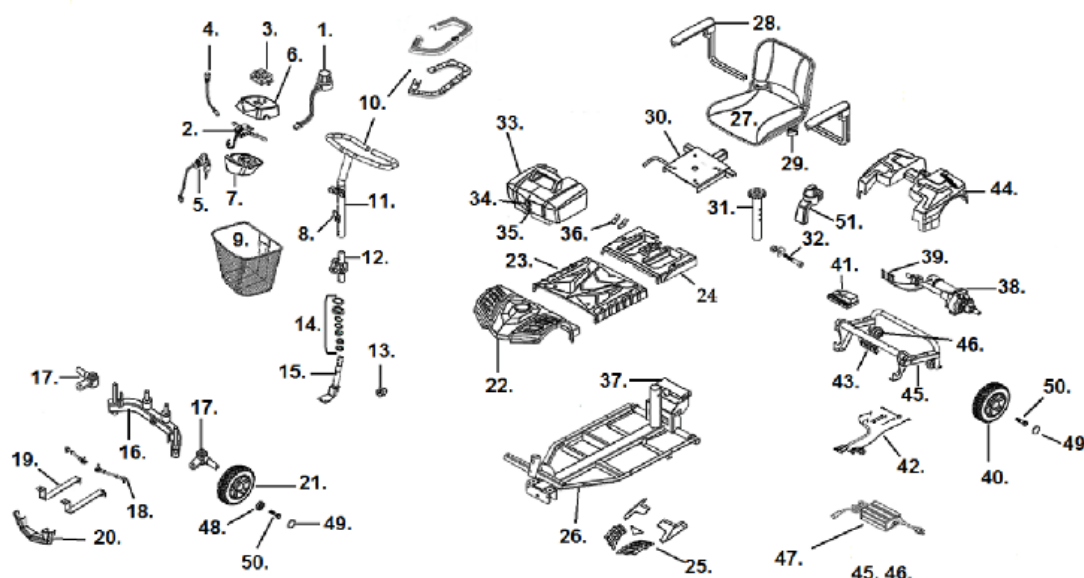

1. Speed Dial- BC31001
Speed Dial Knob- LRM412103-11
2. Throttle Assembly Old [2]
Throttle Assembly New- S12-069-00300
3. Battery Gauge (4 LEDs)- SC31004
Battery Gauge (8 LEDs)- C11-076-00100 [1]
4. Horn Button- SC31034
5. Key Switch Assembly- BC31006
Key- BCKEY
6. Upper Tiller Cover- SC31008
7. Lower Tiller Cover- SC31009
8. Basket Bracket- BC35051
9. Basket- S35003-P
10. Handgrip Foam- SC31002
Handgrip Plastic- SC31054
11. Tiller- SC31001
12. Tiller Swivel Assembly- SC31010
13. Tiller Lock- C08-069-01000
14. Tiller Bearing Set- BC31021
15. "L" Steering Stem- SC31016
16. Steering Fork Assembly- SC31015
17. Steering Axle (left or right)- SC31021
18. Connecting Rod- BC350103
19. Bumper Bracket- SC31022
20. Front Bumper- SC31028
21. Front Wheel Black- SC31017
Deluxe Front Wheel- SC31017-D
22. Front Shroud- SC31013
23. Footrest Shroud- SC31024
24. Deluxe Front Shroud- SC31027

25. Front Shroud- SC31027
26. Front Shroud- SC31027
27. Seat- SC31023
28. Left Armrest- BC35040L
Right Armrest- BC35040R
Arm Pad- BC953014
29. Armrest Knob- S31052
30. Seat Plate- SC31023
31. Seat Post- SC31018 [4]
32. Seat Post Bolt- C07-071-00101
33. Battery Box & Batteries (12V 12ah)- SC31007
Battery Box Only (12V 12ah)- SC31006
Battery 12V 12ah (2 needed)- LRM402109
- EXT Model
Battery Box & Batteries (12V 20ah)- SC31050
Battery Box Only (12V 20ah)- SC31051
Battery 12V 20ah (2 needed)- LRM412123
34. Charger Socket & Harness- SC31031
35. 20 Amp Circuit Breaker- BC20ACB
36. Battery Contact- C12-060-00300
37. Disconnect Handle Assembly- SC31019
38. Motor/Brake/Transaxle Assembly- SC31053
Transaxle- BC350112
Motor- SC31036
Electro Magnetic Brake- SC31037
39. Motor Wire Harness- SC31020
40. Rear Wheel Black- SC31003
Deluxe Rear Wheel- SC31003-D
41. Controller Curtis- SC31052-4
Controller Dynamic- C10-069-00100-4 [3]
42. Wire Harness Complete-
Curtis- SC31026
Dynamic- SC31027 [3]
43. Motor Wire Harness- SC31020
44. Motor Wire Harness- SC31020
45. Motor Wire Harness- SC31020
46. Motor Wire Harness- SC31020
47. Motor Wire Harness- SC31020
48. Motor Wire Harness- SC31020
49. Motor Wire Harness- SC31020
50. Motor Wire Harness- SC31020
51. Motor Wire Harness- SC31020

- | | |
|-----------------------------------|--|
| 24. Battery Box Shroud- SC31027 | Dynamic- DS1-018 [3] |
| 25. Red Decoration- SC31029 | 43. Contact Block Set w/Motor Harness- SC31033 |
| Deluxe Red Decoration- SC31029-D | 44. Rear Shroud- SC31011 |
| Blue Decoration- SC31030 | 45. Rear Frame- SC31012 |
| Deluxe Blue Decoration- SC31030-D | 46. Anti-Tipper Wheel- S31026 |
| 26. Front Frame- SC31014 | 47. Charger- LRM302111 |
| 27. Black Seat- SC35045 | 48. Front Wheel Bearing- S350100-B |
| Deluxe Two-Tone Seat- SC31035 | 49. Hub Cap- SC31025 |
| Deluxe Back Cover- SC31035-BC | 50. Axle Bolt Assembly- SC31049 |
| Deluxe Seat Cover- SC31035-SC | 51. Battery Box Hold Down- SC31047 |

[1] After Serial # VP035868

[2] Stock is gone use new style.

[3] After Serial # VP142824

[4] No longer available in chrome once stock is gone.

[Diagrams](#)

[Home](#)

cout 4_rev. 01_22_2021