


VENTURA 3

1. Throttle Assembly- VENT-01
Throttle Lever Only- VENT-87
2. Face Plate Assembly Folding Seat- VENT-04
Face Plate Assembly Capt. Seat- VENT-5
3. Speed Dial Assembly- S31001
Speed Dial Knob- LRM412103-11
4. Battery Gauge LED- C11-076-00100
Battery Gauge Analog- 11-033-00101
5. Turn Signal Switch Capt. Seat- VENT-72
6. Headlight Switch- S31122
Sub. GT8072W38
Headlight Switch Cover- GT8072W43
7. Fast/Slow Switch- GT8072W41.
8. Horn Switch- S31012
Horn Switch Cover- GT8072W43
9. Flasher Switch- GT8072W32
10. Turn Signal Relay Capt. Seat- VENT-76
11. Key Switch- S41160
Key- S4KEY
12. Handle Bar Assembly- VENT-03
13. Mirror Left- VENT-54
Mirror Right- VENT-53
14. Turn Signal Left- GT8072W35
Turn Signal Right- GT8072W36
Turn Signal Bulb- GT8072W53
15. Basket- S35003-PL
16. Basket Holder- VENT-55
17. Front Decoration Blue- VENT-62
Front Decoration Red- VENT-61
18. Front Tiller Cover- VENT-06
19. Tiller Swivel Assembly- VENT-02
20. Rear Tiller Cover- VENT-07
21. Charger Socket- S31147
22. Head Light Assembly- VENT-88
23. Fork- VENT-48
24. Fork Stem Bearings- VENT-13
25. Front Shroud- VENT-45
26. Front Axle Assembly- VENT-50
27. Coupler- VENT-46
28. Front Wheel Solid- VENT-49
Front Wheel Pneumatic- VENT-58
Pneumatic Tire Only- VENT-59
Inner Tube- VENT-60
29. Hup Cap- VENT-70
30. Rear Axle Bolt- VENT-71
31. Front Frame- VENT-47
32. Anti-tip Wheel- VENT-16
33. Controller- VENT-08
34. Middle Shroud- VENT-11
35. Rear Frame- VENT-18
36. Shroud Kit Blue (screw on)- [1]
Shroud Kit Red (screw on)- [1]
Shroud Kit Blue (snap on)- VENT-81
Shroud Kit Red (snap on)- VENT-80
37. Shock Absorber- VENT-15
38. Motor-Transaxle-Brake- VENT-19 [2]
Trans Axle- VENT-20 [2]
Motor- VENT-85 [2]
Motor Brush- SF5012 [2]
Electro Magnetic Brake- VENT-22
Motor-Transaxle-Brake- VENT-83 [3]
Trans Axle- VENT-84 [3]
Motor Only- VENT-21 [3]
Axle Keyway- 29002W7011

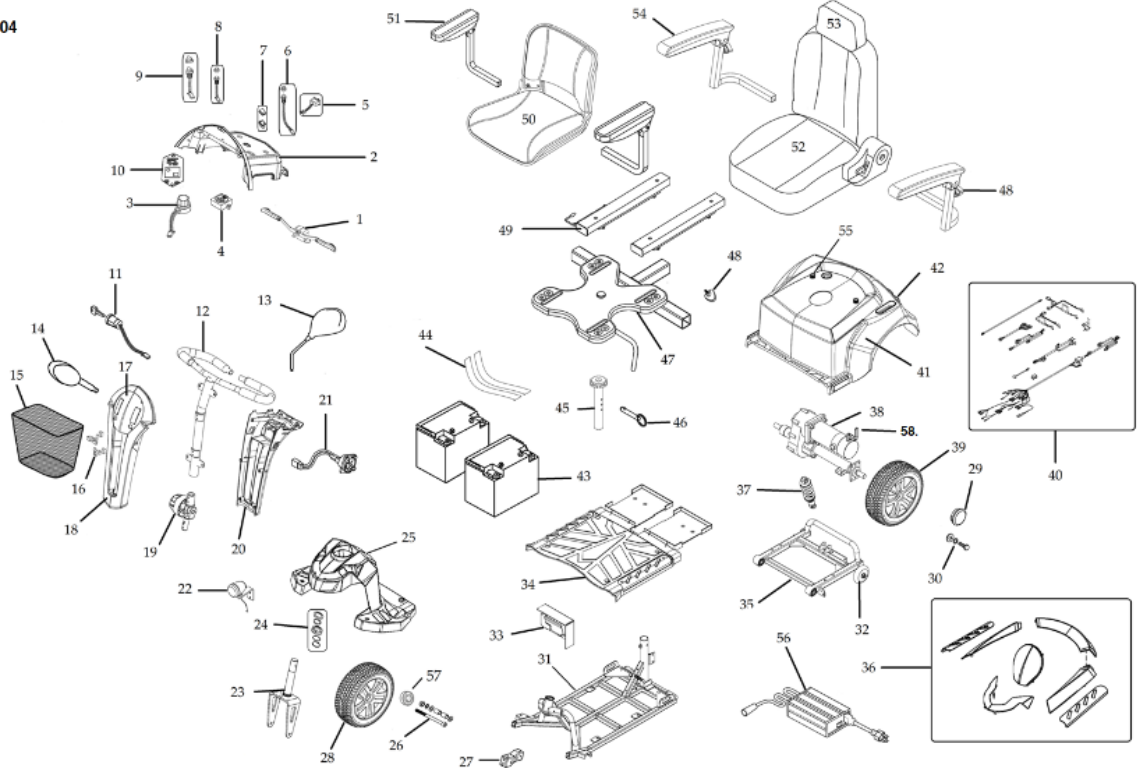

39. Rear Wheel Solid- VENT-28
Rear Wheel Pneumatic- VENT-57
Pneumatic Tire Only- VENT-59
Inner Tube- VENT-60
40. Wire Harness Folding Seat- VENT-37
Wire Harness Capt. Seat- VENT-38
(Includes tiller and battery harness)
40. Battery Harness Only- S41158B
41. Rear Shroud Folding Seat- VENT-12
41. Rear Shroud Capt. Seat- VENT-69
42. Rear Light Capt. Seat- VENT-66
Rear Left Turn Light Capt. Seat- VENT-67
Rear Right Turn Light Capt. Seat- VENT-68
43. Battery 35AH (2 needed)- LRA402208
43. Battery 21AH (2 needed)- LRM412123
44. Battery Strap- VENT-09
45. Seat Post- VENT-36 [4]
46. Seat Post Pin- S31152
47. Seat Plate Folding Seat- VENT-73
Seat Plate Capt. Seat- VENT-74
48. Arm Adjustment Knob- S31052
49. Seat Slides- VENT-65

50. Folding Seat Assembly- VENT-41
51. Left Arm Assy. Folding Seat- VENT-29
Rt Arm Assy. Folding Seat- VENT-30
Left Arm Pad Folding Seat- VENT-31
Right Arm Pad Folding Seat- VENT-32
52. Capt. Seat Assembly 18"- VENT-42
Capt. Seat Assembly 20"- VENT-43
53. Head Rest Capt. Seat 18"- SEC2010N
Head Rest Capt. Seat 20"- SF2010 [2]
54. Left Arm Assy. Capt. Seat- VENT-33
Right Arm Assy. Capt. Seat- VENT-34
Arm Pad Capt. Seat- VENT-35
55. Shroud Knob- VENT-86
56. Charger- LRA402210
57. Front Wheel Bearing- VENT-89
58. Free Wheel Lever & Hdwr- VENT-82

- [1] Stock is gone the part is no longer available.
[2] Before serial number VP030282.
[3] Serial Number VP030282 and higher.
[4] No longer available in chrome.

02/15/2021

Diagrams

Home