


ZOOMER 3

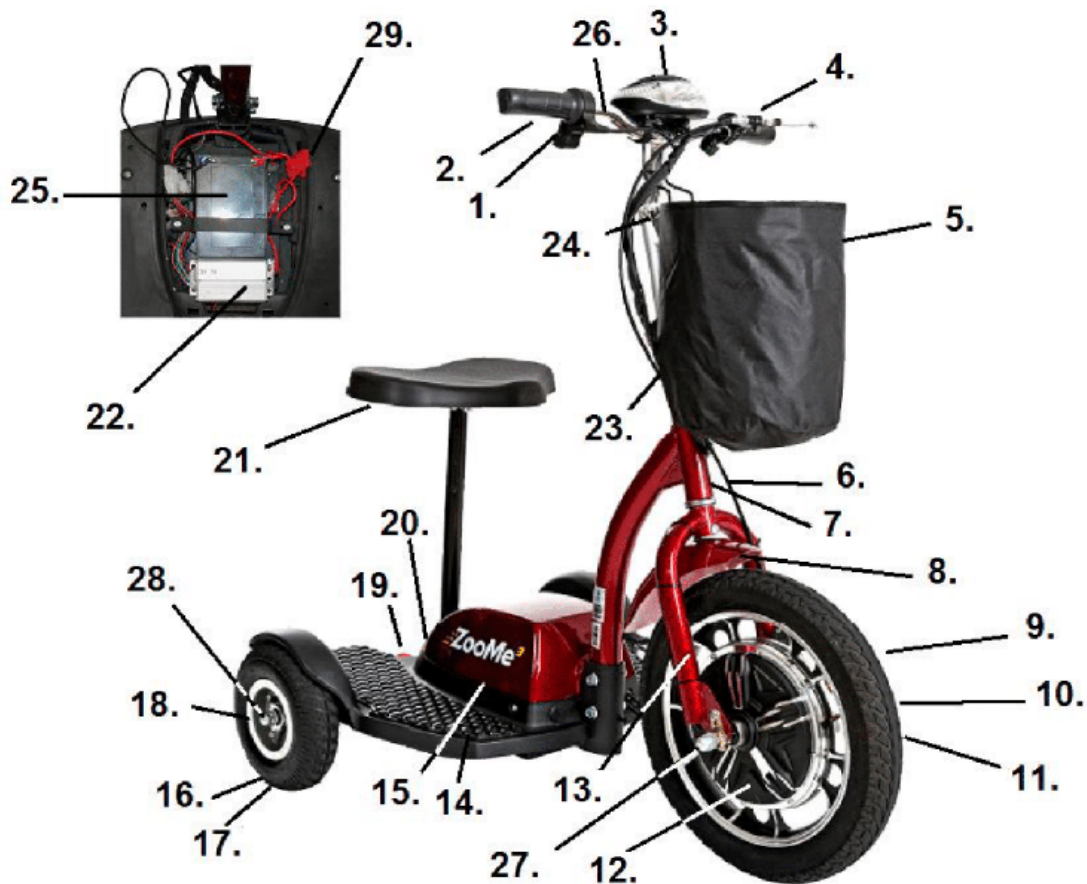

1. Horn Assembly- ZOOME3-11
2. Throttle- ZOOME3-02
3. Headset Assembly- ZOOME3-01
4. Brake Lever- ZOOME3-14
5. Basket- ZOOME3-18
6. Brake Cable- ZOOME3-28
7. Stem Bearing Kit- ZOOME3-25
8. Wheel Fender- ZOOME3-13
9. Front Tire- ZOOME3-08
10. Front Tube- ZOOME3-05
11. Brake Assembly- ZOOME3-16
Brake Pad- ZOOME3-17
12. Motor- ZOOME3-10
13. Front Fork- ZOOME3-22
14. Left- ZOOME3-20L
Right- ZOOME3-20R
15. Battery Cover- ZOOME3-21
Shroud Clips- ZOOME3-31

19. Anti Tip Wheel- ZOOME3-15
20. Charging Port- ZOOME3-04
21. Seat Assembly- ZOOME3-12
Seat Height Clamp- ZOOME3-26
Bottom Seat Clamp- ZOOME3-27
22. Controller- ZOOME3-00
23. Folding Handlebar Assembly- ZOOME3-23
24. Handlebar Locking Lever- ZOOME3-24
25. Battery (3 needed)- LRM402109 *
- * If the battery terminals are soldered to the battery harness, send part # ZOOME3-29 at no charge.
26. Handlebar- ZOOME3-30
27. Front Axle Cap Nut & Locking Nut- ZOOME3-32
28. Rear Axle Cap Nut & Spacer- ZOOME3-33
29. 30 Amp Fuse- BF30AH

16. Back Tire- ZOOME3-09

17. Back Tube- ZOOME3-06

18. Rear Wheel Assembly- ZOOME3-07

Not shown- Charger- ZOOME3-03

Tool Kit- ZOOME3-19

[Diagrams](#)

[Home](#)

ZOOME 3_rev. 02_18_2021