


ZOOMER R418CS

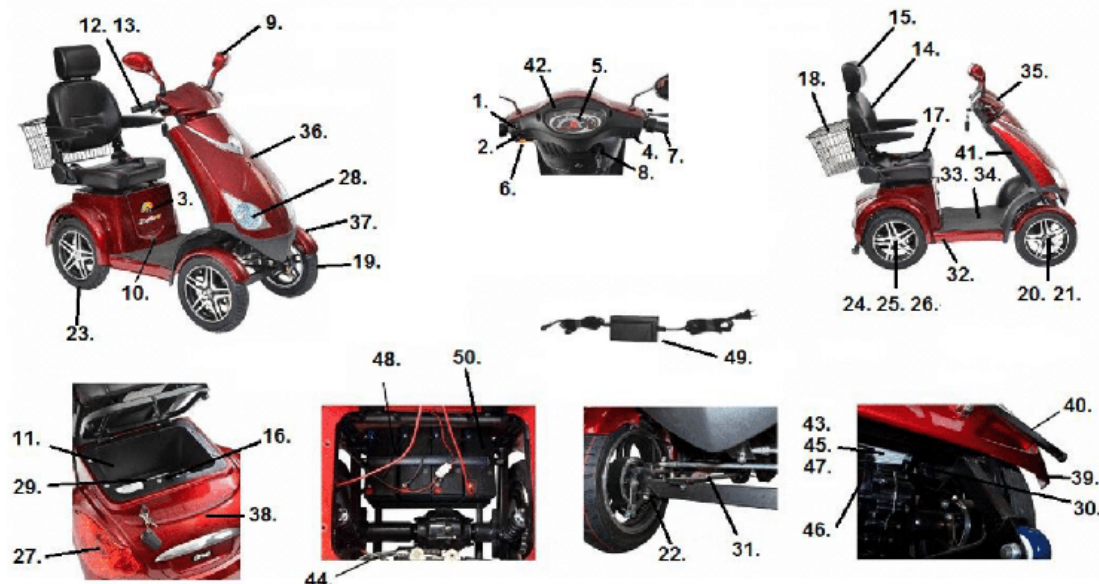

- | | |
|------------------------------------|---------------------------------------|
| 1. Head Light Beam Button- S72-034 | 25. Back Wheel Cap Set- S72-058 |
| 2. Turn Signal Button- S72-033 | 26. Back Left Axle- S72-022 |
| 3. Speed Selector & Key- S36-075 | Back Right Axle- S72-023 |
| 4. Lights Button- S72-037 | 27. Left Taillight- S72-018 |
| 5. Speedometer- S72-039 | Right Taillight- S72-019 |
| Speedometer Bulb- S72-017 | Brake Light Bulb- S72-015 |
| 6. Horn Button- S36-039 | Turn Signal Bulb- S72-016 |
| Horn- S36-051 | 28. Left Headlight- S72-013 |
| 7. Throttle- S72-004 | Right Headlight- S72-014 |
| 8. Ignition Set- S72-020 | 29. Circuit Breaker- S36-049 |
| 9. Mirror Set- S72-041 | 30. Back Shock Set- S72-031 |
| 10. Charger Port- S72-006 | 31. Front Leaf Spring Set- S72-032 |
| 11. Storage Box- S72-044 | 32. Left Running Board- S72-069 |
| 12. Left Hand Grip- S72-040 | Right Running Board- S72-070 |
| 13. Left Brake Handle- S72-027 | 33. Floor Mat- S72-055 |
| Right Brake Handle- S72-028 | 34. Floor Board- S72-054 |
| Back Brake Cable- S72-029 | 35. Headset Shroud- S72-072 |
| Front Brake Cable- S72-030 | 36. Front Body Shroud- S72-071 |
| Brake Pad- S36-029 | Front Shroud Chrome Insert- S72-063 |
| 14. Seat w/Headrest- S36-056 | 37. Left Front Fender- S72-066 |
| Seat w/o Headrest- S36-055 | Right Front Fender- S72-067 |
| 15. Headrest Only- S36-057 | 38. Main Body Shroud- S72-065 |
| 16. Seat Latch- S36-053 | Main Body Chrome Inlay- S72-064 |
| 17. Seat Belt- S36-054 | 39. Rear Bumper Shroud- S72-068 |
| 18. Back Basket- S72-042 | 40. Metal Back Bar- S72-060 |
| 19. Front Wheel Assembly- S72-012 | 41. Black Front Tiller Guard- S72-061 |
| Front Wheel Rim- S72-008 | 42. Black Front Shroud Guard- S72-062 |
| Front Tire- S72-010 | 43. Controller- S72-001 |
| 20. Front Wheel nut Set- S72-057 | 44. Wire Harness- S72-048 |

- 21. Front Wheel Cap Set- S72-059
- 22. Front Left Axle- S72-024
Front Right Axle- S72-025
- 23. Rear Wheel Assembly- S72-011
Back WheelRim- S72-007
Back Tire- S72-009
- 24. Back Wheel Nut Set- S72-056

- 45. Voltage Regulator- S72-047
- 46. Motor- S72-003
Transaxle- S72-073
- 47. Security Alarm Set- S36-025
- 48. Battery Box Wire Set- S72-045
- 49. Charger- S36-005
- 50. Battery (4 needed)- LRM412123

[Diagrams](#)

[Home](#)

Zoom R418CS_rev. 03_10_2021