


SPITFIRE EX2 (3 WHEEL)

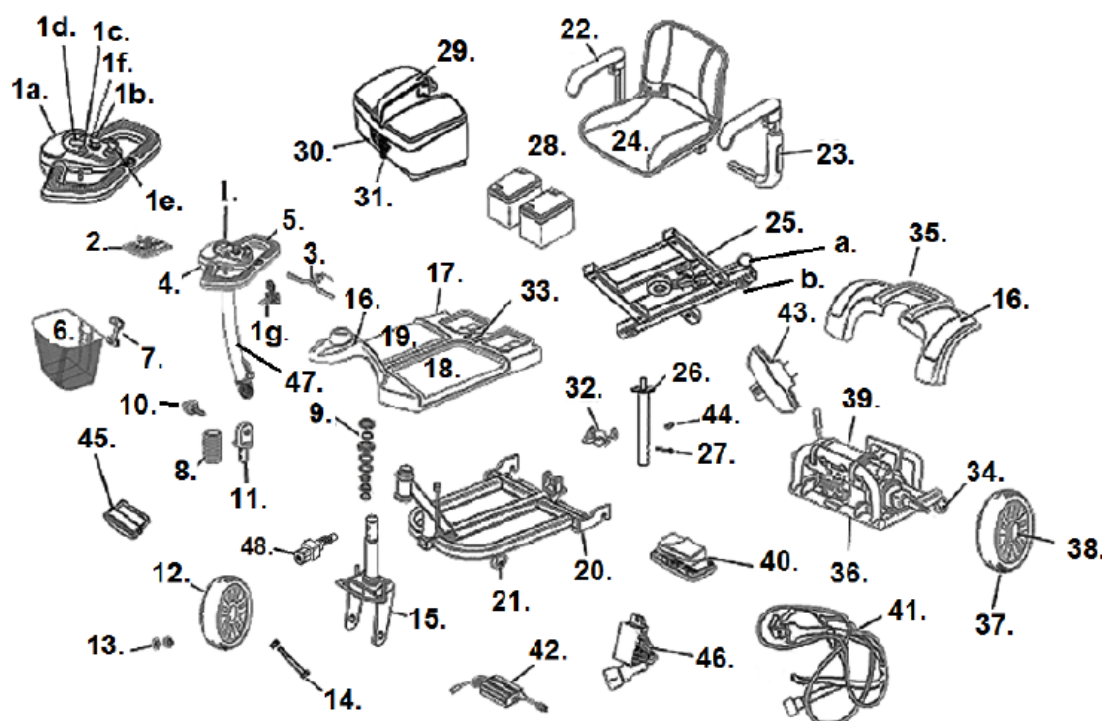

- | | |
|---|---|
| 1. Control Panel Assembly - LRM414100 | 24. Seat Assembly- LRM302116-02 |
| 1a. Control Panel Decorations Blue- LRM412103-03B | Back Cover- LRM302213-08 |
| Ctrl. Panel Decorations Red- LRM412103-03R | Seat Cover- LRM302213-09 |
| 1b. Speed Control- LRM412103-10 | 25. Seat Plate- LDR420011-FN |
| Speed Control Knob- LRM412103-11 | a. Seat Swivel Knob- SFPROSEATKN |
| 1c. Horn & Horn Switch- LRM412103-08 | b. Seat Accessory Knob- LDR410011-12 |
| 1d. Battery Gauge Led- LRM412103-LED | 26. Seat Post- LRM412100-FN |
| 1e. Key Switch- LRM402323-02-HL | 27. Seat Post Pin- S31152 |
| Key- LRM112110-02 | 28. 12ah Battery (2 needed)- LRM402109 |
| 1f. Headlight Switch- LRM414103-16 | 12ah Battery Case Only- LRM412108 |
| 1g. Tiller Charger Port- LRM414103 | 12ah Batter Case & Batteries- LRM412120 |
| 2. PC Board (LED gauge)- LRM402104-02 | 21ah Battery (2 needed)- LRM412123 |
| 3. Throttle Assembly- LRM302102-02 | 21ah Battery Case Only- LRM412121 |
| Reverse Throw Throttle- LRM302102-02RT | 21ah Batter Case & Batteries- LRM412122 |
| 4. Lower Control Panel- LRM414102 | 29. Battery Box Handle- LRM412108-01 |
| 5. Hand Grip- LRM414101-11 | 30. 20 Amp Circuit Breaker- LRM302107 |
| 6. Basket- LRA402226 | Substitute- LRM302207 |
| 7. Basket Bracket- LRM412101-08 | 31. Battery Box Charger Port- LRM414103 |
| 8. Tiller Boot- LRM402200-06 | 32. Battery Case Clamp- AZ3022 |
| 9. Tiller Bearing Set- LRB402348 | 33. Battery Contact Plate- LRM412118-10 |
| 10. Tiller Adjustment Knob- LRA402201-23 | 34. Anti-Tipper Wheel- LRM302114 |
| 11. Lower Tiller Swivel- LRB402345 | 35. Rear Shroud- LRM412119 |
| 12. Front Wheel (Black)- LRM412112-03 | 36. Rear Frame- LRM412124 |
| 13. Front Wheel Bearing (set)- STDS4V4700 | 37. Rear Wheel- LRM412113-02 |

- 13. Front Wheel Bearing (ea)- S1D3414700
- 14. Front Axle & Spacer- LRM412114
- 15. Fork- LRM412111
- 16. Decoration Set (inc. pt 1a.) Blue- LRM412110B
Decoration Set (inc. pt 1a.) Red- LRM412110R
- 17. Front Shroud- LRM412115
- 18. Floor Mat Left- LRM412118L
- 19. Floor Mat Right- LRM412118R
- 20. Front Frame- LRM412116
- 21. Front Anti-Tipper Wheel & Hdw.- LRM302114-F
- 22. Armrest Pad- LRM302115-01
- 23. Armrest Assembly Left- LRM302115L-02
Armrest Assembly Right- LRM302115R-02
Armrest Knob- S31052

- 38. Hub Cap- LRM412202-HC
- 39. Motor/Gearbox/Brake Assembly- LRM412106
Motor- LRM302106-02
Gearbox- LRM412106-01
Brake- LRM412106-03
- 40. Controller- LRM412105
- 41. Wire Harness- LRM414109
- 42. Charger- LRM302111
- 43. Frame Latch- LRM412133-01
- 44. Seat Post Screw- LRM412100-14
- 45. Head Light Assembly- LRM414107
- 46. Contact Block Set- LRM412101-12
- 47. Tiller- LRM414111
- 48. Tiller Lock- LRM412131-03

[Diagrams](#)

[Home](#)

Spitfire EX2 (3 wheel)_rev. 02_09_2021