


PANTHER

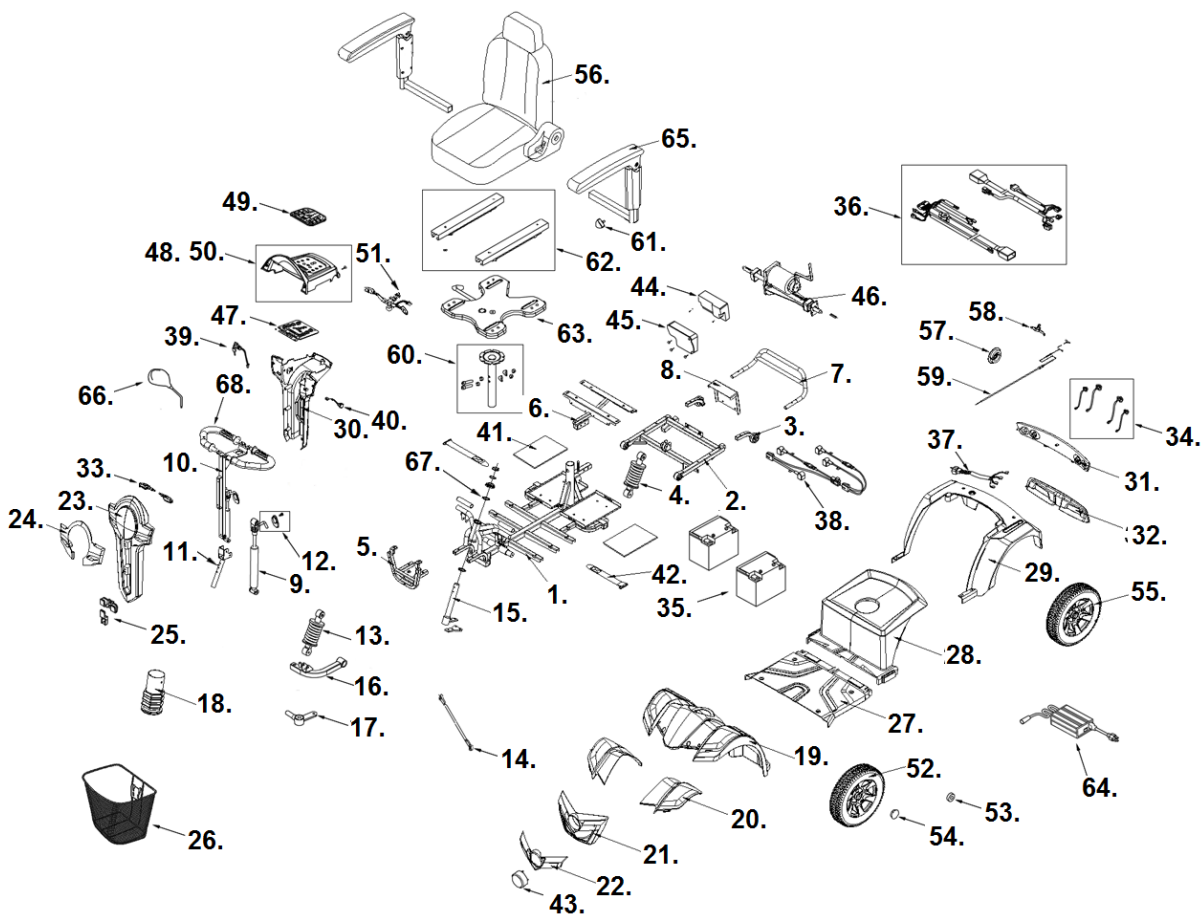

1. Front Frame- C01-077-00101
2. Rear Frame- C01-077-00200
3. Anti-Tipper Wheel- C28-032-00100
4. Rear Shock- C04-040-00100
5. Front Bumper- C01-077-00400
6. Rear Shroud Bracket- C01-077-00500
7. Rear Bumper- C01-077-00600
8. Controller Bracket- C10-077-00100
9. Tilt Piston- C02-029-00200
10. Upper Tiller- C02-077-00200
11. Lower Tiller- C02-036-00400
12. Tiller Release Handle- PANTHER-01
13. Front Shock- C03-002-00300
14. Tie Rod- PANTHER-02
15. Steering Fork- PANTHER-03
16. Control Arm Left- C03-077-00201
 Control Arm Right- C03-077-00301
17. Steering Knuckle- C03-001-00107
18. Tiller Boot- C25-045-00100
19. Front Shroud- C08-077-00100
20. Front Shroud Decoration Red- PANTHER-04
 Front Shroud Decoration Blue- PANTHER-05
21. Front Bumper- C08-077-00400
22. Front Bumper Decoration Red- C08-077-00518
 Front Bumper Decoration Blue- C08-077-00551
23. Front Tiller Cover Red- C08-077-00618
 Front Tiller Cover Blue- C08-077-00651
24. Front Tiller Decoration- C08-077-00700
25. Basket Mounting Bracket- BC35051
26. Basket- C23-051-00200 [1]
 Substitute- S35003-PL
27. Floor Shroud- C08-077-01000
28. Battery Shroud- C08-077-01100
29. Rear Shroud Red- C08-077-01218
 Rear Shroud Blue- C08-077-01251
30. Rear Tiller Shroud- C08-077-00900
31. Light Shroud Decoration- C08-077-01400
32. Light Shroud- C08-077-01300
33. Front Right Turn Signal Light- C15-062-00401
 Front Left Turn Signal Light- C15-062-00801
34. Rear Right Turn Signal Light- C14-062-00600
 Rear Left Turn Signal Light- C14-062-01000
 Rear Right Stop Signal Light- C14-077-00100
 Rear Left Stop Signal Light- C14-077-00200
35. Battery 50 Ah (2 needed)- LRB402308
 Battery 75 Ah (2 needed)- LRC402407
36. Wire Harness Assembly- C18-077-00100
37. Rear Light Harness- C18-062-00400
38. Negative Battery Harness- C17-029-00403
 Positive Battery Harness- C17-029-00303
 Battery Harness 125A Fuse- 6AYK5

39. Key Switch- C18-001-01501
 Key- C37-001-02300
40. Charger Socket- C12-051-00200
41. Battery Pad- C25-005-00100
42. Battery Strap- C31-029-00101
43. Head Light- C13-077-00100
44. Controller- C10-077-00200
45. Controller Cover- C26-051-00501
46. Motor, Brake, Transaxle Assy.- C09-077-00100
 Transaxle Only- C09-076-01200
 Motor Key Way- 29002W7011
 Motor Only- C09-077-00300
 Motor Brush Set (4 ea)- C09-029-07900
 Motor Brush Covers (4 ea)- C09-029-04500
 Electro Magnetic Brake- C09-076-00700
 Free Wheel Lever- C09-077-00401
47. Control Panel PC Board- C10-051-00507
48. Upper Tiller Cover- C08-077-00865
49. Overlay- C10-051-00601
50. Battery Indicator- C27-077-00100
51. Throttle- S12-051-00100
52. Front Wheel Assembly- S05-051-00400
 Tire- C06-051-00100
 Inner Tube- C06-051-00201
 Front Axle Bolt- S350112AN
53. Front Wheel Bearing- STDS2A5303
54. Hub Cap- No Longer Available.
55. Rear Wheel Assembly Left- S06-051-00600
 Rear Wheel Assembly Right- S05-051-00500
 Tire- C06-051-00100
 Inner Tube- C06-051-00201
 Rear Axle Bolt- S350112AN
56. 20” Seat Assy. Arms & Seat Belt- S07-062-00100
 22” Seat Assy. Arms & Seat Belt- S07-062-00200
 Headrest Only- C07-062-00400
57. Brake- PANTHER-08
58. Brake Lever- PANTHER-09
 Conversion Kit Lt To Rt Hand- S35-077-00100
59. Brake Cable Assembly- PANTHER-10
60. Seat Post Assembly- PANTHER-11
61. Adjustment Knob- S31052
62. Seat Slider Set- C07-016-00600
63. Seat Plate- C07-062-00202
64. Charger- LRA402210
65. Arm Pad Only- C07-061-01900
 Arm Rest Assembly Left- C07-061-01700
 Arm Rest Assembly Right- C07-061-01800
66. Right Mirror- VENT-53
 Left Mirror- VENT-54
67. Steering Bearing Assembly- S02-001-00601
68. Handlebar (left/right)- C02-062-00101

[1] When stock is gone the part is no longer available.