


790 3H & 5B SERIAL

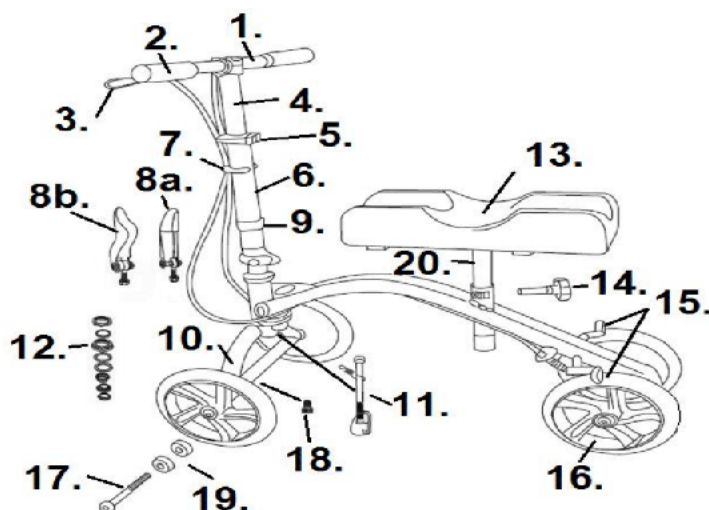

- 1. Handlebar- 9503H79001
- 2. Hand Grip- 9503H79002
- 3. Brake Assembly 3H S/N Left- 9503H79003
Brake Assembly 3H S/N Right- 9503H79004
Brake Assembly 5B S/N Left- 9505B79003
Brake Assembly 5B S/N Right- 9505B79004
Cable Only (85.5")- 9503H79023
- 4. Handlebar Column- 9503H79005
- 5. Handlebar Adj. Lock 3H S/N- 9503H79006
Handlebar Adj. Lock 5B S/N- 9505B79006
- 6. Steering Column- 9503H7907
- 7. Brake Cable Guide- 9503H79008
- 8a. Steering Column Lock- [1]
- 8b. Blade Steering Column Lock- [1]
- 9. Folding Latch Safety Lock- [1]
(Not needed with 8b.)
- 10. Steering Fork- 9503H79011
- 11. Column to Fork Locking Bolt- 9503H79012

- 12. Fork Stem Bearing- 9503H79013
- 13. Knee Pad- 9503H79014 [2]
(Stock is gone, order new style pad.
Knee Pad- 9503H79014-N [1]
(Will fit either new or old-style seat)
Knee Pad Hardware- 9503H79024
- 14. Knee Pad Height Adj. Knob- [1]
- 15. Wheel Lock 3H S/N Left- 9503H7901
Wheel Lock 3H S/N Right- 9503H790
Wheel Lock 5B S/N Left- 9505B7901
Wheel Lock 5B S/N Right- 9505B790
- 16. Caster- 9503H79018
- 17. Caster Axle- 9503H79019 [2]
- 18. Axle Screw Set (4 ea)- 9503H79020
- 19. Caster Bearing (pr)- 9503H79021
- 20. Knee Pad Post- [1]
(Stock is gone, order new style pad
Knee Pad Post- 9503H79022N [3]

[1] No longer available

[2] When stock is gone the part is no longer available.

[3] Serial # 3H160523661 and higher.

07/19/2

Diagrams

Home