


RTL799

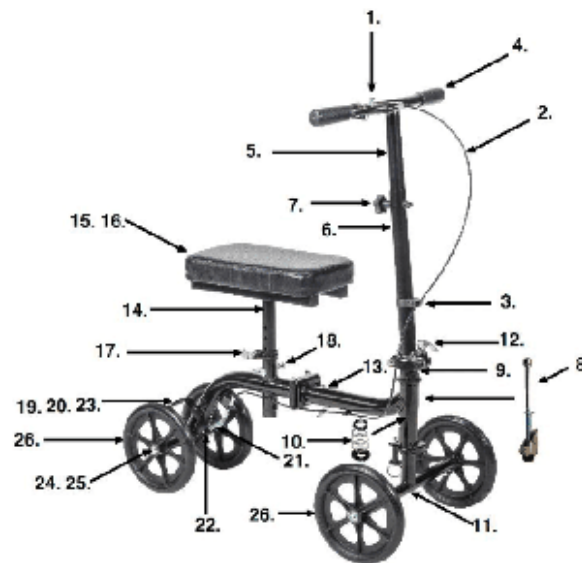

- | | |
|---|--|
| 1. Handbrake & Cable- 95021S79910 | 14. Knee Post- 95021S79911 |
| 2. Brake Cable Only- 95021S79902 | 15. Knee Pad & Hardware- 95021S79912 |
| 3. Cable Clip & Hdwr- 95021S79918 | 16. Knee Pad screws (4 ea)- 95021S79901 |
| 4. Hand Grip- 95021S79917 | 17. Knee Post Lever & Hdwr- 95021S79913 |
| 5. Upper Handlebar & Hdwr - 95021S79921 | 18. Knee Post Adj. Pin- 95021S79914 |
| 6. Lower Handlebar & Hdwr - 95021S79923 | 19. Wheel Lock Left- 95021S79903 |
| 7. Tiller Adj. Knob- 95021S79907 | 20. Wheel Lock Right- 95021S79904 |
| 8. Tiller to Fork Locking Bolt Assy.- 95021S79920 | 21. Wheel Lock Spring- 95021S79905 |
| 9. Folding Hinge- 95021S79919 | 22. Brake Connecting Bar & Hdwr- 95021S79906 |
| 10. Fork Stem Bearing (set)- 95021S79916 | 23. Wheel Lock Hdwr- 95021S79927 |
| 11. Front Fork- 95021S79922 | 24. Axle Bolt- 95021S79908 |
| 12. Folding Lever Assy.- 95021S79924 | 25. Axle Locking Bolts (4 ea)- 95021S79925 |
| 13. Frame Folding Lever- 95021S79926 | 26. Wheel- 95021S79909 |

[Diagrams](#)

[Home](#)