


KANGA TILT IN SPACE

PEDIATRIC

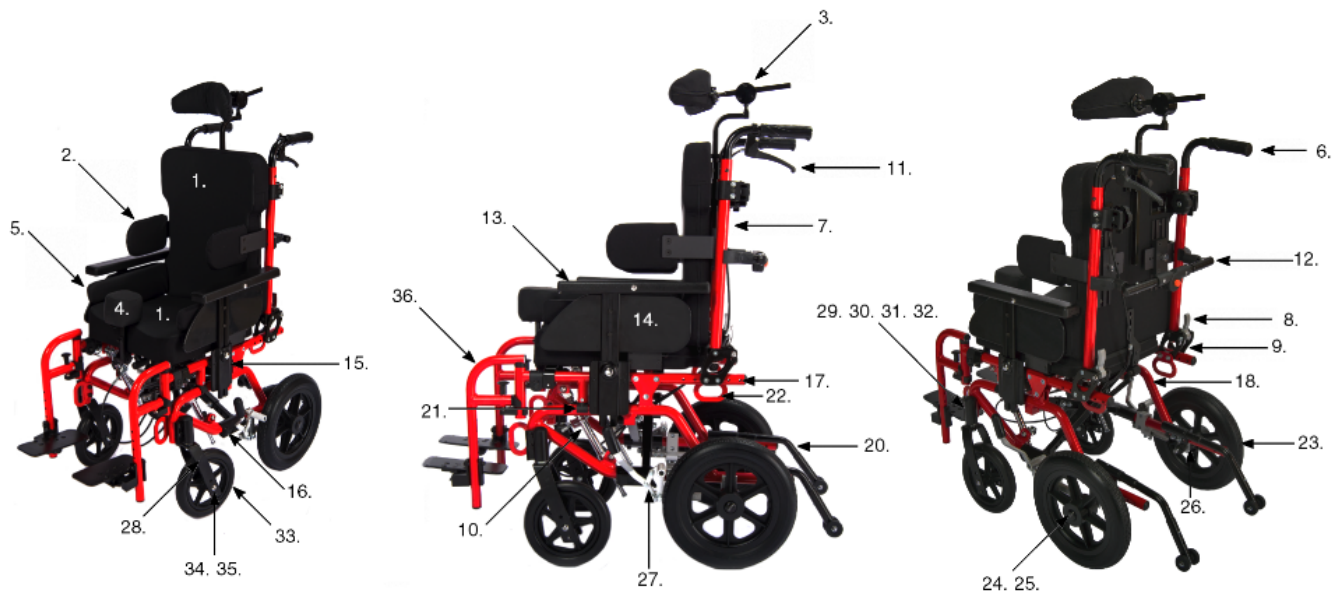

1. Seat & Back Package 10"- 1300-921110
Seat & Back Package 12"- 1300-921112
Seat & Back Package 14"- 1300-921114
2. Trunk Support 10" (pr)- 3040-021010
Trunk Support 12" (pr)- 3040-021012
Trunk Support 14" (pr)- 3040-021014
3. Headrest & Mounting Hdw 10"- 4008-401010
Headrest & Mounting Hdw 12"- 4008-401012
Headrest & Mounting Hdw 14"- 4008-401014
4. Abductor & Mounting Hdw 10"- 11060-10110
Abductor & Mounting Hdw 12"- 11060-10112
Abductor & Mounting Hdw 14"- 11060-10114
5. Thigh Support & Mounting Hdw 10"- 11072-34410
Thigh Support & Mounting Hdw 12"- 11072-34412
Thigh Support & Mounting Hdw 14"- 11072-34414
6. Push Handle & Handgrip- KANGA-01
Handgrip- STDS871
7. Back Tube- KANGA-02
8. Back Tube Mounting Brkt (pr)- KANGA-14
9. Tilt Piston Left- KGP2-01
Tilt Piston Right- KGP2-02

10. Tilt Release Handle & Cable- KGP2-12
11. Back Release Assembly- KANGA-08
12. Spreader Bar Assembly- KANGA-07
13. Arm Pad- KANGA-09
14. Armrest Assembly Left- KANGA-10
Armrest Assembly Right- KANGA-11
15. Armrest Receiver- KANGA-12
16. Cross Frame- KGP2-07
17. Adj Depth Seat Frame Left KGP2-09
Adj Depth Seat Frame Right KGP2-08
18. Side Frame Left- KGP2-10
Side Frame Right- KGP2-11
19. Anti-Tipper Receiver- KGP2-04
20. Anti-Tipper (pr)- KGP2-13
21. Seat Saddle- KANGA-05
22. Bus Tie Down & Hardware (4 ea)1 TR 8029
23. Rear Wheel- KGP2-06
24. Rear Wheel Bearing - STDS1012
25. Rear Axle Receiver (ea)- KGP2-03
26. Quick Release Axle- KGP2-05

27. Wheel Lock Left- KANGA-03
Wheel Lock Right- KANGA-04
28. Fork- STDS3003
29. Fork Housing Left- KG 1049
Fork Housing Right- KG 1050
30. Fork Housing Bolts (4 ea)- KG 1071
31. Fork Stem Bearing- STDS1060
32. Dust Cap- STDS1051
33. Caster- STDS1A3500
34. Caster Bearing- STDS1011
35. Caster Axle- STDS1002-1
36. Footrest (pr)- KGP-FR-AA
Elevating Leg Rest- KGP-ELR-G

Accessory Not Shown

- Polycarbonate Tray 10"- 12011-10
- Polycarbonate Tray 12"- 12011-12
- Polycarbonate Tray 14"- 12011-14

04/23/2021

[Diagrams](#)
[Home](#)