


GVM3

1C SERIAL NUMBER

1. Handgrip- STDS3J2403
2. Back upholstery, 16"- STDS2AM3-011-16
18"- STDS2AM3-011-18
20"- STDS2AM3-011-20
3. Side Frame right- STDS2AM3-015R-T
left - STDS2AM3-015L-T
4. Armpad, Nylon - STDS2AM3-001-T
5. Armrest Assembly Desk Left- STDS2AM3-002-T
Right- STDS2AM3-001-T
6. Anti-Tippers- GCSTDS819
7. Frame plug- STDS000005
8. Stem bearing, sold 1ea- STDS1060
9. Fork- STDS3003
10. Crossframe- 16"- STDS2AM3-016X6
18"- STDS2AM3-016X8
20"- STDS2AM3-016X2
11. Caster- STDS1002 [1]
Caster- STDS2AQRTM3-01 [2]
12. Axle- STDS1002-1
13. Caster bearing- STDS1011
14. Dust cap- STDS8M4651
15. Frame plug- STDS2AM3-014
16. Brake assembly, right- STDS2AM3-007-T
left - STDS2AM3-008-T
17. Wheel bearing, 1ea- STDS1012
18. Rear Axle, Nut & Washer- STDS2AM3-010-T
Quick Release Axle- STDS2AQRTM3-02
19. Handrim- STDS004012-T
20. Handrim screws and spacers kit- STDS1073-T
21. Rear wheel- STDS2A1600-T
22. Crossframe assist w/ insert- STDS2AM3-009

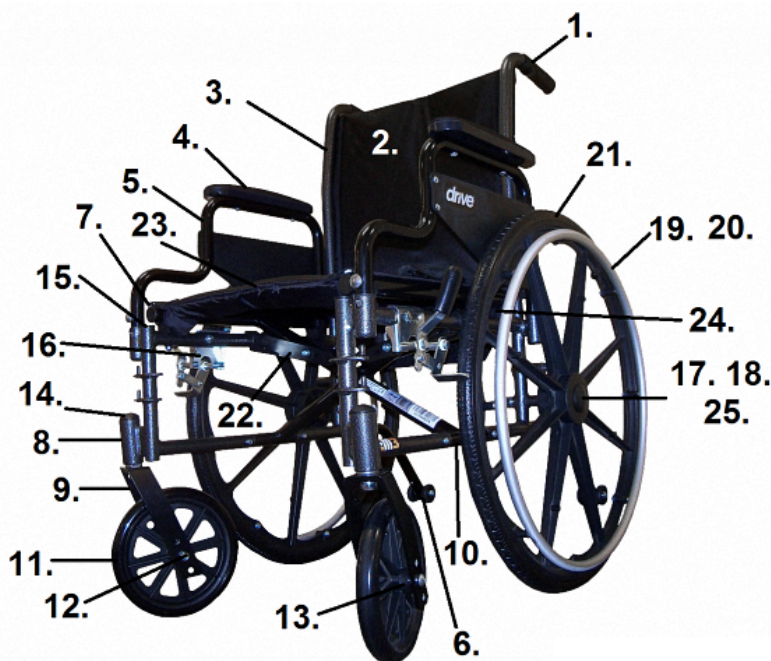

23. Seat upholstery, 16"- STDS2AM3-012-16
18"- STDS2AM3-012-18
20"- STDS2AM3-012-20
24. Front/Rear Seat Saddle (pr)- STDS2AM3-013-T
25. Hub Cap- STDS1000C

[1] With out Q/R rear axle.
[2] With Q/R rear axle.

04/28/2021

[Diagrams](#)
[Home](#)