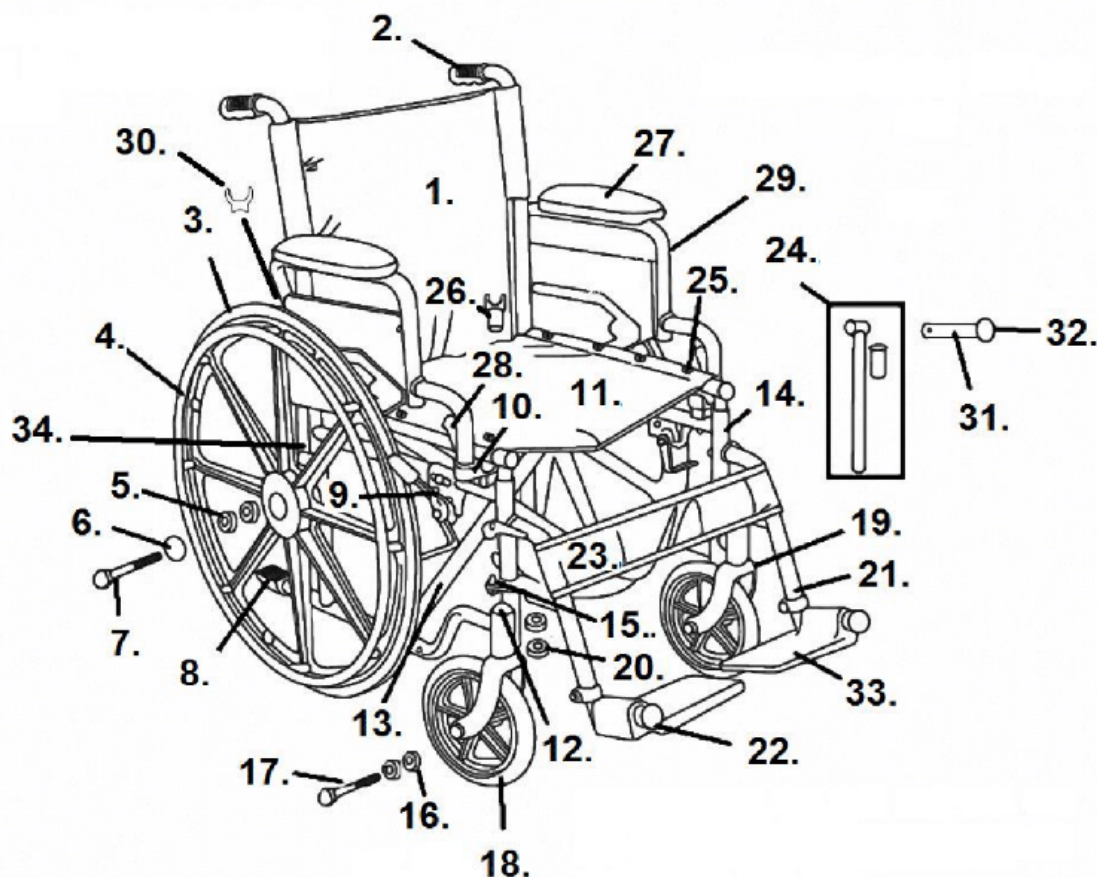


BLUE STREAK (5N SERIAL #)

- | | |
|---|---|
| 1. Back Upholstery 18"- STDS5N2208 | 17. Caster Axle- STDS5N2231 |
| 2. Handgrip- WB 8036 | 18. Caster- STDS3J2432 |
| 3. Rear Wheel Assembly- STDS3J2424 | 19. Fork- STDS5N2233 |
| 4. Hand Rim- STDS3J2400 | 20. Fork Stem Bearing- STDS8M4601
(13mm ID, 28mm OD) |
| Hand Rim Hardware (9 ea)- STDS004609-9 | 21. Footrest (pr)- STDS4S22SF |
| 5. Rear Wheel Bearing- STDS3J2413
(11mm ID, 35mm OD) | 22. Footrest Bumper- STDS1026 |
| 6. Hub Cap- STDS3J2421 | 23. Calf Strap- STDS5N2201 |
| 7. Rear Axle- STDS4S2225 | 24. Slide Tube w/Bushing- STDS5N2210 |
| 8. Tipping Sleeve- STDS4020 | 25. Seat Upholstery Hardware (8 ea)- STDS3J4510 |
| 9. Wheel Lock Left- STDS4S221L | 26. Seat Rail Guide- STDS1070 |
| Wheel Lock Right- STDS4S221R | 27. Arm Pad- STDS5N2219 |
| Wheel Lock Tip- STDS8M4603 | 28. Arm Release Lever- STDS5N2219 |
| 10. Seat/Arm Saddle- STDS5N2202 | 29. Armrest Desk Left- STDS5N22DL |
| 11. Seat Upholstery 18"- STDS3J2418 | Armrest Desk Right- STDS5N22DR |
| 12. Dust Cap- STDS8M4651 | 30. Arm Guide Assembly- STDS5N2205 |
| 13. Cross Frame 18"- [1] | 31. Slide Tube Retainer- STDS5N2235 |
| 14. Side Frame Left- [1] | 32. Frame Plug- STDS5N2234 |
| Side frame Right- [1] | 33. Footplate- STDS4S0906-1 |
| 15. Hanger Pins (4 ea)- STDS1069-4 | 34. Rear Armrest Receiver- [1] |
| 16. Caster Bearings- STDS1011 | |

(8mm ID, 22mm OD)

[1] Part is no longer available.

[Diagrams](#)

[Home](#)

k (5N)rev. 04_16_2021