




CIRRUS IV 5D SERIAL

AFTER 6/10 HEIGHT ADJUSTABLE ARMS



**Only parts listed below
are available.**

1. Back Upholstery 16"- STDS5D2441
18"- STDS5D2443
20"- STDS5D2445
2. Hand Grip- STDS3J2403
3. Rear Wheel- STDS5D2424
4. Handrim- STDS3J2400
5. Handrim Screw- STDS004609-9
6. Hub Cap- STDS5D2421
7. Rear Wheel Axle- STDS5D4109
8. Rear Wheel Bearing- STDS3J2413
9. Caster Axle- STDS8M4631
10. Caster Bearing- STDS1011HD
11. Caster- STDS4S2432
12. Fork- STDS8M4633
13. Fork Stem Bearing- STDS8M4601
14. Fork Stem Cap- STDS8M4651
15. Brake Right- STDS5D241R
16. Brake Tip- STDS8M4603
17. Cross Frame 16"- STDS5D2416
18"- STDS5D2418
20"- STDS5D2420
18. Cross Frame Assist 20"- STDS5D2440
19. Seat Upholstery 16"- STDS5D2442
18"- STDS5D2444
20"- STDS5D2446
Seat Screws (8 ea)- STDS001072-8
20. Arm Pad Desk- STDS5D2446
Full- STDS5D2446
21. Arm Release Lever- STDS3J2419
22. Adjustable Height Arm
Desk Left- STDS5D24DL
Right- STDS5D24DR
Full Left- STDS5D24FL
Right- STDS5D24FR
Side Panel- STDS5D2412
23. Seat Upholstery Screws (8ea)- STDS1072-8
Back Upholstery Screws (8ea)- STDS5D5504
24. Front Seat Saddle/Arm Receiver- STDS3J2412
25. Rear Seat Saddle- STDS3J2414
26. Rear Arm Receiver- STDS5D4067
27. Side Frame Left- STDS5DC4SFL
28. Tipping Sleeve- STDS3J2422
29. Backrest Cane- STDS5D4015
30. Backrest Adj. Knob- STDS5D895
31. Frame Plugs (4 ea)- STDS5D0005
32. Footrest- STDS3J24SF
Elevating Legrest- LK3JELR
33. Arm Height Adj. Lever- STDS5D2447

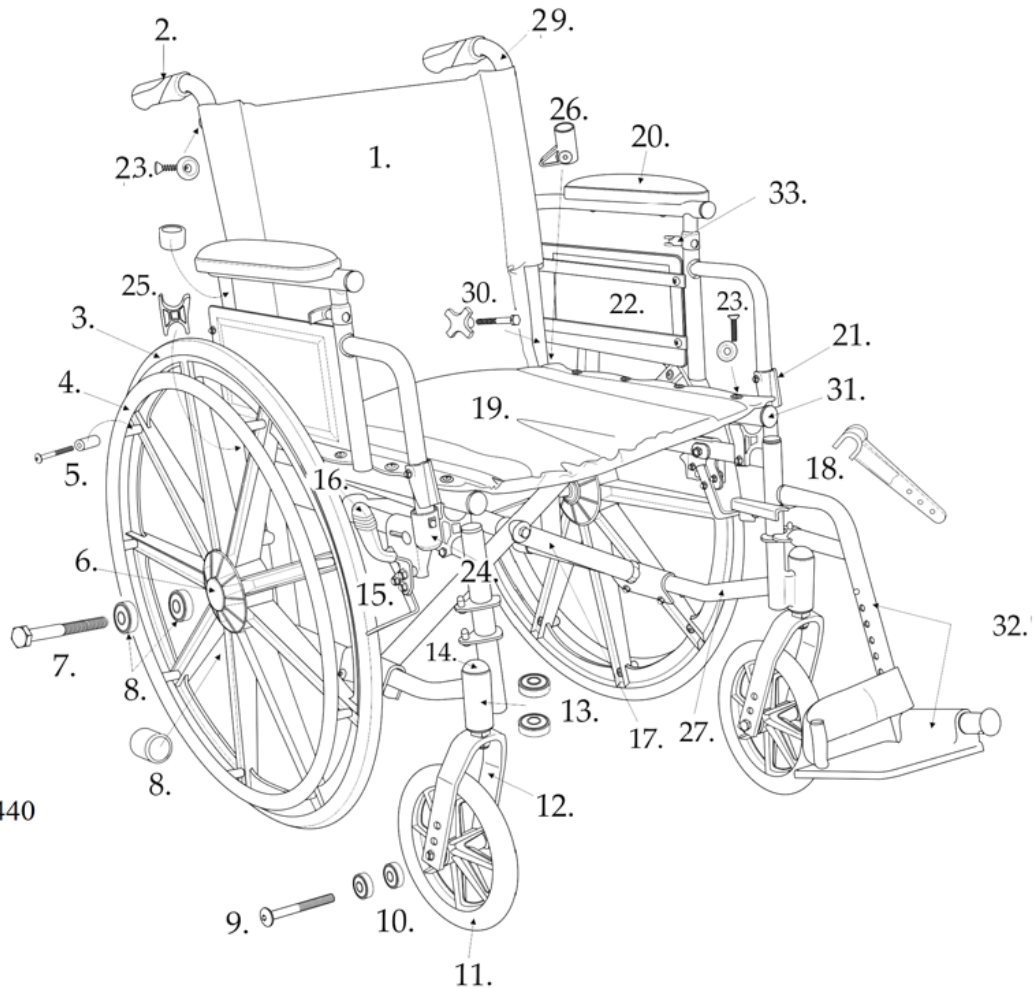

04/19/2021