


SILVER SPORT VI

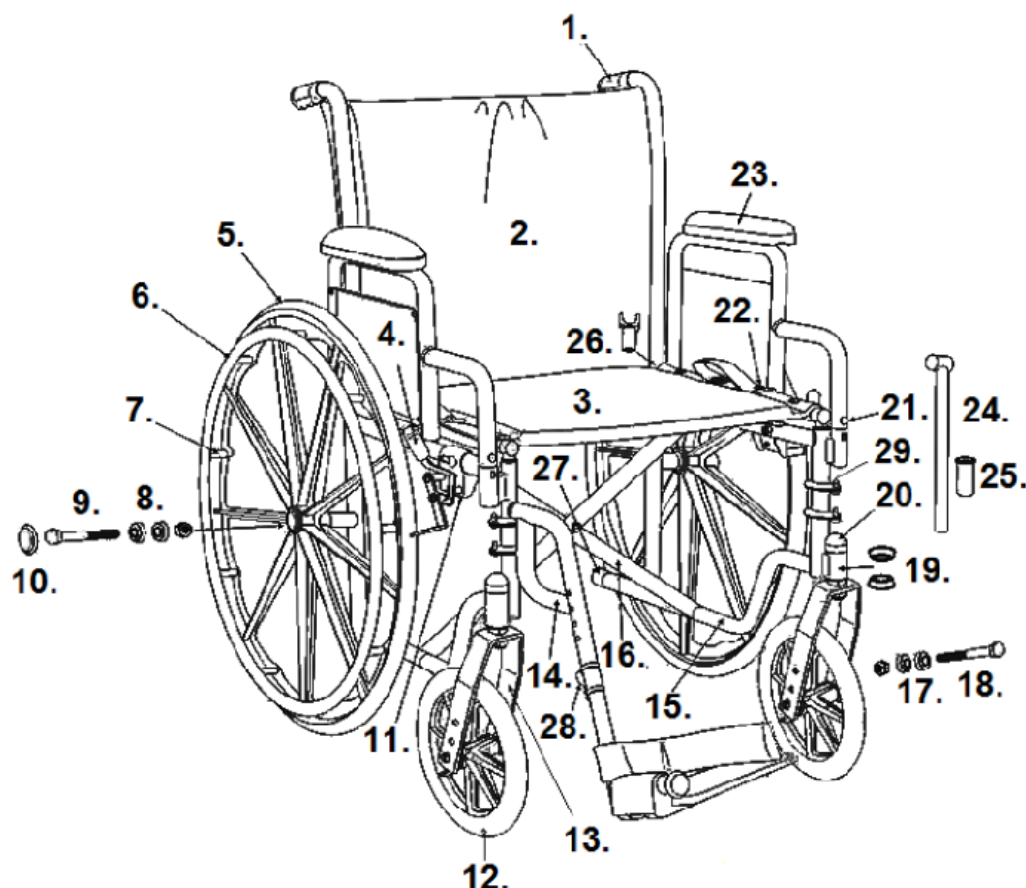

- | | |
|---|--|
| 1. Handgrip- WB 8036 | 16. Cross Frame 18"- STDS3J45X8
Cross Frame 20"- STDS3J46X2 |
| 2. Back Upholstery 18"- STDS3J4608
Back Upholstery 20"- STDS3J4602 | 17. Caster Bearing (11mm)- STDS4S3602 |
| 3. Seat Upholstery 18"- STDS3J4618
Seat Upholstery 20"- STDS3J4620 | 18. Caster Axle- STDS8M4631 |
| 4. Wheel Lock Tip- STDS8M4603 | 19. Fork Stem Bearing (12.5mm)- STDS8M4601 |
| 5. Rear Wheel- STDS3J2424 | 20. Fork Stem Cap- STDS8M4651 |
| 6. Hand Rim- STDS3J2400 | 21. Arm Release Button- STDS3J4503 |
| 7. Hand Rim Hardware (9 ea)- STDS004609-9 | 22. Uphol. Hardware Back/Seat (8 ea)- STDS2015 |
| 8. Rear Wheel Bearing (11mm)- STDS3J2413 | 23. Desk Arm Pad- STDSDSKVB
Full Arm Pad- STDSFLVB |
| 9. Rear Axle- STDS3J4626 | Desk Arm Left- STDSSSDDAL
Desk Arm Right- STDSSSDDAR |
| 10. Hub Cap- STDS3J2421 | Full Arm Left- STDSSSDFAL
Full Arm Right- STDSSSDFAR |
| 11. Wheel Lock Left- STDS8M462L
Wheel Lock Right- STDS8M462R | 24. Slide Tube & Bushing- STDS3J4601 |
| 12. Caster- STDS4S2432 | 25. Bushing- STDS002601 |
| 13. Fork- STDS8M4633 | 26. Seat Rail Guide- STDS1070 |
| 14. Footrest- STDS3J24SF
Elevating Leg Rest- LK3JELR | 27. Tipping Sleeve- STDS3J2422 |
| 15. Side Frame Left- STDS3J46L1
Side Frame Right- STDS3J46R1 | 28. Footrest Adj. Lever- STDS3J4502 |
| | 29. Hanger Pins (4 ea)- STDS4S1069 |