


SENTRA EC

2A SERIAL

**Only parts listed below
are available.**

1. Back Upholstery- (16") STDS8M4606
(18") STDS2003BLK, [1] (20") STDS2006BLK
(22") STDS2030BLK, (24") STDSIS2634
2. Back Upholstery Screw- STDS5D5504 (8 ea)
3. Rear Wheel- (Black) STDS1006, (Chrome) STDS1000
22" & 24"- STDS1000HD
4. Rear Axle Bearing- STDS1012
22" & 24"- STDS1012HD
5. Rear Axle Bolt- (Removable Arm) STDS1005
22" & 24"- STDS1053
6. Hub Cap- STDS1000C
7. Tipping Sleeve- STDS4020
8. Hand rim- (Black) STDS004012
(Chrome - Fixed Spacer) STDS005008
9. Rubber Brake Tip- STDS4011
10. Brake Assm.- (Rem Arm) STDS4010L or R
(Fixed Arm) STDS4019L or R
11. Front Axle Bolt- STDS1002-I
12. Front Axle Bearing- 14" - 20" STDS1011
22" - 24" STDS1011HD
13. Front Caster Fork- STDS3003
14. Fork Stem Bearings- STDS1060
15. Fork Stem Cap- STDS1051
16. Seat Upholstery- (18") STDS8M4618
(20") STDS8M4620, (22") STDS2025BLK
(24") STDS2332BLK
17. Seat Rail Guide- STDS1070
18. Seat Upholstery Screw- STDS1072-8 (8 ea)
19. Handgrip- STDS871
20. Arm Assm.- (Desk) STDSDDAECR & L
(Full) STDSDFAECR & L
(Adj Desk) STDSADDASPL or R
(Adj Full) STDSADFASPL or R
21. Arm Pad- (Desk) STDSDSKVB
(Full) STDSFLVB
22. Slide Tube- (14" - 20") STDS002600,
(22" - 24") STDS005200

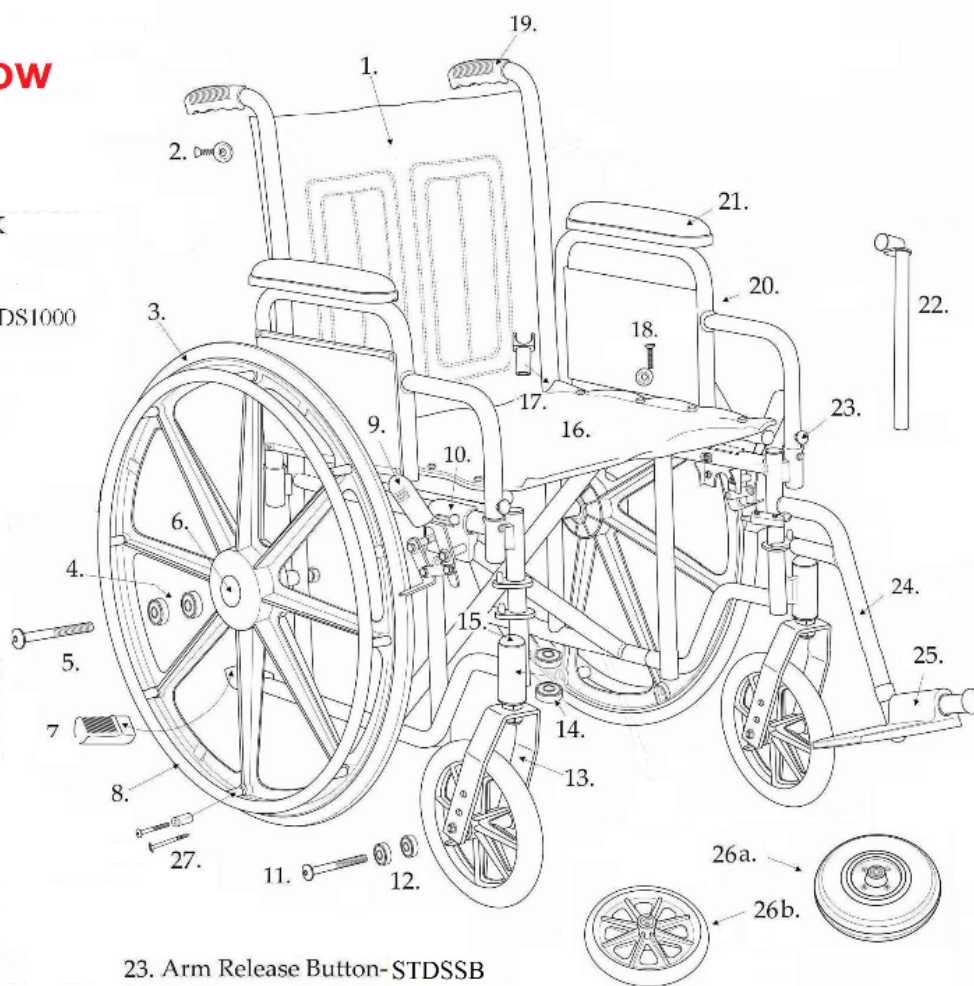

23. Arm Release Button- STDSSB
24. Footrest- (16" - 24") STDSF-TF
25. Footplate- STDS00004A
- 26a. Caster- (22" - 24") STDS1075
b. Caster- (14" - 20") STDS1002
27. Handrim Hardware- (Black Rim Only) STDS1073

[1] When stock is gone the parts is no longer available.

04/27/2021

Diagrams

Home