


SENTRA EC

M SERIAL

Only parts listed below are available.

1. Back Upholstery 20"- STDS1S2630
22"- STDS1S2632
24"- STDS1S2634
2. Back Uphl Screws (8 ea)- STDS5D5504
3. Rear Wheel (16"-20")- STDS1006
(22"-24")- STDS1000HD
4. Rear Wheel Bearing (16"-20")- STDS1012
(22"-24")- STDS1S1012
5. Rear Axle (16"-20")- STDS1005
(22"-24")- STDS1S1053
6. Hub Cap- STDS1000C
7. Tipping Sleeve- STDS4020
8. Hand Rim Black- STDS004012
Chrome- STDS005008
9. Brake Tip- STDS4011
10. Brake Left- STDS4019L
Right- STDS4019R
Substitute Left- STDS4010L
Right- STDS4010R
11. Front Axle (16"-20")- STDS1002-1
(22"-24")- STDS8M4631
12. Front Wheel Bearing (16"-20")- STDS1011
(22"-24")- STDS1011HD
13. Fork (16"-20")- STDS3003
(22"-24")- STDS3045-2
14. Fork Stem Bearing (16"-20")- STDS1060
(22"-24")- STDS8M4601
15. Dust Cap- STDS1051
16. Seat Upholstery 18"- STDS1S4618
20"- STDS2008BLK
22"- STDS2025BLK
24"- STDS2332BLK
17. Seat Rail Guide- STDS1070
18. Seat Uphl Screws (8 ea)- STDS1072-8
19. Handgrip- STDS871
20. Arm Assembly Desk Left- STDSDDAECL
Right- STDSDDAECL
Full Left- STDSDFAECL
Right- STDSDFAECL
Adj. Ht. Desk Left- STDSADDASPL
Right- STDSADDASPL
Full Left- STDSADFASPL
Right- STDSADFASPL

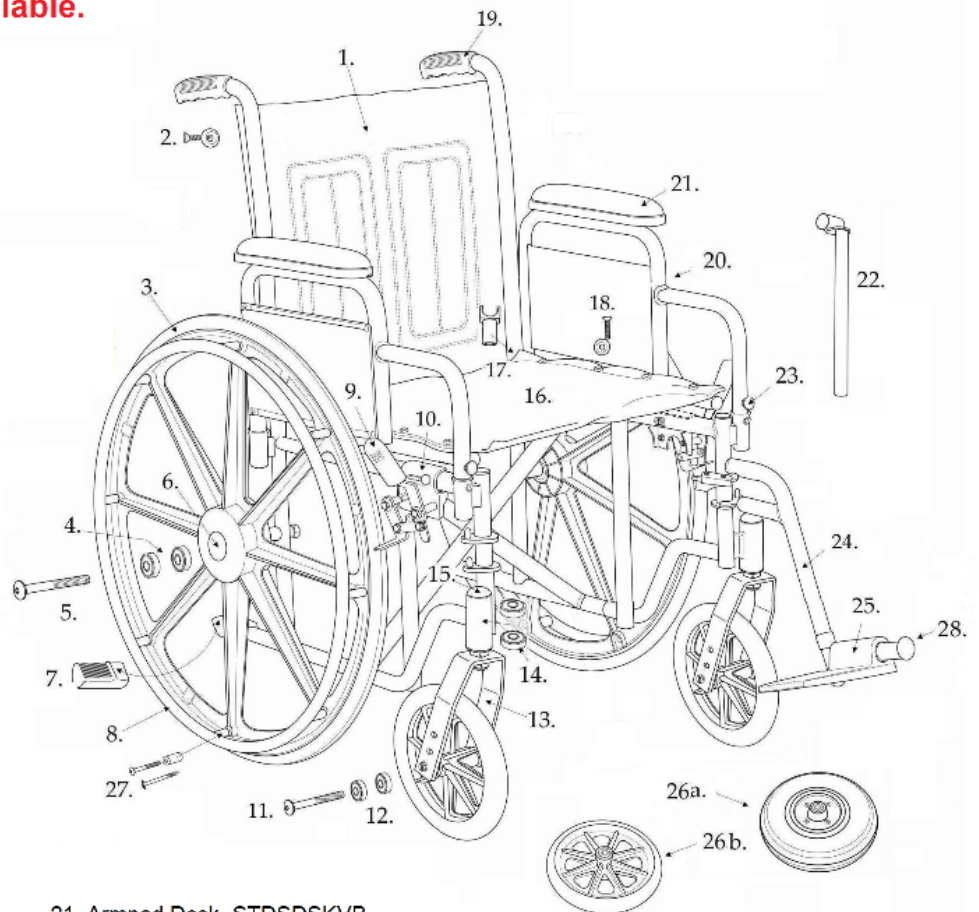

21. Armpad Desk- STDSDSKVB
Full- STDSFLVB
22. Slide Tube (16"-20")- STDS002600
(22"-24")- STDS005200
23. Arm Release Button- STDS5B
24. Footrest (16"-20")- PSTDSF-TF
(22"-24")- STDSF-TF

25. Footplate- STDS00004A
- 26a. Caster (22"-24")- STDS1075-2
- 26b. Caster (16"-20")- STDS1002
27. Handrim Hardware Blk Rim- STDS1073
28. Rubber Bumper- STDS1026

04/27/2021

[Diagrams](#)
[Home](#)