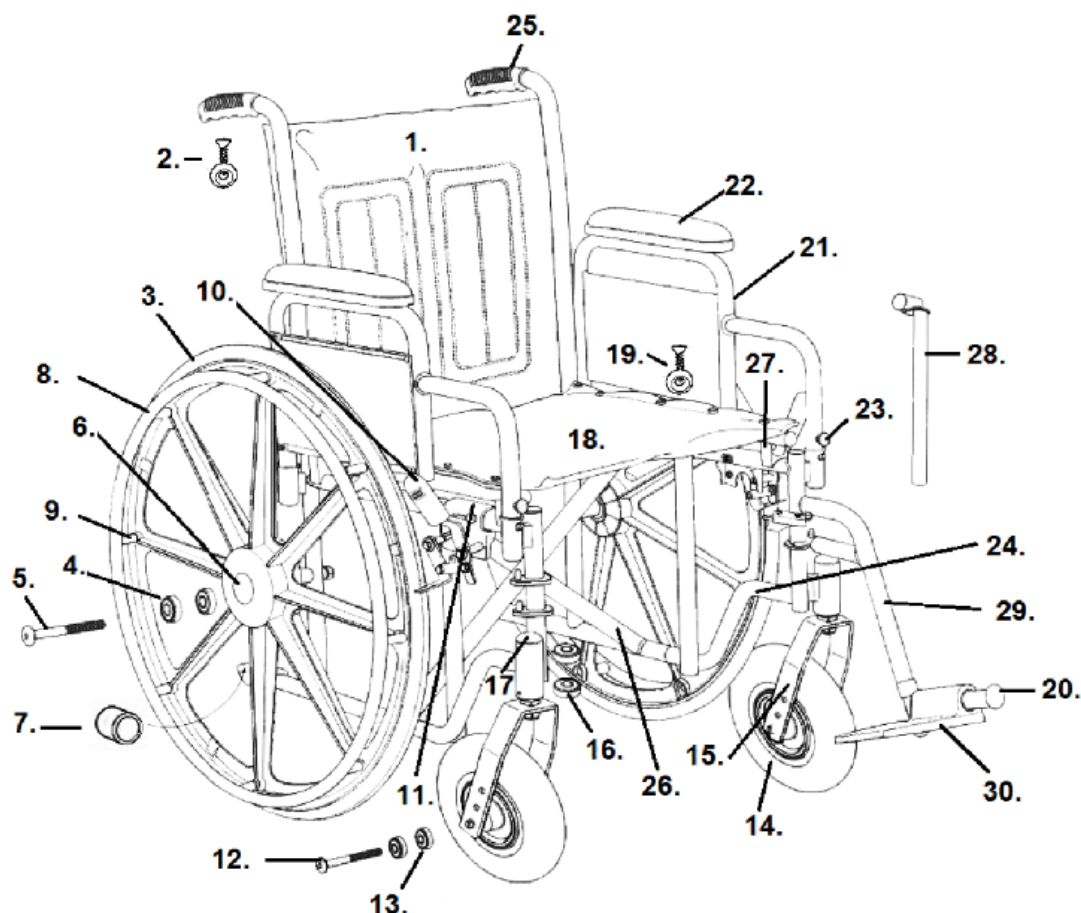


SENTRA EC (Y, 1S & 4S SERIAL #)

- | | |
|--|--|
| 1. Back Upholstery 20"- STDS1S2630
Back Upholstery 22"- STDS1S2632
Back Upholstery 24"- STDS1S2634 | 21. Armrest Assembly Desk Left- STDSDDAECL
Armrest Assembly Desk Right- STDSDDAECL
Armrest Assembly Full Left- STDSDFAECL
Armrest Assembly Full Right- STDSDFAECL
A/R Assy. Adj. Ht. Desk Left- STDSADDASPL
A/R Assy. Adj. Ht. Desk Right- STDSADDASPR
Armrest Assy. Adj. Ht. Full Left- STDSADFASPL
Armrest Assy. Adj. Ht. Full Right- STDSADFASPR |
| 2. Back Upholstery Hardware (8 ea)- STDS5D5504 | 22. Arm Pad Desk- STDSDSKVB
Arm Pad Full- STDSFLVB |
| 3. Rear Wheel- STDS1S1000HD | 23. Arm Release Button- STDSSB |
| 4. Rear Axle Bearing (ea)- STDS1S1012 | 24. Side Frame Left- STDS1S262L
Side Frame Right- STDS1S262R
Side Frame Left- STDS4S262L-N [1]
Side Frame Right- STDS4S262R-N [1] |
| 5. Rear Axle- STDS1S1053 | 25. Hand grip- STDS871 |
| 6. Hub Cap- STDS1055 | 26. Cross Frame 20"- STDSY2620
Cross Frame 22"- STDSY2622
Cross Frame 24"- STDSY2624 |
| 7. Tipping Sleeve- STDS3J2422 | 27. Seat Saddle- STDS1M2218 |
| 8. Hand Rim- STDS004010 | |
| 9. Hand Rim Hardware (8 ea)- STDS1016-8 | |
| 10. Wheel Lock Tip- STDS8M4603 | |
| 11. Wheel Lock Left- STDS4019L
Wheel Lock Right- STDS4019R | |
| 12. Caster Axle- STDS8M4631 | |
| 13. Caster Axle Bearing (ea)- STDS1011HD | |
| 14. Caster- STDS1075-2 | |
| 15. Fork- STDS1S3045 | |
| 16. Fork Stem Bearing (ea)- STDS8M4601 | |
| 17. Dust Cap- STDS1051 | |

- 18. Seat Upholstery 20"- STDS1S2650
 Seat Upholstery 22"- STDS1S2652
 Seat Upholstery 24"- STDS1S2654
- 19. Seat Upholstery Hardware- STDS2015 [3]
- 20. Footrest Bumper- STDS1026

- 28. Slide Tube- STDS005200
- 29. Footrest (pr)- STDSF-TF
- 30. Foot Plate- STDS00004A
- [1] 20" side frame after serial # 4S1608153724
- [1] 22" side frame after serial # 4S1609111400
- [1] 24" side frame after serial # 4S1610141127
- [2] When stock is gone the part is not available.
- [3] 10 needed sold 8 each.

[Diagrams](#)

[Home](#)

i-4S}_rev. 08_25_2021