


SENTRA HD (Y, 1S & 4S SERIAL#)

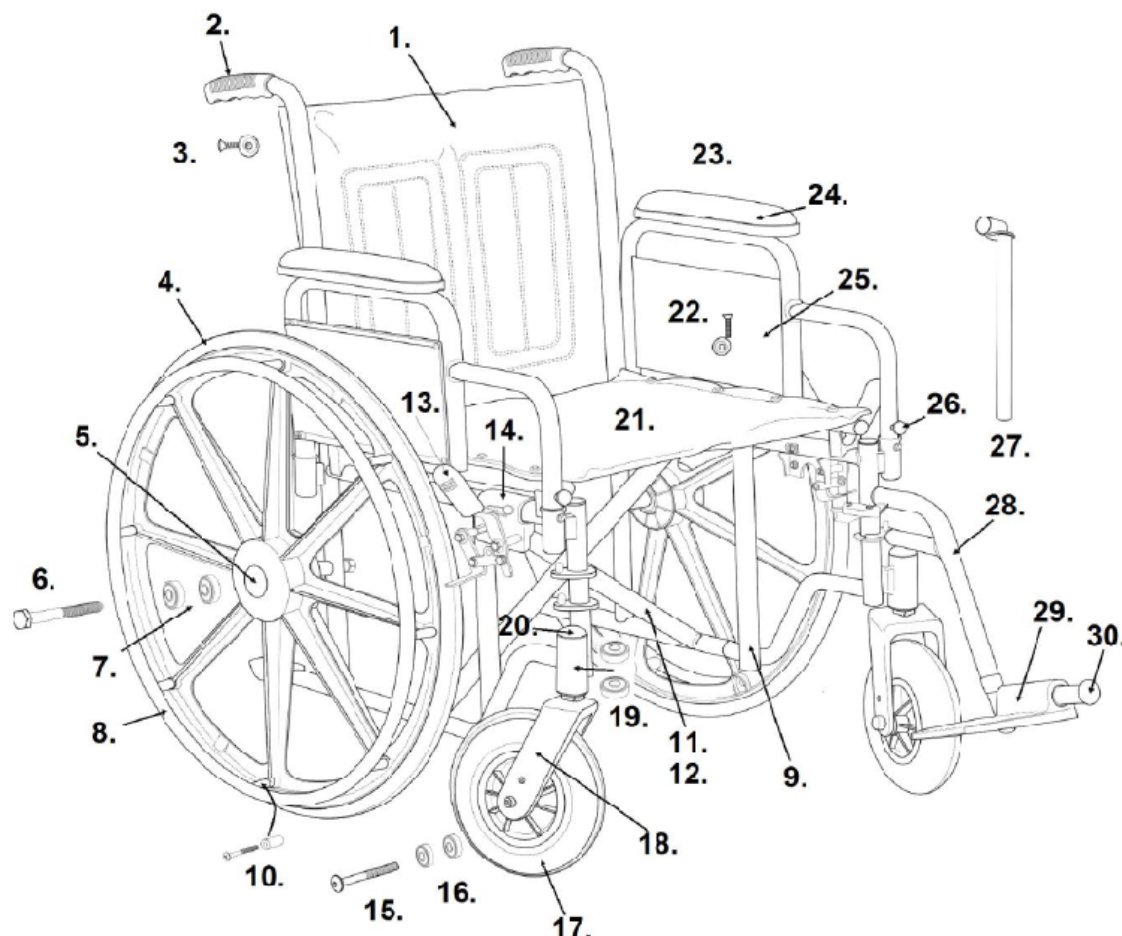

- | | |
|---|--|
| 1. Back Upholstery 20"- STDS2329BLK
Back Upholstery 22"- STDS2331BLK
Back Upholstery 24"- STDS2333BLK | 17. Caster- STDS1075 |
| 2. Handgrip- STDS871 | 18. Caster Fork- STDS3045-2 |
| 3. Back Upholstery Hardware (8 ea)- STDS5D5504 | 19. Fork Stem Bearing- STDS1060 |
| 4. Rear Wheel Assy. s/n 1S & 4S- STDS1S1000HD
Rear Wheel Assy. s/n Y- STDS1000HD | 20. Fork Stem Dust Cap- STDS1051 |
| 5. Hub Cap s/n 1S & 4S- STDS1055
Hub Cap s/n Y- STDS1000C | 21. Seat Upholstery 20"- STDS2A2328BLK
Seat Upholstery 22"- STDS2A2330BLK
Seat Upholstery 24"- STDS2A2332BLK |
| 6. Rear Axle Bolt s/n 1S & 4S- STDS1S1053
Rear Axle Bolt s/n Y- STDS1052 | 22. Seat Upholstery Hardware (8 ea)- STDS001062 |
| 7. Rear Axle Bearing s/n 1S & 4S- STDS1S1012
Rear Axle Bearing s/n Y- STDS1012HD | 23. Arm Assembly Desk Left- STDSDDASL
Arm Assembly Desk Right- STDSDDASR
Arm Assembly Full Left- STDSDFASL
Arm Assembly Full Right- STDSDFASR |
| 8. Hand Rim s/n 1S & 4S- STDS004010
Hand Rim s/n Y- STDS005008 | Arm Assy. Adj. Desk Left- STDSADDASPL
Arm Assy. Adj. Desk Right- STDSADDASPR
Arm Assy. Adj. Full Left- STDSADFASPL
Arm Assy. Adj. Full Right- STDSADFASPR |
| 9. Side Frame s/n 1S or 4S Right- STDS4S521R
Side Frame s/n 1S or 4S Left- STDS4S521L | 24. Arm Pad Desk- STDSDSKVB |

- 10. Hand Rim Hardware (8 ea)- STSTD1016-8
- 11. Cross Frame 20"- [1]
 - Cross Frame 22"- STDS2A5222
 - Cross Frame 24"- [1]
- 12. Cross Brace Bolt- STDS001061
- 13. Wheel Lock Tip- STDS8M4603
- 14. Wheel Lock Left- STDS4010L
 - Wheel Lock Right- STDS4010R
- 15. Front Axle Bolt- STDS1014
- 16. Caster Bearings- STDS1011

- Arm Pad Full- STDSFLVB
- 25. Arm Rest Panel Desk- STDS002044
 - Arm Rest Panel Full- STDS002043
- 26. Arm Release Button- STDSB
- 27. Slide Tube- STDS005200
- 28. Footrest- STDSF-TF
 - Elevating Leg Rest- STDELRTF
- 29. Footplate- STDS00004A
- 30. Rubber Bumper- STDS1026

[1] The part is no longer available.

[Diagrams](#)

[Home](#)

Sentra HD (Y-15-45)_rev. 04_28_2021