


SENTRA HD XW

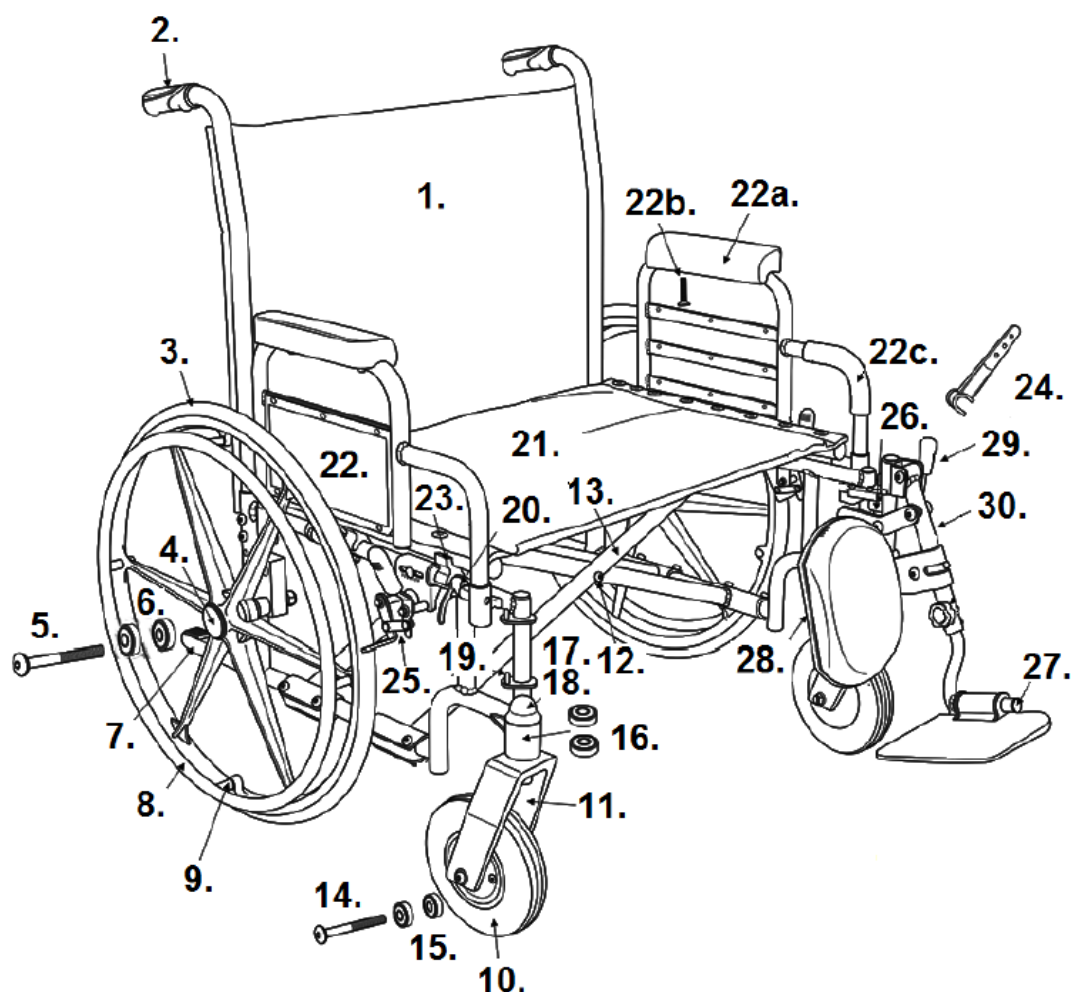

- 1. Back Upholstery 26"- STDS2389BLK
Back Upholstery 28"- STDS2391BLK
Back Upholstery 30"- STDS2393BLK
- 2. Handgrip- TRHG
- 3. Wheel Assembly- STDS1076
- 4. Rear Bearing Hub Cap- STDS2A5301
- 5. Rear Axle Bolt & Nut- STDS2A5302
- 6. Rear Wheel Bearing- STDS2A5303
- 7. Tipping Sleeve- STDS4020
- 8. Hand Rim- [1]
- 9. Hand Rim Hardware (ea)- STDS2A5311
- 10. Caster & Fork- STDS1077
Caster Only- STDS2A5320
- 11. Fork Only- [1]
- 12. Cross Frame Hinge Bolt- STDS001061
- 13. Cross Frame 26"- [1]

- 21. Seat Upholstery 26"- STDS2388BLK
Seat Upholstery 28"- STDS2390BLK
Seat Upholstery 30"- STDS2392BLK
- 22. Desk Arm Assembly Left- STDSHDDDAL
Desk Arm Assembly Right - STDSHDDAR
Full Arm Assembly Left - STDSHDDFAL
Full Arm Assembly Right - STDSHDDFAR
- 22a. Arm Pad Desk Length- STDS2A5333
Arm Pad Full Length- STDS2A5331
- 22b. Arm Rest Pad Bolts- STDS2A5334
- 22c. Arm Rest Pad Foam- STDS2A5336
- 23. Seat Rail Saddle- STDS4058
- 24. Cross Frame Assist 26"- [1]
Cross Frame Assist 28"- [1]
Cross Frame Assist 30"- [1]
- 25. Wheel Lock Left- STDS4032L

Cross Frame 28"- [1]

Cross Frame 30"- [1]

14. Front Axle Bolt & Nut- STDS2A5304

15. Front Caster Bearings- STDS002050

16. Fork Stem Bearings- STDS2050

17. Lock nut for Fork- STDS2A5316

18. Dust Cap for Fork- STDS2A5323

19. Hanger Pins (4 ea)- STDS1069-4

20. Arm Release Lever- [1]

Wheel Lock Right- STDS4032R

25a. Wheel Lock Tip- STDS4011

26. Frame Bumper- STDS000026

27. Footrest Bumper- STDS1026

28. Calf Pad Mtg. Plate & Screw- [1]

29. Adjustment Lever Left- STDS2A536L

Adjustment Lever Right- STDS2A536R

30. Elevating Leg Rest (pr)- HDEL R

N/S Footrest- HDSF

[1] The part is no longer available.

[Diagrams](#)

[Home](#)

Sentra HD_rev. 04_28_2021